

## Application guide

IODOSORB Cadexomer Iodine Powder promotes wound bed preparation by removing barriers to healing from the wound.\*<sup>1-8</sup> With a simple and easy application and removal,<sup>1,6</sup> IODOSORB Dressing is clinically proven to treat infection in chronic wounds.<sup>1,8-11</sup>

## Applying IODOSORB Powder



- Wash hands thoroughly before and after use
- Using gloved hands, clean the wound and the surrounding area with either a gentle stream of water or saline
- **DO NOT DRY** the wound surface



- Apply IODOSORB Powder to the wound surface to form a layer 3mm deep, which conforms to the shape of the wound



- Cover the wound completely with an appropriate secondary dressing.
- Apply compression bandaging if appropriate
- Any remaining IODOSORB Powder should be discarded due to loss of sterility

## Changing IODOSORB Powder



- IODOSORB Powder should be changed when it has become saturated with wound fluid and all the iodine has been released
- This is indicated by loss of colour, usually two to three times a week
- If the wound is discharging heavily, daily changes may be needed



- If necessary, soak the dressing for a few minutes, then remove
- Gently remove the IODOSORB Powder using a stream of water or saline
- Gently blot any excess fluid, leaving the wound surface slightly moist, before reapplying IODOSORB Powder



- Gently blot any excess fluid, leaving the wound surface slightly moist, before reapplying IODOSORB Powder

\*e.g. slough, pus, debris, and biofilm.

## IODOSORB<sup>◇</sup> Cadexomer Iodine Powder

For detailed product information, including indications for use, contraindications, precautions and warnings, please consult the product's Instructions for Use (IFU) prior to use.

Advanced Wound Management  
Smith & Nephew Medical Ltd  
Croxley Park, Building 5,  
Lakeside, Hatters Lane, Watford,  
Hertfordshire WD18 8YE, UK

[www.smith-nephew.com](http://www.smith-nephew.com)

T +44 (0) 1923 477100  
F +44 (0) 1923 477101

◇Trademark of Smith+Nephew  
All Trademarks acknowledged  
©April 2023 Smith+Nephew  
AWM-AWC-39439 | GMC1302



### IODOSORB Cadexomer Iodine Powder

Code	Size	Carton
66001286	7 x 3g	7

**References:** 1. Skog E, Arnesjo B, Troeng T, et al. *British Journal of Dermatology*. 1983;109:77-83. 2. Troeng T, Skog E, Arnesjo B, et al. Stuttgart: Schattauer Verlag; 1983. 3. Holloway GA, Johansen KH, Barnes RW, Pierce GE. *The Western Journal of Medicine*. 1989;151(1):35-38. 4. Malone M, Johani K, Jensen SO, et al. *J Antimicrob Chemother*. 2017;72(7):2093-2101. 5. Hansson C, Persson L-M, Stenquist B, et al. *International Journal of Dermatology*. 1998;37(390-396). 6. Moberg S, Hoffman L, Grennert M-L, Holst A. *Journal of American Geriatrics Society*. 1983;31(6):462-465. 7. Harcup JW, Saul PA. *The British Journal of Clinical Practise*. 1986;40(9):360-364. 8. Lantis JC. Paper presented at: WUWHs; 2016; Florence. 9. Hillstrom L. *Acta Chir Scand Suppl*. 1988;544:53-56. 10. Schwartz JA, Lantis JC 2nd, Gendics C, et al. *Int Wound J*. 2013;10(2):193-199. 11. Malone M, Schwarzer S, Radzieta M, et al. *Int Wound J*. 2019;16(6):1477-1486.