




1. Pass suture through suture threader loop.

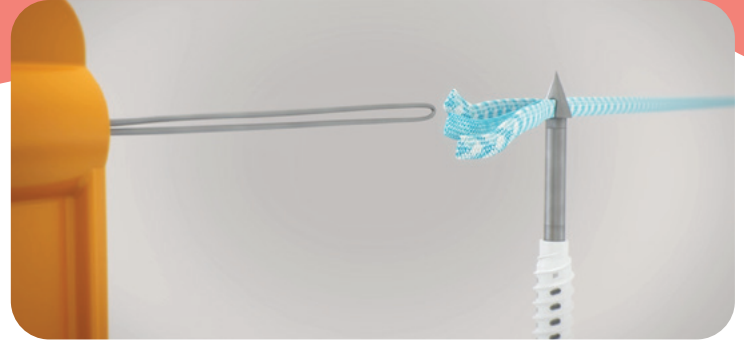

2. Pull suture threader to load implant with suture.  
Repeat 1 + 2 until all desired suture is loaded.




3. Locate insertion site. Prepare hole with recommended hole prep device listed in IFU.




4. Introduce device into joint and remove any slack in suture prior to insertion.




5. Insert distal implant into bone until proximal implant's first thread is positioned partially within the bone. Tension sutures to ensure appropriate tissue compression. Cleat sutures if desired. Verify the proximal anchor's first thread is positioned partially within the bone.

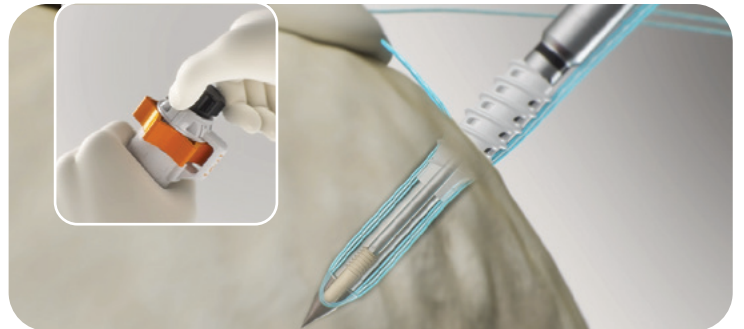

6. Descend internal plug by rotating smaller, most proximal knob. Rotate clockwise until a minimum of one click is heard.

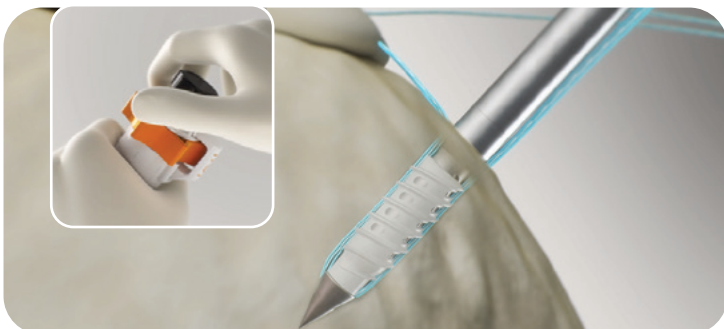

7. Turn the larger, more distal knob clockwise to advance the proximal implant until the laser mark is flush with the surface of the bone. To remove inserter from the joint pull back on the handle.

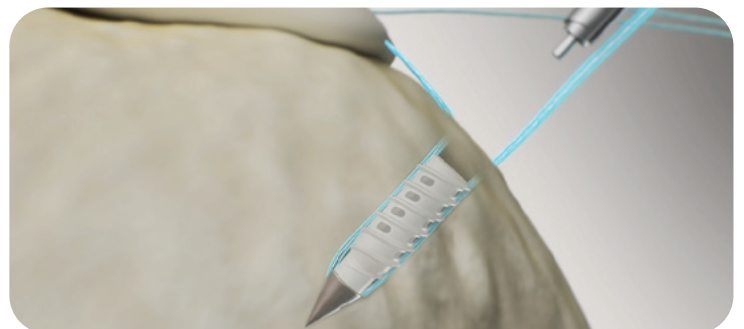

8. Pull back on inserter to remove.

## Ordering information

HEALICOIL <sup>®</sup> Knotless Suture Anchors	
Reference #	Description
72205137	HEALICOIL Knotless PK Suture Anchor, 5.0mm
72205138	HEALICOIL Knotless PK Suture Anchor, 5.0mm, Self-Tapping
72205308	HEALICOIL Knotless Suture Anchor Drill Bit, Spade Tip, 4.75mm, reusable
72201915	Tapered Awl, 3.8mm, reusable
72202621	Tapered Awl, 3.8mm, disposable
72203710	5.5mm Reusable Threaded Dilator
72203952	5.5mm Disposable Threaded Dilator

All photos and diagrams are for illustrative purposes only. Products not to scale.