

+ Using  
RAPID RHINO<sup>◇</sup>  
NASASTENT<sup>◇</sup>  
Dissolvable  
Nasal Dressing


 smith&nephew

RAPID RHINO<sup>◇</sup>  
NASASTENT<sup>◇</sup>  
Dissolvable Nasal Dressing

## NASASTENT Dissolvable Nasal Dressing

NASASTENT dressing is a dissolvable intranasal splint made from plant-based CarboxyMethyl Cellulose (CMC). As it absorbs nasal fluids, it turns into a hydrocolloidal gel that naturally drains from the nasal cavity.

### NASASTENT Dissolvable Nasal Dressing features

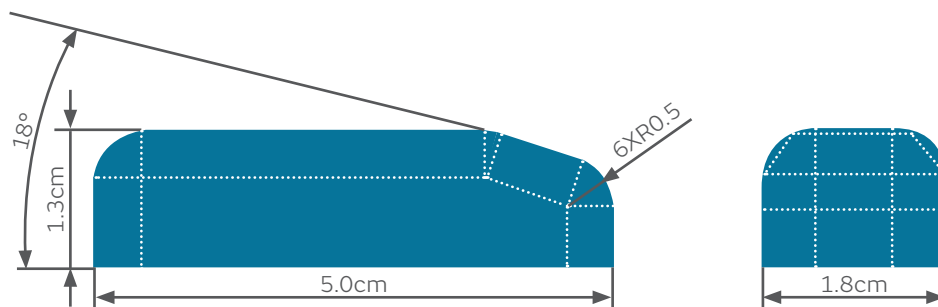

NASASTENT dressings tapered edge is designed to accommodate faster and easier placement.





Figure 1



Figure 2



Figure 3

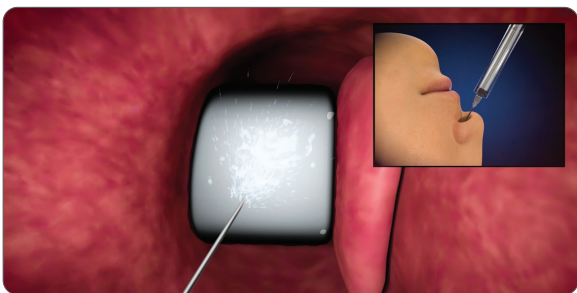


Figure 4


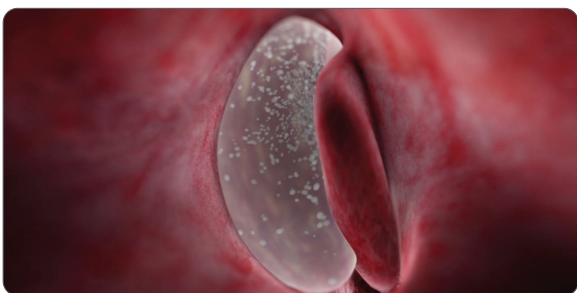


Figure 5

## Surgical technique


1. Immediately prior to insertion, peel open the NASASTENT dressing pouch and remove the device from the protective tray (**Figure 1**).

**NOTE:** NASASTENT dressing may be trimmed prior to insertion according to surgeons needs.

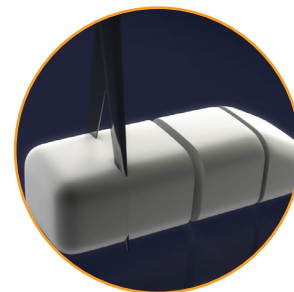

2. Using dry forceps, grasp the NASASTENT dressing (**Figure 2**).

**NOTE:** NASASTENT dressing does not require hydration prior to placement.

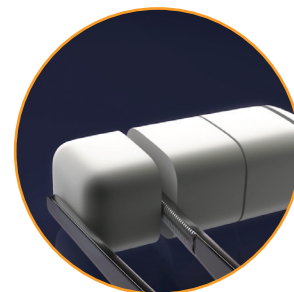

3. Place the NASASTENT dissolvable dressing into the surgical site (**Figure 3**).

4. After placement, NASASTENT dressing maybe hydrated as desired (**Figure 4**).

5. NASASTENT dressing will convert to a dense gel and dissolve through normal outflow. Saline nose spray may be used after placement of the NasaStent dressing (**Figure 5**).

**NOTE:** Do not excessively irrigate after placement of NASASTENT dressing or use excessive force while irrigating.

# Ordering information

## RAPID RHINO<sup>®</sup> NASASTENT<sup>®</sup> Dissolvable Nasal Dressing

Reference #	Description
RR1000	RAPID RHINO NASASTENT Dissolvable Nasal Dressing

## Additional instruction

To order the instruments used in this technique, call **+1 800 343 5717** in the U.S. or contact an authorized Smith+Nephew representative. Prior to performing this technique, consult the Instructions for Use documentation provided with individual components – including indications, contraindications, warnings, cautions and instructions.

**CAUTION:** U.S. Federal law restricts this device to sale by or on the order of a physician.

Learn more at [smith-nephew.com](http://smith-nephew.com)



Manufactured by:  
ArthroCare Corporation  
7000 West William  
Cannon Drive  
Austin, TX 78735, USA

[www.smith-nephew.com](http://www.smith-nephew.com)  
T +978 749 1000  
US Customer Service:  
+1 800 343 5717

Distributed by:  
Sports Medicine  
Smith & Nephew, Inc.  
150 Minuteman Road  
Andover, MA 01810

<sup>®</sup>Trademark of Smith+Nephew.  
All trademarks acknowledged.  
©2021 Smith+Nephew. All rights reserved. Printed in USA. 32927 (60083) V1 10/21