

Stem and Femoral Ball Head Combinations

POLARSTEM®

Smith+Nephew

Lit. No. 04758 Ed. 10/24 V12


The following matrix shows the compatibility of Femoral Ball Heads and Stems distributed by Smith+Nephew and used in Total Hip Arthroplasty. Each compatibility statement is based on the current evidence about the technical safety of the corresponding product combination. Combinations listed in the matrix may not be approved in all individual markets or geographies. The information contained in this matrix does not supersede the instructions for use in force in the markets where the products are being used. Please refer to your local Smith+Nephew representative to confirm the approval status in your country or region if you have questions about how Smith+Nephew products can be used.

Compatibility status:

✓	Combination is approved by Smith+Nephew
✗	Combination is not approved by Smith+Nephew

Legal Manufacturer:

Smith+Nephew Orthopaedics AG, Theilerstrasse 1A, 6300 Zug, Switzerland
Smith+Nephew Inc. 1450 Brooks Road, Memphis, Tennessee 38116, USA

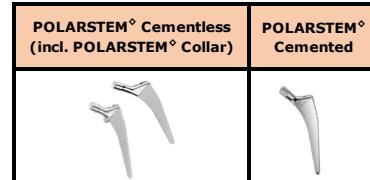

Femoral Ball Head	Diameter	Neck length				SAP no.	Size std. 01 ³	All other sizes ²	All sizes ²
OXINIUM®	22	+0	+4	+8	+12	713422-00/-04/-08/-12	✓	✓	✓
	26	+0	+4	+8	+12	713426-00/-04/-08/-12	✗	✗	✗
	28	-3	+0	+4	+8	71342803	✓	✓	✓
	30	+0	+4	+8	+12	713428-00/-04/-08/-12	✓	✓	✓
	32	-3	+0	+4	+8	713428-00/-04/-08/-12	✓	✓	✓
	36	-3	+0	+4	+8	71343216	✓	✓	✓
	40 ¹	-4	+0	+4	+8	71343603	✓	✓	✓
	44 ¹	-4	+0	+4	+8	71342340 & sleeve: 71344245	✓	✓	✓
	22	+0	+4	+8	+12	71342340 & sleeve: 713442-47/-48/-49	✓	✓	✓
	26	+0	+4	+8	+12	71342344 & sleeve: 71344245	✓	✓	✓
CoCrMo	22	+0	+4	+8	+12	713022-00/-04/-08/-12	✓	✓	✓
	26	+0	+4	+8	+12	713026-00/-04/-08/-12	✗	✗	✗
	28	-3	+0	+4	+8	71302803	✓	✓	✓
	30	+0	+4	+8	+12	713028-00/-04/-08/-12	✓	✓	✓
	32	-3	+0	+4	+8	71303216	✓	✓	✓
	36	-3	+0	+4	+8	71303603	✓	✓	✓
	40 ¹	-4	+0	+4	+8	713036-00/-04/-08/-12	✓	✓	✓
	44 ¹	-4	+0	+4	+8	71342640 & sleeve: 71344245	✓	✓	✓
	22	+0	+4	+8	+12	71342640 & sleeve: 713442-47/-48/-49	✓	✓	✓
	26	+0	+4	+8	+12	71342644 & sleeve: 71344245	✓	✓	✓
Stainless Steel	22	+0	+4	+8	+12	712922-00/-04/-08/-12	✓	✓	✓
	26	+0	+4	+8	+12	712926-00/-04/-08/-12	✗	✗	✗
	28	-3	+0	+4	+8	712928-03/-00/-04/-08/-12/-16	✓	✓	✓
	32	-3	+0	+4	+8	712932-03/-00/-04/-08/-12/-16	✓	✓	✓
BIOLOX® delta	32	+0	+4	+8		765391-60/-61/-62	✓	✓	✓
	36	+0	+4	+8	+12	765391-65/-66/-67/-53	✓	✓	✓
	40	+0	+4	+8		713460-04/-05/-06	✓	✓	✓
	28	S	M	L		750074-51/-52/-53	✓	✓	✓
	32	S	M	L	XL	750074-60/-61/-62/-63	✓	✓	✓
BIOLOX® OPTION	36	S	M	L	XL	750074-48/-49/-50/-47	✓	✓	✓
	28 ¹	S	M	L	XL	75103513 & sleeve: 751035-17/-18/-19/-20	✓	✓	✓
	32 ¹	S	M	L	XL	75103514 & sleeve: 751035-17/-18/-19/-20	✓	✓	✓
	36 ¹	S	M	L	XL	75103515 & sleeve: 751035-17/-18/-19/-20	✓	✓	✓
	40 ¹	S	M	L	XL	75103516 & sleeve: 751035-17/-18/-19/-20	✓	✓	✓

¹ These femoral heads must only be used together with the corresponding Ti-sleeves.

² Includes all POLARSTEM stem types (Standard, Lateral, Valgus and Medial). Valgus and Medial are only available for the cementless collarless type. The available sizes and catalogue numbers of the stems can be found in the corresponding surgical technique.

³ SAP no.: 75100462 and 75018399.

This compatibility matrix can only be accessed online via Smith+Nephew's website ifu.smith-nephew.com.

It is the responsibility of the user to consult the Smith+Nephew website to ensure the currency of compatibility information.

Stem and Femoral Ball Head Combinations
SL-PLUS[°], SL-PLUS[°] MIA and SLR-PLUS[°]

Smith+Nephew

Lit. No. 04758 Ed. 10/24 V12

The following matrix shows the compatibility of Femoral Ball Heads and Stems distributed by Smith+Nephew and used in Total Hip Arthroplasty. Each compatibility statement is based on the current evidence about the technical safety of the corresponding product combination.

Combinations listed in the matrix may not be approved in all individual markets or geographies. The information contained in this matrix does not supersede the instructions for use in force in the markets where the products are being used. Please refer to your local Smith+Nephew representative to confirm the approval status in your country or region if you have questions about how Smith+Nephew products can be used.

Compatibility status:

✓	Combination is approved by Smith+Nephew
✗	Combination is not approved by Smith+Nephew

Legal Manufacturer:

Smith+Nephew Orthopaedics AG, Theilerstrasse 1A, 6300 Zug, Switzerland
Smith+Nephew Inc. 1450 Brooks Road, Memphis, Tennessee 38116, USA

Femoral Ball Head	Diameter	Neck length				SAP no.	SL-PLUS [°]	SL-PLUS [°] Japan	SL-PLUS [°] MIA	SLR-PLUS [°]	
OXINIUM [°]	22	+0	+4	+8	+12	713422-00/-04/-08/-12	✓	✓	✓	✓	
	26	+0	+4	+8	+12	713426-00/-04/-08/-12	✗	✗	✗	✗	
	28	-3	+0	+4	+8	+12	713428-03/-00/-04/-08/-12	✓	✓	✓	✓
	32	-3	+0	+4	+8	+12	71342816	✗	✗	✗	✓
	36	-3	+0	+4	+8	+12	713432-03/-00/-04/-08/-12	✓	✓	✓	✓
	40 ¹	-4	+0	+4	+8		71343216	✗	✗	✗	✓
	44 ¹	-4	+0	+4	+8		713436-03/-00/-04/-08/-12	✓	✓	✓	✓
	22	+0	+4	+8	+12	71342340 & sleeve; 713442-45/-47/-48/-49	✓	✓	✓	✓	
CoCrMo	26	+0	+4	+8	+12	71342344 & sleeve; 713442-45/-47/-48/-49	✓	✓	✓	✓	
	28	-3	+0	+4	+8	+12	713022-00/-04/-08/-12	✓	✓	✓	✓
	32	-3	+0	+4	+8	+12	713026-00/-04/-08/-12	✓	✓	✓	✓
	36	-3	+0	+4	+8	+12	713028-03/-00/-04/-08/-12	✓	✓	✓	✓
	40 ¹	-4	+0	+4	+8		713032-03/-00/-04/-08/-12	✓	✓	✓	✓
	44 ¹	-4	+0	+4	+8		71303216	✓	✓	✓	✓
	22	+0	+4	+8	+12	713036-03/-00/-04/-08/-12	✓	✓	✓	✓	
	26	+0	+4	+8	+12	71342640 & sleeve; 713442-45/-47/-48/-49	✓	✓	✓	✓	
Stainless Steel	28	-3	+0	+4	+8	+12	71342644 & sleeve; 713442-45/-47/-48/-49	✓	✓	✓	✓
	32	-3	+0	+4	+8	+12	712922-00/-04/-08/-12	✓	✓	✓	✓
	26	+0	+4	+8	+12		712926-00/-04/-08/-12	✓	✓	✓	✓
	28	-3	+0	+4	+8	+12	712928-03/-00/-04/-08/-12/-16	✓	✓	✓	✓
BIOLOX® delta	32	-3	+0	+4	+8	+12	712932-03/-00/-04/-08/-12/-16	✓	✓	✓	✓
	32	+0	+4	+8			765391-50/-61/-62	✓	✓	✓	✓
	36	+0	+4	+8	+12		765391-65/-66/-67/-53	✓	✓	✓	✓
	40	+0	+4	+8			713460-04/-05/-06	✓	✓	✓	✓
	28	S	M	L			750074-51/-52/-53	✓	✓	✓	✓
BIOLOX® OPTION	32	S	M	L	XL		750074-60/-61/-62/-63	✓	✓	✓	✓
	36	S	M	L	XL		750074-48/-49/-50/-47	✓	✓	✓	✓
	28 ¹	S	M	L	XL		75103513 & sleeve; 751035-17/-18/-19/-20	✓	✓	✓	✓
	32 ¹	S	M	L	XL		75103514 & sleeve; 751035-17/-18/-19/-20	✓	✓	✓	✓
	36 ¹	S	M	L	XL		75103515 & sleeve; 751035-17/-18/-19/-20	✓	✓	✓	✓
	40 ¹	S	M	L	XL		75103516 & sleeve; 751035-17/-18/-19/-20	✓	✓	✓	✓

¹ These femoral heads must only be used together with the corresponding Ti-sleeves.

² The available sizes and catalogue numbers of the stems can be found in the corresponding surgical technique.

³ SAP no. 75000172

This compatibility matrix can only be accessed online via Smith+Nephew's website ifu.smith-nephew.com.

It is the responsibility of the user to consult the Smith+Nephew website to ensure the currency of compatibility information.

◊Trademark of Smith+Nephew. All Trademarks acknowledged.

Page 2/7

The following matrix shows the compatibility of Femoral Ball Heads and Stems distributed by Smith+Nephew and used in Total Hip Arthroplasty. Each compatibility statement is based on the current evidence about the technical safety of the corresponding product combination. Combinations listed in the matrix may not be approved in all individual markets or geographies. The information contained in this matrix does not supersede the instructions for use in force in the markets where the products are being used. Please refer to your local Smith+Nephew representative to confirm the approval status in your country or region if you have questions about how Smith+Nephew products can be used.

Compatibility status:

✓	Combination is approved by Smith+Nephew	SBG [◊]
	Combination is not approved by Smith+Nephew	

Legal Manufacturer:

	Smith+Nephew Orthopaedics AG, Theilerstrasse 1A, 6300 Zug, Switzerland	SBG [◊]
	Smith+Nephew Inc. 1450 Brooks Road, Memphis, Tennessee 38116, USA	
	CeramTec GmbH, CeramTec Platz 1-9, 73207 Plochingen, Germany	

Femoral Ball Head	Diameter	Neck length				SAP no.	All sizes ²
OXINIUM [◊]	22	+0	+4	+8	+12	713422-00/-04/-08/-12	✓
	26	+0	+4	+8	+12	713426-00/-04/-08/-12	
	28	-3				71342803	✓
		+0	+4	+8	+12	713428-00/-04/-08/-12	✓
						71342816	
	32	-3				71343203	✓
		+0	+4	+8	+12	713432-00/-04/-08/-12	✓
						71343216	
	36	-3				71343603	✓
		+0	+4	+8	+12	713436-00/-04/-08/-12	✓
CoCrMo	40 ¹	-4				71342340 & sleeve: 71344245	✓
		+0	+4	+8		71342340 & sleeve: 71344247/-48/-49	✓
	44 ¹	-4				71342344 & sleeve: 71344245	✓
		+0	+4	+8		71342344 & sleeve: 71344247/-48/-49	✓
	22	+0	+4	+8	+12	713022-00/-04/-08/-12	✓
	26	+0	+4	+8	+12	713026-00/-04/-08/-12	
	28	-3				71302803	✓
		+0	+4	+8	+12	713028-00/-04/-08/-12	✓
						71302816	
	32	-3				71303203	✓
Stainless Steel		+0	+4	+8	+12	713032-00/-04/-08/-12	
						71303216	
	36	-3				71303603	✓
		+0	+4	+8	+12	713036-00/-04/-08/-12	✓
	40 ¹	-4				71342640 & sleeve: 71344245	✓
BIOLOX® delta		+0	+4	+8		71342640 & sleeve: 71344247/-48/-49	✓
	44 ¹	-4				71342644 & sleeve: 71344245	✓
		+0	+4	+8		71342644 & sleeve: 71344247/-48/-49	✓
	22	+0	+4	+8	+12	712922-00/-04/-08/-12	
	26	+0	+4	+8	+12	712926-00/-04/-08/-12	
BIOLOX® OPTION	28	-3	+0	+4	+8	712928-03/-00/-04/-08/-12	
						71292816	
	32	-3	+0	+4	+12	712932-03/-00/-04/-08/-12	
						71293216	
	36	-3	+0	+4	+12		
BIOLOX® delta	32	+0	+4	+8		765391-60/-61/-62	✓
	36	+0	+4	+8	+12	765391-65/-66/-67/-53	✓
	40	+0	+4	+8		713460-04/-05/-06	✓
	28	S	M	L		750074-51/-52/-53	✓
	32	S	M	L	XL	750074-60/-61/-62/-63	✓
BIOLOX® OPTION	36	S	M	L	XL	750074-48/-49/-50/-47	✓
	28 ¹	S	M	L	XL	75103513 & sleeve: 751035-17/-18/-19/-20	✓
	32 ¹	S	M	L	XL	75103514 & sleeve: 751035-17/-18/-19/-20	✓
	36 ¹	S	M	L	XL	75103515 & sleeve: 751035-17/-18/-19/-20	✓
	40 ¹	S	M	L	XL	75103516 & sleeve: 751035-17/-18/-19/-20	✓

¹ These femoral heads must only be used together with the corresponding Ti-sleeves.

² The available sizes and catalogue numbers of the stems can be found in the corresponding surgical technique.

This compatibility matrix can only be accessed online via Smith+Nephew's website ifu.smith-nephew.com.

It is the responsibility of the user to consult the Smith+Nephew website to ensure the currency of compatibility information.

Stem and Femoral Ball Head Combinations
NANOS° and PROXYPLUS°

Smith+Nephew

Lit. No. 04758 Ed. 10/24 V12

The following matrix shows the compatibility of Femoral Ball Heads and Stems distributed by Smith+Nephew and used in Total Hip Arthroplasty. Each compatibility statement is based on the current evidence about the technical safety of the corresponding product combination. Combinations listed in the matrix may not be approved in all individual markets or geographies. The information contained in this matrix does not supersede the instructions for use in force in the markets where the products are being used. Please refer to your local Smith+Nephew representative to confirm the approval status in your country or region if you have questions about how Smith+Nephew products can be used.

Compatibility status:

✓	Combination is approved by Smith+Nephew
✗	Combination is not approved by Smith+Nephew
	Combination under consideration (do not combine until compatibility is confirmed)

Legal Manufacturer:

Smith+Nephew Orthopaedics AG, Theilerstrasse 1A, 6300 Zug, Switzerland
OHST medical technology AG, Grünauer Fenn 3, 14712 Rathenow, Germany
Smith+Nephew Inc. 1450 Brooks Road, Memphis, Tennessee 38116, USA



Femoral Ball Head	Diameter	Neck length				SAP no.	All sizes ²
OXINIUM°	22	+0	+4	+8	+12	713422-00/-04 713422-08/-12	✓
	26	+0	+4	+8	+12	713426-00/-04/-08/-12	✗
	28	-3	+0	+4		71342803	✓
				+8	+12	713428-00/-04 713428-08/-12	✓
					+16	71342816	✗
	32	-3	+0	+4		71343203	✓
				+8	+12	713432-00/-04 713432-08/-12	✓
					+16	71343216	✗
	36	-3	+0	+4		71343603	✓
				+8	+12	713436-00/-04 713436-08/-12	✓
CoCrMo	40 ¹	-4	+0	+4	+8	71342340 & sleeve; 713442-45/-47/-48/-49	✗
	44 ¹	-4	+0	+4	+8	71342344 & sleeve; 713442-45/-47/-48/-49	✗
	22	+0	+4			713022-00/-04	✓
				+8	+12	713022-08/-12	✓
	26	+0	+4	+8	+12	713026-00/-04/-08/-12	✗
	28	-3	+0	+4		71302803	✓
				+8	+12	713028-00/-04 713028-08/-12	✓
					+16	71302816	✗
	32	-3	+0	+4		71303203	✓
				+8	+12	713032-00/-04 713032-08/-12	✓
Stainless Steel	36	-3	+0	+4		71303603	✓
				+8	+12	713036-00/-04 713036-08/-12	✓
	40 ¹	-4	+0	+4	+8	71342640 & sleeve; 713442-45/-47/-48/-49	✗
	44 ¹	-4	+0	+4	+8	71342644 & sleeve; 713442-45/-47/-48/-49	✗
	22	+0	+4	+8	+12	712922-00/-04/-08/-12	✗
	26	+0	+4	+8	+12	712926-00/-04/-08/-12	✗
	28	-3	+0	+4	+8	712928-03/-00/-04/-08/-12/-16	✗
	32	-3	+0	+4	+8	712932-03/-00/-04/-08/-12/-16	✗
	36	+0	+4	+8		765391-60/-61/-62	✓
	40	+0	+4	+8		765391-65/-66/-67	✓
BIOLOX® delta	32	+0	+4	+8		76539153	✓
	36	+0	+4	+8		713460-04/-05/-06	✗
	40	+0	+4	+8		750074-51/-52/-53	✓
	28	S	M	L		750074-60/-61/-62	✓
	32	S	M	L	XL	75007463	✓
BIOLOX® OPTION	36	S	M	L		750074-48/-49/-50	✓
					XL	75007447	✓
	28 ¹	S	M	L	XL	75103513 & sleeve; 751035-17/-18/-19/-20	✗
	32 ¹	S	M	L	XL	75103514 & sleeve; 751035-17/-18/-19/-20	✗
	36 ¹	S	M	L	XL	75103515 & sleeve; 751035-17/-18/-19/-20	✗
	40 ¹	S	M	L	XL	75103516 & sleeve; 751035-17/-18/-19/-20	✗

¹ These femoral heads must only be used together with the corresponding Ti-sleeves.

² The available sizes and catalogue numbers of the stems can be found in the corresponding surgical technique.

This compatibility matrix can only be accessed online via Smith+Nephew's website ifu.smith-nephew.com.

It is the responsibility of the user to consult the Smith+Nephew website to ensure the currency of compatibility information.

Stem and Femoral Ball Head Combinations
SYNERGY[◊], SMF[◊] and ANTHOLOGY[◊]

Smith+Nephew

Lit. No. 04758 Ed. 10/24 V12

The following matrix shows the compatibility of Femoral Ball Heads and Stems distributed by Smith+Nephew and used in Total Hip Arthroplasty. Each compatibility statement is based on the current evidence about the technical safety of the corresponding product combination.


Combinations listed in the matrix may not be approved in all individual markets or geographies. The information contained in this matrix does not supersede the instructions for use in force in the markets where the products are being used. Please refer to your local Smith+Nephew representative to confirm the approval status in your country or region if you have questions about how Smith+Nephew products can be used.

Compatibility status:

✓	Combination is approved by Smith+Nephew
✗	Combination is not approved by Smith+Nephew

Legal Manufacturer:

Smith+Nephew Orthopaedics AG, Thelerstrasse 1A, 6300 Zug, Switzerland
Smith+Nephew Inc. 1450 Brooks Road, Memphis, Tennessee 38116, USA

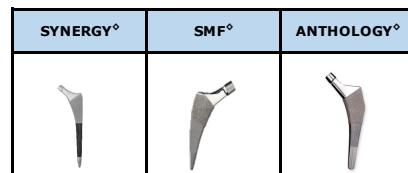

Femoral Ball Head	Diameter	Neck length				SAP no.	All sizes ²	All sizes ²	All sizes ²
OXINIUM [◊]	22	+0	+4	+8	+12	713422-00/-04/-08/-12	✓	✓	✓
	26	+0	+4	+8	+12	713426-00/-04/-08/-12	✓	✓	✓
	28	-3	+0	+4	+8	713428-03/-00/-04/-08/-12/-16	✓	✓	✓
	32	-3	+0	+4	+8	713432-03/-00/-04/-08/-12/-16	✓	✓	✓
	36	-3	+0	+4	+8	713436-03/-00/-04/-08/-12	✓	✓	✓
	40 ¹	-4	+0	+4	+8	71342340 & sleeve: 713442-45/-47/-48/-49	✓	✓	✓
	44 ¹	-4	+0	+4	+8	71342344 & sleeve: 713442-45/-47/-48/-49	✓	✓	✓
	22	+0	+4	+8	+12	713022-00/-04/-08/-12	✓	✓	✓
CoCrMo	26	+0	+4	+8	+12	713026-00/-04/-08/-12	✓	✓	✓
	28	-3	+0	+4	+8	713028-03/-00/-04/-08/-12/-16	✓	✓	✓
	32	-3	+0	+4	+8	713032-03/-00/-04/-08/-12/-16	✓	✓	✓
	36	-3	+0	+4	+8	713036-03/-00/-04/-08/-12	✓	✓	✓
	40 ¹	-4	+0	+4	+8	71342640 & sleeve: 713442-45/-47/-48/-49	✓	✓	✓
	44 ¹	-4	+0	+4	+8	71342644 & sleeve: 713442-45/-47/-48/-49	✓	✓	✓
	22	+0	+4	+8	+12	712922-00/-04/-08/-12	✗	✗	✗
	26	+0	+4	+8	+12	712926-00/-04/-08/-12	✗	✗	✗
Stainless Steel	28	-3	+0	+4	+8	712928-03/-00/-04/-08/-12/-16	✗	✗	✗
	32	-3	+0	+4	+8	712932-03/-00/-04/-08/-12/-16	✗	✗	✗
	32	+0	+4	+8	+12	765391-60/-61/-62	✓	✓	✓
	36	+0	+4	+8	+12	765391-65/-66/-67/-53	✓	✓	✓
BIOLOX [®] delta	40	+0	+4	+8	+12	713460-04/-05/-06	✓	✓	✓
	28	S	M	L		750074-51/-52/-53	✓	✓	✓
	32	S	M	L	XL	750074-60/-61/-62/-63	✓	✓	✓
	36	S	M	L	XL	750074-48/-49/-50/-47	✓	✓	✓
BIOLOX [®] OPTION	28 ¹	S	M	L	XL	75103513 & sleeve: 751035-17/-18/-19/-20	✓	✓	✓
	32 ¹	S	M	L	XL	75103514 & sleeve: 751035-17/-18/-19/-20	✓	✓	✓
	36 ¹	S	M	L	XL	75103515 & sleeve: 751035-17/-18/-19/-20	✓	✓	✓
	40 ¹	S	M	L	XL	75103516 & sleeve: 751035-17/-18/-19/-20	✓	✓	✓

¹ These femoral heads must only be used together with the corresponding Ti-sleeves.

² The available sizes and catalogue numbers of the stems can be found in the corresponding surgical technique.

This compatibility matrix can only be accessed online via Smith+Nephew's website ifu.smith-nephew.com.

It is the responsibility of the user to consult the Smith+Nephew website to ensure the currency of compatibility information.

Stem and Femoral Ball Head Combinations
REDAPT°, SPECTRON°, and ECHELON°

Smith+Nephew

Lit. No. 04758 Ed. 10/24 V12


The following matrix shows the compatibility of Femoral Ball Heads and Stems distributed by Smith+Nephew and used in Total Hip Arthroplasty. Each compatibility statement is based on the current evidence about the technical safety of the corresponding product combination. Combinations listed in the matrix may not be approved in all individual markets or geographies. The information contained in this matrix does not supersede the instructions for use in force in the markets where the products are being used. Please refer to your local Smith+Nephew representative to confirm the approval status in your country or region if you have questions about how Smith+Nephew products can be used.

Compatibility status:

✓	Combination is approved by Smith+Nephew
✗	Combination is not approved by Smith+Nephew

Legal Manufacturer:

Smith+Nephew Orthopaedics AG, Theilerstrasse 1A, 6300 Zug, Switzerland
Smith+Nephew Inc. 1450 Brooks Road, Memphis, Tennessee 38116, USA

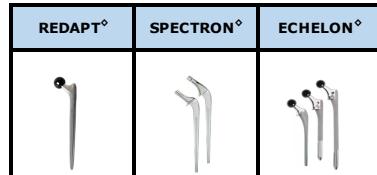

Femoral Ball Head	Diameter	Neck length			SAP no.	All sizes ²	All sizes ²	All sizes ²	
OXINIUM°	22	+0	+4	+8	+12	713422-00/-04/-08/-12	✓	✓	
	26	+0	+4	+8	+12	713426-00/-04/-08/-12	✓	✓	
	28	-3	+0	+4	+8	+12 +16	713428-03/-00/-04/-08/-12/-16	✓	✓
	32	-3	+0	+4	+8	+12 +16	713432-03/-00/-04/-08/-12/-16	✓	✓
	36	-3	+0	+4	+8	+12	713436-03/-00/-04/-08/-12	✓	✓
	40 ¹	-4	+0	+4	+8		71342340 & sleeve: 713442-45/-47/-48/-49	✓	✓
CoCrMo	44 ¹	-4	+0	+4	+8		71342344 & sleeve: 713442-45/-47/-48/-49	✓	✓
	22	+0	+4	+8	+12		713022-00/-04/-08/-12	✓	✓
	26	+0	+4	+8	+12		713026-00/-04/-08/-12	✓	✓
	28	-3	+0	+4	+8	+12 +16	713028-03/-00/-04/-08/-12/-16	✓	✓
	32	-3	+0	+4	+8	+12 +16	713032-03/-00/-04/-08/-12/-16	✓	✓
	36	-3	+0	+4	+8	+12	713036-03/-00/-04/-08/-12	✓	✓
Stainless Steel	40 ¹	-4	+0	+4	+8		71342640 & sleeve: 713442-45/-47/-48/-49	✓	✓
	44 ¹	-4	+0	+4	+8		71342644 & sleeve: 713442-45/-47/-48/-49	✓	✓
	22	+0	+4	+8	+12		712922-00/-04/-08/-12	✗	✗
	26	+0	+4	+8	+12		712926-00/-04/-08/-12	✗	✗
BIOLOX® delta	28	-3	+0	+4	+8	+12 +16	712928-03/-00/-04/-08/-12/-16	✗	✗
	32	-3	+0	+4	+8	+12 +16	712932-03/-00/-04/-08/-12/-16	✗	✗
	36	+0	+4	+8			765391-60/-61/-62	✓	✓
	40	+0	+4	+8			765391-65/-66/-67/-53	✓	✓
	44 ¹	+0	+4	+8			713460-04/-05/-06	✓	✓
	28 ¹	S	M	L			750074-51/-52/-53	✓	✓
BIOLOX® OPTION	32 ¹	S	M	L	XL		750074-60/-61/-62/-63	✓	✓
	36 ¹	S	M	L	XL		750074-48/-49/-50/-47	✓	✓
	28 ¹	S	M	L	XL		75103513 & sleeve: 751035-17/-18/-19/-20	✓	✓
	32 ¹	S	M	L	XL		75103514 & sleeve: 751035-17/-18/-19/-20	✓	✓
	36 ¹	S	M	L	XL		75103515 & sleeve: 751035-17/-18/-19/-20	✓	✓
	40 ¹	S	M	L	XL		75103516 & sleeve: 751035-17/-18/-19/-20	✓	✓

¹ These femoral heads must only be used together with the corresponding Ti-sleeves.

² The available sizes and catalogue numbers of the stems can be found in the corresponding surgical technique.

This compatibility matrix can only be accessed online via Smith+Nephew's website ifu.smith-nephew.com.

It is the responsibility of the user to consult the Smith+Nephew website to ensure the currency of compatibility information.

The following matrix shows the compatibility of Femoral Ball Heads and Stems distributed by Smith+Nephew and used in Total Hip Arthroplasty. Each compatibility statement is based on the current evidence about the technical safety of the corresponding product combination.

Combinations listed in the matrix may not be approved in all individual markets or geographies. The information contained in this matrix does not supersede the instructions for use in force in the markets where the products are being used. Please refer to your local Smith+Nephew representative to confirm the approval status in your country or region if you have questions about how Smith+Nephew products can be used.

Compatibility status:

✓	Combination is approved by Smith+Nephew		CPCS® CoCrMo
	Combination is not approved by Smith+Nephew		

Legal Manufacturer:

	Smith+Nephew Orthopaedics AG, Theilerstrasse 1A, 6300 Zug, Switzerland
	Smith+Nephew Inc. 1450 Brooks Road, Memphis, Tennessee 38116, USA

Femoral Ball Head	Diameter	Neck length				SAP no.	All sizes ²
OXINIUM®	22	+0	+4	+8	+12	713422-00/-04/-08/-12	✓
	26	+0	+4	+8	+12	713426-00/-04/-08/-12	✓
	28	-3	+0	+4	+8	713428-03/-00/-04/-08/-12/-16	✓
	32	-3	+0	+4	+8	713432-03/-00/-04/-08/-12/-16	✓
	36	-3	+0	+4	+8	713436-03/-00/-04/-08/-12	✓
	40 ¹	-4	+0	+4	+8	71342340 & sleeve: 713442-45/-47/-48/-49	✓
	44 ¹	-4	+0	+4	+8	71342344 & sleeve: 713442-45/-47/-48/-49	✓
CoCrMo	22	+0	+4	+8	+12	713022-00/-04/-08/-12	✓
	26	+0	+4	+8	+12	713026-00/-04/-08/-12	✓
	28	-3	+0	+4	+8	713028-03/-00/-04/-08/-12/-16	✓
	32	-3	+0	+4	+8	713032-03/-00/-04/-08/-12/-16	✓
	36	-3	+0	+4	+8	713036-03/-00/-04/-08/-12	✓
	40 ¹	-4	+0	+4	+8	71342640 & sleeve: 713442-45/-47/-48/-49	✓
	44 ¹	-4	+0	+4	+8	71342644 & sleeve: 713442-45/-47/-48/-49	✓
Stainless Steel	22	+0	+4	+8	+12	712922-00/-04/-08/-12	
	26	+0	+4	+8	+12	712926-00/-04/-08/-12	
	28	-3	+0	+4	+8	712928-03/-00/-04/-08/-12/-16	
	32	-3	+0	+4	+8	712932-03/-00/-04/-08/-12/-16	
BIOLOX® delta	32	+0	+4	+8		765391-60/-61/-62	✓
	36	+0	+4	+8	+12	765391-65/-66/-67/-53	✓
	40	+0	+4	+8		713460-04/-05/-06	✓
	28	S	M	L		750074-51/-52/-53	✓
	32	S	M	L	XL	750074-60/-61/-62/-63	✓
	36	S	M	L	XL	750074-48/-49/-50/-47	✓
BIOLOX® OPTION	28 ¹	S	M	L	XL	75103513 & sleeve: 751035-17/-18/-19/-20	✓
	32 ¹	S	M	L	XL	75103514 & sleeve: 751035-17/-18/-19/-20	✓
	36 ¹	S	M	L	XL	75103515 & sleeve: 751035-17/-18/-19/-20	✓
	40 ¹	S	M	L	XL	75103516 & sleeve: 751035-17/-18/-19/-20	✓

¹ These femoral heads must only be used together with the corresponding Ti-sleeves.

² The available sizes and catalogue numbers of the stems can be found in the corresponding surgical technique.

This compatibility matrix can only be accessed online via Smith+Nephew's website ifu.smith-nephew.com.

It is the responsibility of the user to consult the Smith+Nephew website to ensure the currency of compatibility information.