

Cleaning

Adherence to facility directives concerning hygiene is of prime importance. The instructions supplied with all cleaning agents as well as sterilization and/or disinfection units must be followed.

Cleaning of the RENASYS EZ outer casing should be done in accordance with the guidelines below:

- Wipe down the device surface, including vent on underside of unit, with a damp soft cloth.
- Use a low-level cleaning agent or disinfectant and ensure its compatibility with plastics.

Follow manufacturer's guidelines for use of cleaning agents.

- Dampen another soft cloth with clean water and use this to wipe down all surfaces to remove any excess solution.
- Dry with a separate soft cloth.
- Do not use plastic solvents or abrasives.

Do not immerse any part of RENASYS EZ in fluid or use an unnecessarily wet cloth. No fluids should be allowed to enter the device. If any liquids penetrate the device contact your local authorized provider.

Wound Management

Smith & Nephew, Inc.	Customer Care Center	www.smith-nephew.com
970 Lake Carillon Drive	1-800-876-1261	www.myrenasys.com
Suite 110	T 727-392-1261	
St. Petersburg, FL 33716	F 727-392-6914	
USA		