

## + Evidence in focus



Publication summary: Joy H, et al. *J Wound Care* (2015)\*

Smith+Nephew

# Use of ALLEVYN<sup>◇</sup> LIFE Foam Dressing with a programme of wound care training and education helped to reduce dressing changes and dressing costs, as well as release nurse visits compared with prior practice

## + Plus points

Compared with previous dressings, use of ALLEVYN LIFE Dressing:

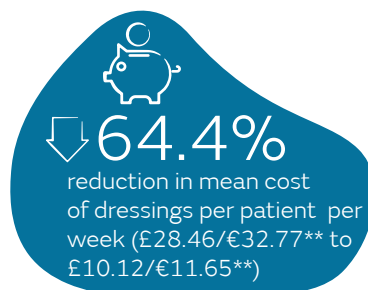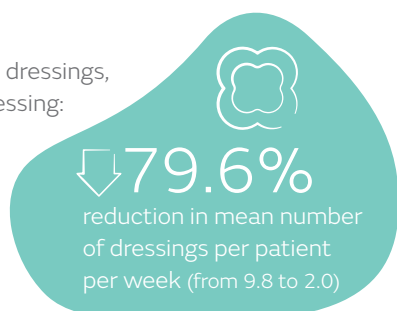


## Overview

- A targeted programme on wound care practice including use of ALLEVYN LIFE Dressing at one healthcare partnership in the UK
- Community nurses received training and education, including dressing selection and change frequency, over two weeks, followed by a one-week implementation phase
- A total of 37 patients were treated mostly within residential care (40.5%) or at home (43.2%)
- Most patients had chronic wounds of more than six weeks in duration (73.0%) and three (8.1%) had infected wounds

## Results

- Overall, 97.3% of patients had a reduction from baseline (prior practice) in dressing change visits after implementation of ALLEVYN LIFE Dressing
- Implementation of ALLEVYN LIFE Dressings reduced:
  - Mean frequency of dressing change visits per week by 50% (from 3.6 to 1.8)
  - Mean number of dressings per patient per dressing change by 47.6% (from 2.1 to 1.1)
  - Mean number of dressings per patient per week by 79.6% (from 9.8 to 2.0)
  - Patients requiring dressing changes  $\geq 3$  times per week (Figure)
- Mean dressing wear time increased by 2.3 days with ALLEVYN LIFE Dressing (from 2.5 days at baseline to 4.8 days post implementation)
- Use of ALLEVYN LIFE Dressing reduced dressing cost per patient per week by 64.4% (£18.34/€21.12\*\*) and was estimated to release 1,274 visits (658 hours) per week in a local healthcare practice of 708 patients (Figure)
  - 66,269 visits (34,239 hours) per year

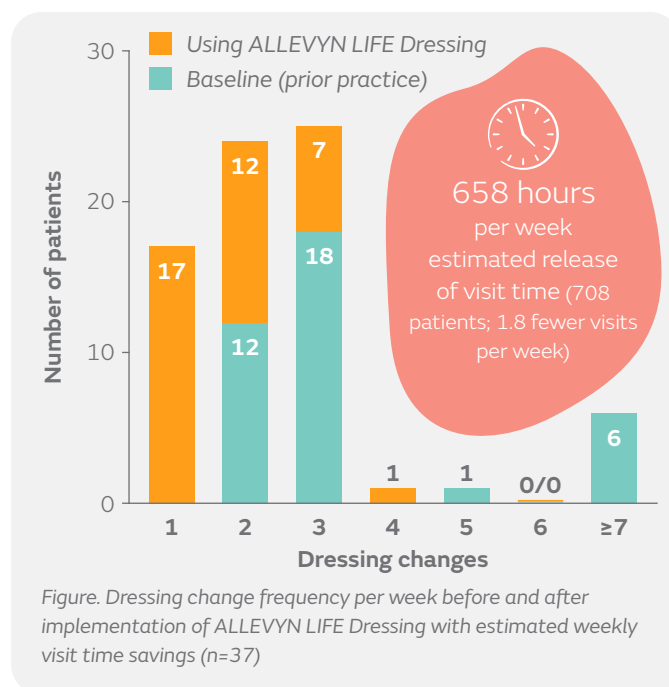

## Conclusions

Use of ALLEVYN LIFE Dressing with a training and education programme, helped to reduce dressing change frequency, weekly dressing costs per patient and nurse visits compared with prior practice, resulting in a substantial release of valuable nurse time.

## Citation

\*Joy H, Bielby A, Searle R. A collaborative project to enhance efficiency through dressing change practice. *J Wound Care*. 2015;24(7):314-317. Available at: [Journal of Wound Care](#)

For detailed product information, including indications for use, contraindications, precautions and warnings, please consult the product's applicable Instructions for Use (IFU) prior to use.

\*\*Costs based on a conversion rate of 1.1514 (GBP to EUR) in February 2024

Smith+Nephew, Croxley Park, Building 5, Hatters Lane, Watford, Hertfordshire WD18 8YE T +44 (0)1923 477100 F +44 (0)1923 477101  
Smith+Nephew, Kay Fiskers Plads 9, 2300 København S, Denmark T: +45 45 80 61 00. Smith+Nephew, Kroksläatts Fabriker 39, 431 37 Mölndal, Sweden, T: +46 31 74 65 800  
Smith+Nephew, Snarøyveien 36, 1364 Fornebu, Norway T: +47 66 84 20 20. Smith+Nephew, Lentäjantie 1, 01530 Vantaa, Finland T: +358 (0)207 866 333  
42448-en V1 0224. Published September 2020. ©2024 Smith+Nephew. <sup>◇</sup>Trademark of Smith+Nephew. All Trademarks acknowledged.

[www.smith-nephew.com/education](http://www.smith-nephew.com/education)