

Maintaining the power console

There are no user serviceable parts within the console, however there are several maintenance items to be observed.

CAUTION - Unplug the unit before starting any maintenance on the console.

The fan slots should be kept free from obstructions and periodically be inspected for excessive build-up of dust and/or foreign material. A vacuum cleaner should be used to clean the fan slots of any loose debris.

The slots on the bottom of the console should be kept free from obstructions and be periodically inspected for build up of dust and/or foreign material. A vacuum cleaner should be used to clean the slots of any loose debris.

The inside of the console clamp should be inspected periodically for build-up of deposits and/or debris. A damp cloth soaked in mild detergent can be used to remove material. Do not soak the inside of the clamp. Excessive fluid could cause damage.

At the end of the console's useful life, dispose of the console according to local regulations.

Cleaning the Power Console

Disconnect from electrical power source. Wipe down console and foot switch with a clean, damp cloth. Household all-purpose cleaners can be used to clean all surfaces. **DO NOT IMMERSE.** Do not sterilize or immerse in disinfectant solution.

CAUTION - Do not sterilize or immerse the Power Console or the Foot Switch.

Replacing the Power Cord

If the power cord is damaged, it can be removed from the Power Console. **FIRST, REMOVE THE PLUG FROM THE WALL SOCKET.** Do not pull on the cord itself. Remove plug from console, again without pulling on the cord. Contact a Smith & Nephew representative to order a replacement power cord.

Replacing the Foot Switch

If the foot switch is damaged, it should be removed from the Power Console. Contact a Smith & Nephew representative to order a replacement foot switch.

Wound Management

Smith & Nephew, Inc.	Customer Care Center	www.smith-nephew.com
970 Lake Carillon Drive	1-800-876-1261	www.versajet.info
Suite 110	T 727-392-1261	
St. Petersburg, FL 33716	F 727-392-6914	
USA		