



**Smith+Nephew**

## OXINIUM<sup>◇</sup> total knee arthroplasty (TKA)


An analysis of National Joint  
Registry for England, Wales and  
Northern Island (UK NJR) data

**+ Evidence in focus**


March 2026

# Key points

OXINIUM<sup>◇</sup> Knees (JOURNEY<sup>◇</sup> II, LEGION<sup>◇</sup> and GENESIS<sup>◇</sup> II TKA) demonstrated:

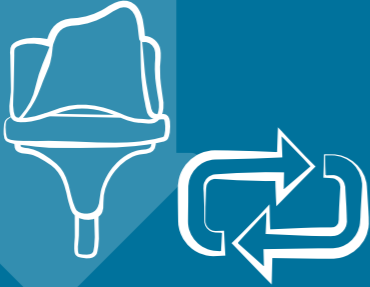

Numerically **lower risk of all-cause revision** versus class average and CoCr equivalent design<sup>2</sup>

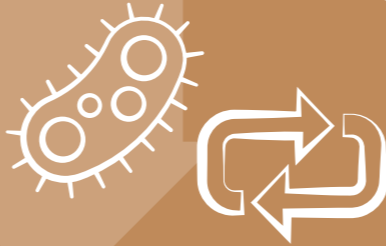

Significantly **reduced risk of revision for infection** versus the equivalent CoCr design (p<0.002)<sup>2</sup>



Significantly **reduced risk of revision from aseptic femoral loosening** (p<0.001)<sup>2</sup>

# Registry analysis and OXINIUM<sup>®</sup> TKA survivorship



## Context

Previous registry analyses demonstrated that **OXINIUM implants can approximately halve the risk of revision** for patients under the age of 65<sup>1</sup>

### Purpose

- New registry data<sup>2</sup> from the UK NJR was analysed to explore the influence of implant **material** and **design** on survivorship
- Registry analyses offer a real-world understanding of the influence of the OXINIUM technology and implant design on reducing the risk of all-cause revisions, aseptic loosening, infection and stability

### Considerations

The analyses performed:


- Included the data for all patients in the UK NJR who received a cemented TKA
- Adjusted for factors that might skew outcomes such as age, body mass index (BMI), complications
- Compared OXINIUM implants to CoCr implants with the same design and the class average

# OXINIUM<sup>◇</sup> JOURNEY<sup>◇</sup> II/ LEGION<sup>◇</sup>/GENESIS<sup>◇</sup> II implants: all-cause revision

The OXINIUM knee portfolio (with patella resurfacing) shows a numerically **lower risk of revision** when adjusting for population variables<sup>2</sup>

Cox Proportional Hazards model for All Reasons for Revision (95% CI). Adjustment includes Age Group, Gender, Year Cohort, BMI Group, ASA, Patella Used, PS/CR Constraint

OXINIUM vs CoCr	OXINIUM vs All Other Cemented TKR	CoCr vs All Other Cemented TKR
0.89 (0.76–1.04) p=0.131	0.90 (0.81–1.01) p=0.065	0.99 (0.92–1.06) p=0.724

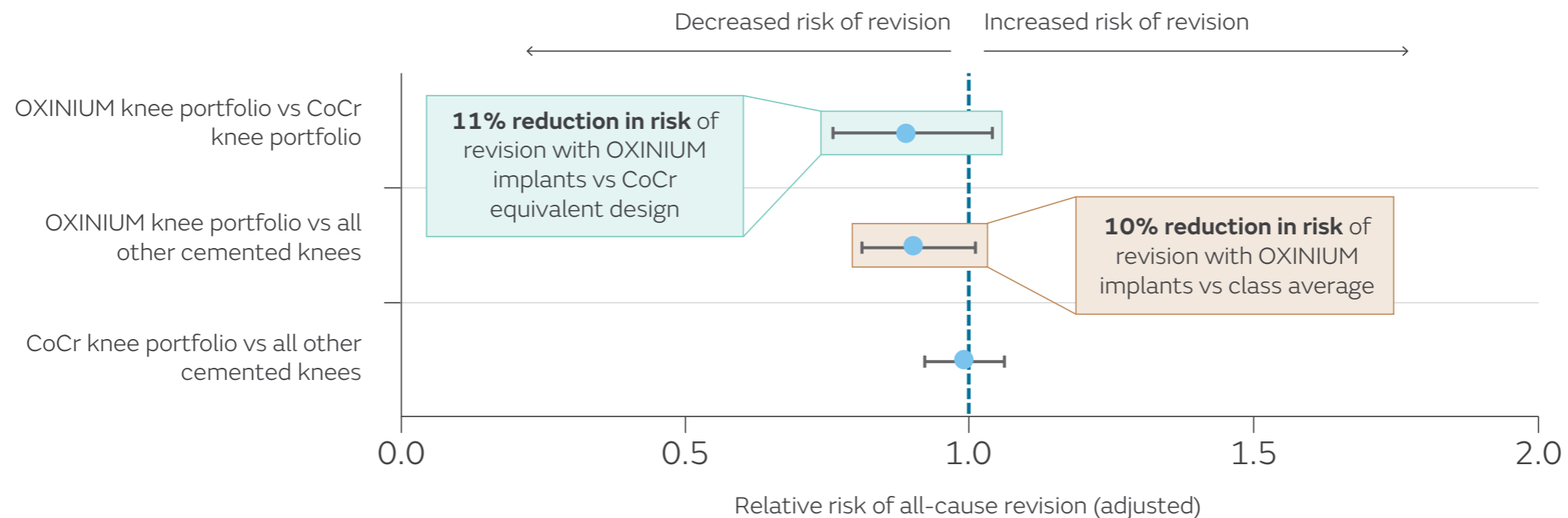


Abbreviations: ASA = American Society of Anaesthesiologists; CI = confidence interval; CoCr = cobalt-chromium; CR = cruciate-retaining; PS = posterior-stabilised; TKR = total knee replacement.

# OXINIUM<sup>◇</sup>JOURNEY<sup>◇</sup> II/ LEGION<sup>◇</sup>/GENESIS<sup>◇</sup> II implants: aseptic femoral loosening

OXINIUM knee implants demonstrates **significantly lower risk of revision for aseptic femoral loosening**, a leading cause for revision in TKAs,<sup>3,4</sup> compared to the class average in the UK NJR<sup>2</sup> when adjusting for population variables. This registry analysis suggests that implant material has a greater contribution to reducing aseptic loosening than implant design.

Cox Proportional Hazards model for Revision for Aseptic Loosening of Femoral Component (95% CI). Adjustment includes Age Group, Gender, Year Cohort, BMI Group, ASA, Patella Used, PS/CR Constraint

OXINIUM vs CoCr	OXINIUM vs All Other Cemented TKR	CoCr vs All Other Cemented TKR
0.37 (0.24–0.59) p<0.001	0.52 (0.38–0.71) p<0.001	0.99 (0.85–1.14) p=0.842

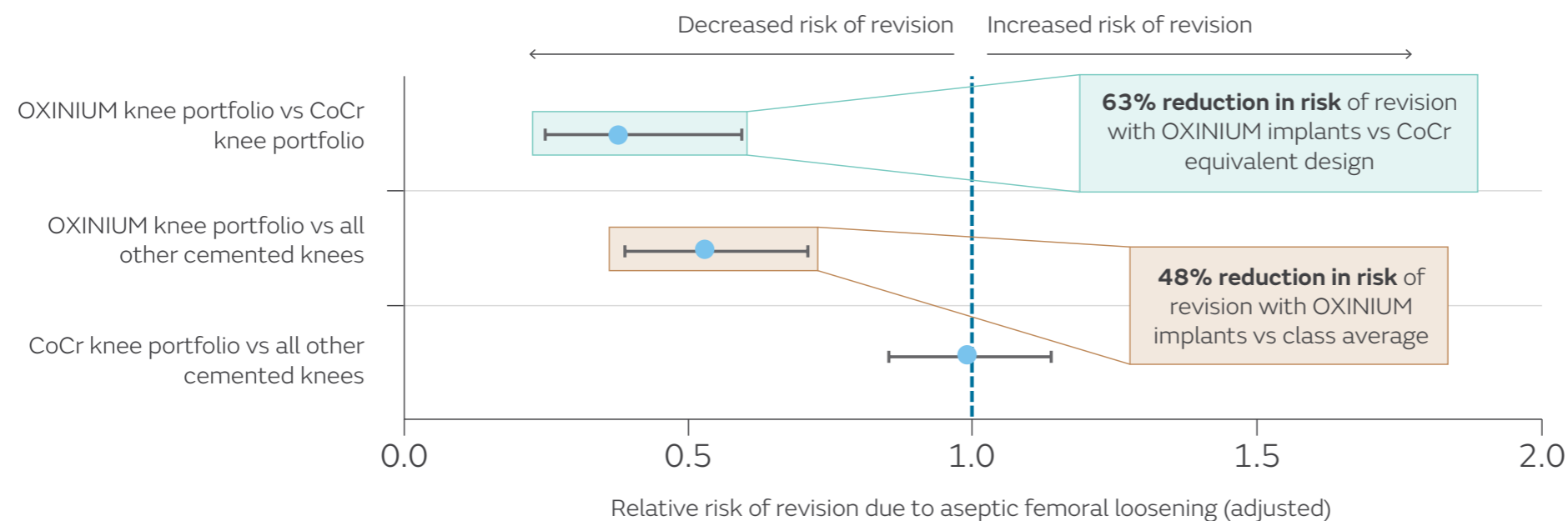

In vitro testing has repeatedly shown a lower pro-inflammatory response to OXINIUM Oxidized Zirconium compared to CoCr<sup>5,6</sup>

More information is available in the [OXINIUM material science compendium](#)

Abbreviations: ASA = American Society of Anaesthesiologists; CI = confidence interval; CoCr = cobalt-chromium; CR = cruciate-retaining; PS = posterior-stabilised; TKR = total knee replacement; UK NJR = National Joint Registry for England, Wales and Northern Ireland.


# OXINIUM<sup>◇</sup> JOURNEY<sup>◇</sup> II/ LEGION<sup>◇</sup>/ GENESIS<sup>◇</sup> II implants: infection



The OXINIUM knee portfolio demonstrates a **significantly lower risk of revision for infection**, a major cause for TKA revisions,<sup>3,4</sup> compared to equivalent CoCr designs<sup>2</sup> and a numerically lower risk of revision versus the class average<sup>2</sup> when adjusting for population variables


Cox Proportional Hazards model for Revision for Infection (95% CI). Adjustment includes Age Group, Gender, Year Cohort, BMI Group, ASA, Patella Used, PS/CR Constraint

OXINIUM vs CoCr	OXINIUM vs All Other Cemented TKR	CoCr vs All Other Cemented TKR
0.70 (0.56–0.88) p=0.002	0.86 (0.72–1.02) p=0.083	1.08 (1.00–1.17) p=0.039

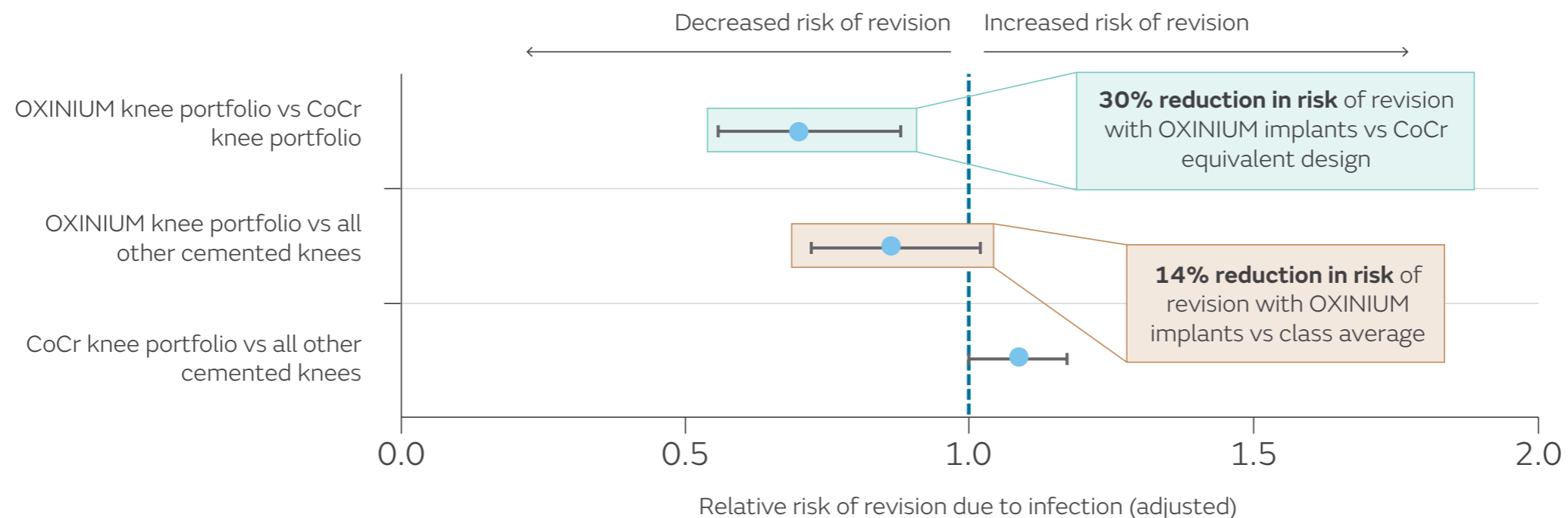

Abbreviations: ASA = American Society of Anaesthesiologists; CI = confidence interval; CoCr = cobalt-chromium; CR = cruciate-retaining; PS = posterior-stabilised.

# Summary of OXINIUM<sup>®</sup> UK NJR data analysis

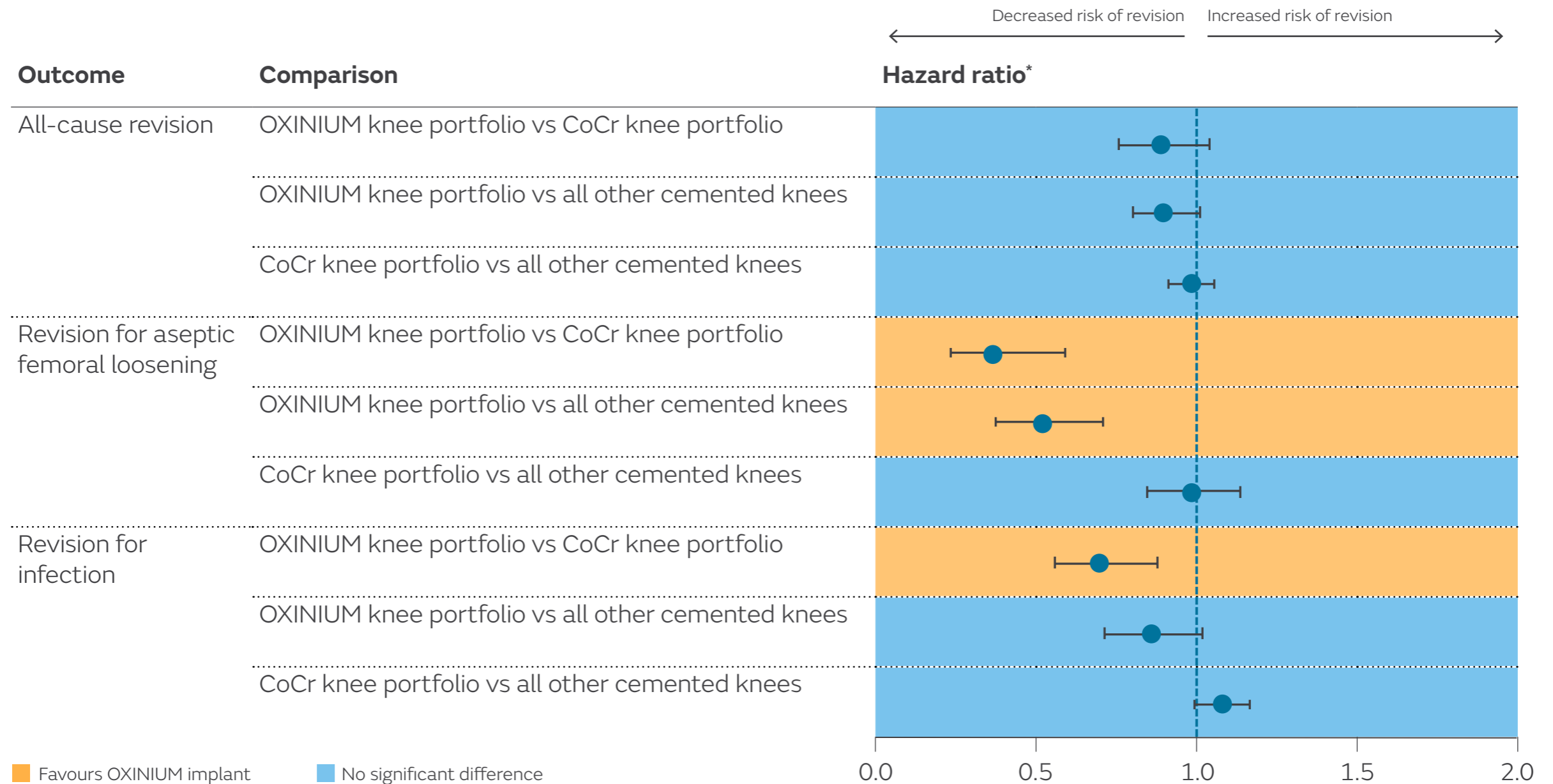

\*Adjusted for population variables (age group, gender, year, BMI group, ASA grade, PS/CR and patella resurfacing).  
 Abbreviations: CoCr = cobalt-chromium; CR = cruciate-retaining; PS = posterior-stabilised; TKA = total knee arthroplasty; UK NJR = National Joint Registry for England, Wales and Northern Ireland.

# References and acknowledgements



1. Civinini R, Carulli C, Matassi F, Lepri AC, Sirleo L, Innocenti M. The survival of total knee arthroplasty: current data from registries on tribology. *HSS Journal*. 2017;13:28–31.
2. National Joint Registry for England, Wales and Northern Ireland: Bespoke report for Genesis II/ Legion/ Journey II: Oxinium vs CoCr. 16 December 2025. Report available upon request.\*
3. Lum ZC, Shieh AK, Dorr LD. Why total knees fail - a modern perspective review. *World J Orthop*. 2018;9(4):60–64.
4. Sharkey PF, Lichstein PM, Shen C, Tokarski AT, Parvizi J. Why are total knee arthroplasties failing today - has anything changed after 10 years? *J Arthroplasty*. 2014;29(9):1774–8.
5. Dalal A, Pawar V, McAllister K, Weaver C, Hallab NJ. Orthopedic implant cobalt-alloy particles produce greater toxicity and inflammatory cytokines than titanium alloy and zirconium alloy-based particles in vitro, in human osteoblasts, fibroblasts, and macrophages. *J Biomed Mater Res Part A*. 2012;100A:2147–2158.
6. Hallab NJ, McAllister H, Jacobs JJ, Pawar V. Zirconium-alloy and zirconium-oxide particles produce less toxicity and inflammatory cytokines than cobalt-alloy and titanium-alloy particles in vitro, in human osteoblasts, fibroblasts and macrophages. Poster presented at: Orthopaedic Research Society Annual Meeting; February 4–7, 2012; San Francisco, CA, USA.

\*We thank the patients and staff of all the hospitals in England, Wales and Northern Ireland who have contributed data to the National Joint Registry. We are grateful to the Healthcare Quality Improvement Partnership (HQIP), the NJR Steering Committee and staff at the NJR Centre for facilitating this work. The views expressed represent those of Smith+Nephew and do not necessarily reflect those of the National Joint Registry Steering Committee or the Health Quality Improvement Partnership (HQIP) who do not vouch for how the information is presented. The data used for this analysis was obtained from the National Joint Registry ("NJR"), part of the Healthcare Quality Improvement Partnership ("HQIP"). HQIP, the NJR and/or its contractor, Northgate Public Services (UK) Limited ("NPS") take no responsibility (except as prohibited by law) for the accuracy, currency, reliability and correctness of any data used or referred to in this report, nor for the accuracy, currency, reliability and correctness of links or references to other information sources and disclaims all warranties in relation to such data, links and references to the maximum extent permitted by legislation including any duty of care to third party readers of the data analysis.

# Smith+Nephew

Products may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Please contact your Smith+Nephew representative or distributor if you have questions about the availability of Smith+Nephew products in your area. For detailed product information, including indications for use, contraindications, precautions and warnings, please consult the product's applicable Instructions for Use (IFU) prior to use.

Smith & Nephew, Inc.  
1450 Brooks Road  
Memphis, TN 38116  
USA  
Telephone: 1-901-396-2121  
Information: 1-800-821-5700  
Orders/Inquiries: 1-800-238-7538

**Developed by Evidence Communications,  
Global Clinical & Medical Affairs**

**[www.smith-nephew.com](http://www.smith-nephew.com)**

◊Trademark of Smith+Nephew

All Trademarks acknowledged

© 2026 Smith+Nephew

36855 V2 03/26. Published March 2026