


Application Guide


IODOSORB[◊] application



Clean the wound and the surrounding area with either a gentle stream of sterile water or saline. Do not dry the wound surface.

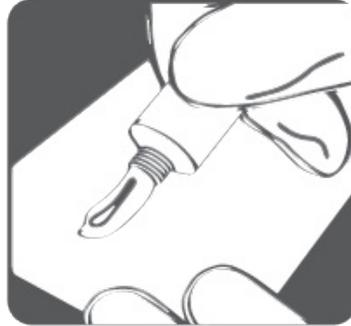

Invert cap to puncture seal in tube. Spread IODOSORB[◊] in 1/8 - 1/4 in. thickness to dry sterile gauze, sufficient to cover all parts of the wound. Larger amounts are unnecessary though not problematic.

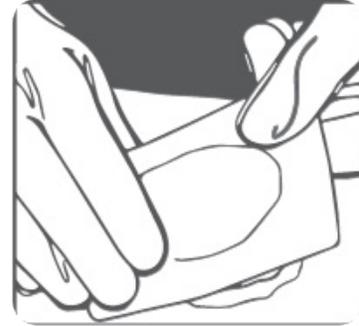

With gloved hands, position the prepared gauze onto the wound.

A single application should not exceed 1.8oz (50g) gel and the total amount used in one week should not exceed 5.3oz (150g). When managing venous stasis ulcers, compression bandages should be used in addition to IODOSORB[◊].

Changing IODOSORB[◊]:

Each time the dressing is changed, the remaining IODOSORB[◊] should be gently washed from the wound or ulcer surface with a suitable cleansing solution (sterile water or saline) or with a sterile, wet swab.



IODOSORB[◊] should be changed three times a week or when all the IODOSORB[◊] has changed in color from brown to yellow/grey.



Remove IODOSORB[◊] with a gentle stream of sterile water or saline, using a sterile wet swab if necessary.



Gently blot any excess fluid, leaving the wound surface slightly moist, before re-applying IODOSORB[◊].

For detailed product information, including indications for use, contraindications, effects, precautions, warnings, and important safety information, please always consult each product's Instructions for Use (IFU) prior to use.

Facility recommendations

- When all beads are saturated with fluid and all iodine is released, the brown color of IODOSORB[®] changes to yellow/grey. This is an indication to change IODOSORB[®].
- A single application should not exceed 1.8oz (50g) gel and the total amount used in one week should not exceed 5.3oz (150g).
- The number of applications should be reduced as the exudate diminishes.
- When managing venous stasis ulcers, compression bandages should be used in addition to IODOSORB[®].
- When To Discontinue
 - IODOSORB[®] should be discontinued when the wound area is free of exudate and edema or complete healing occurs.
 - The duration of treatment should not exceed three months in any single course of treatment.
- To minimize the possibility of cross-contamination, contents of a tube should be limited to use on one patient.
- IODOSORB[®] is not effective in cleaning dry wounds.

Ordering Information

Products	Reference Number	Size	Carton/Packaging
IODOSORB [®] 40g	6602-1250-40	40g (1.4oz.)	Carton of 1
IODOSORB [®] 10g	6602-1250-40	40g (1.4oz.)	Carton of 1

For detailed product information, including indications for use, contraindications, effects, precautions, warnings, and important safety information, please always consult each product's Instructions for Use (IFU) prior to use.