

+ ALLEVYN[◇] Dressings

Trusted to work

Simplifying dressing decisions
in acute settings.

Smith+Nephew



ALLEVYN[◇]
Wound Dressings

Shaping what's possible in wound care

www.smith-nephew.com/allevyn

DISCOVER WHAT'S POSSIBLE HERE →




Contents

Complexity, variability and increasing demand impacts care	03
Choosing the right dressing shouldn't complicate wound care	04
ALLEVYN ^o Range: Dressing selection guide	06
ALLEVYN ^o Foam Dressings: Trusted to work in every environment	07
— ALLEVYN ^o LIFE Foam Dressings	08
— ALLEVYN ^o GENTLE BORDER Foam Dressings	09
— ALLEVYN ^o GENTLE BORDER LITE Foam Dressings	10
— ALLEVYN ^o NON-ADHESIVE Foam Dressings	11
— ALLEVYN ^o Ag Antimicrobial Foam Dressings	12
Ordering ALLEVYN ^o LIFE Dressings	13
Ordering ALLEVYN ^o GENTLE BORDER and ALLEVYN ^o GENTLE BORDER LITE Dressings	14
Ordering ALLEVYN ^o NON-ADHESIVE Dressings	15
Ordering ALLEVYN ^o Ag Antimicrobial Dressings	16



Complexity, variability and increasing demand impacts care delivery and patient outcomes

In hospitals, managing wound care and selecting dressings often involves **navigating through different levels of complexity:**

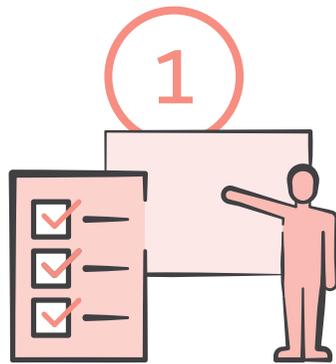


Variability in training and wound care practices:

Only 35% nurses rated their **knowledge of wound care products as good**¹



Managing complex patient profiles with different wound types:
For example, **patients with co-morbidities** and **poor skin integrity**

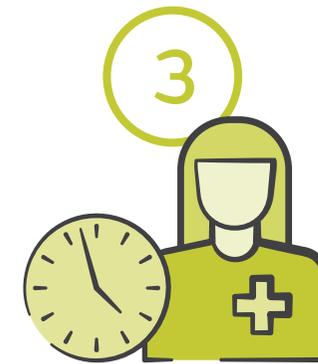

Limited nursing resource and time:
More than **70% of healthcare workers** are impacted by **staffing shortages**²

Choosing the right dressing shouldn't complicate wound care

Common **challenges with foam dressings** include leakage with higher exuding wounds, the dressing not staying in place, and increased dressing changes due to lack of durability.

The ALLEVYN Range **simplifies the choice** with advanced dressings designed to **meet the needs of wound care nurses and their patients**. Providing you with the **essentials you expect** from a foam dressing, and more.

Absorbent¹⁻³

The ALLEVYN hydrocellular foam core **innovatively balances absorption and retention** to create a moist wound healing environment⁵⁻⁹ and help minimise risk of maceration.⁵⁻⁸



Protective^{1,4}

The ALLEVYN breathable, waterproof¹⁻³ top film **provides a bacterial barrier**,¹⁰ while the dressing remains intact. Many ALLEVYN dressings redistribute pressure* to **minimise the risk** of pressure injury.^{1,4}




Gentle^{6,7}

The ALLEVYN gentle wound contact adhesives are **easy to apply and remove**^{5,8,11,12} and are shown to **minimise pain and trauma** during dressing changes and skin inspection.^{5,6,13-15}



ALLEVYN[®] Dressings

Trusted to work

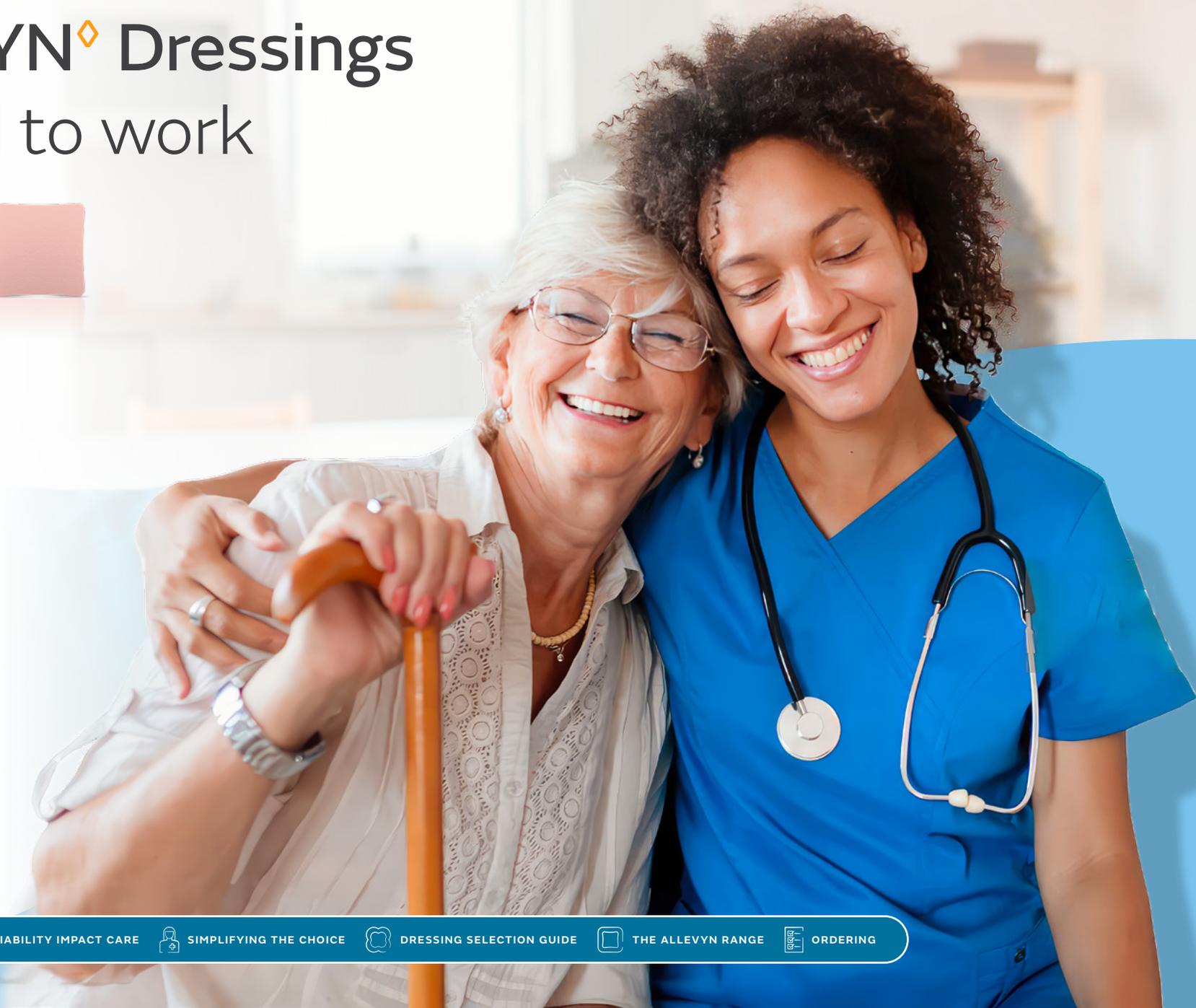

COMPLEXITY & VARIABILITY IMPACT CARE



SIMPLIFYING THE CHOICE



DRESSING SELECTION GUIDE



THE ALLEVYN RANGE



ORDERING

ALLEVYN[®] Range: Dressing selection guide

Primary indications					Exudate	Adhesion	Recommendation
 PRESSURE INJURY	 PRESSURE INJURY PREVENTION	 DIABETIC FOOT ULCERS	 LEG ULCERS		 MODERATE - HIGH		
 PRESSURE INJURY	 PRESSURE INJURY PREVENTION	 DIABETIC FOOT ULCERS	 SURGICAL WOUNDS	 SKIN TEARS	 MODERATE		
 PRESSURE INJURY	 PRESSURE INJURY PREVENTION	 DIABETIC FOOT ULCERS	 LEG ULCERS	 SKIN TEARS	 LOW		
 PRESSURE INJURY	 DIABETIC FOOT ULCERS	 LEG ULCERS			 MODERATE		
 PRESSURE INJURY	 DIABETIC FOOT ULCERS	 LEG ULCERS	 SURGICAL WOUNDS	 BURNS	 MODERATE	 Ag ADHESIVE  Ag NON-ADHESIVE  Ag GENTLE BORDER	

Refer to individual IFUs for a full list of indications.

ALLEVYN[®] Foam Dressings

Trusted to work in every environment

ALLEVYN Foam Dressings are **trusted leaders in fluid management for moist healing of every wound**¹⁻⁵ and are our **first line of defence for pressure injuries**. Whether it's different settings, different patients, different wounds, or different needs, the ALLEVYN family of dressings is **trusted with improving patient quality of life**.^{1-3,9,10}



Learn more about the ALLEVYN Range:



ALLEVYN[◇] LIFE Foam Dressings



An all-in-one dressing for **effective exudate management** and **pressure injury prevention**. The **ExuLOCK[◇]** advanced lock-away core helps to **prevent leakages**, **reduce dressing changes** and helps wounds to **heal undisturbed for longer**.¹⁻⁴




ExuMASK[◇] change indicator technology

- **Minimises visual impact** of absorbed exudate^{2,3}
- Works as an indicator as to when to change the dressing which helps **minimise clinically unnecessary changes**.^{2,3}



ExuLOCK[◇] hyper-absorbent lock-away technology

- **Absorbs exudate** and spreads it laterally across the dressing to utilise the entire dressing area^{1,2,4}
- **Locks in exudate** helping to prevent leakage^{1,2,4}

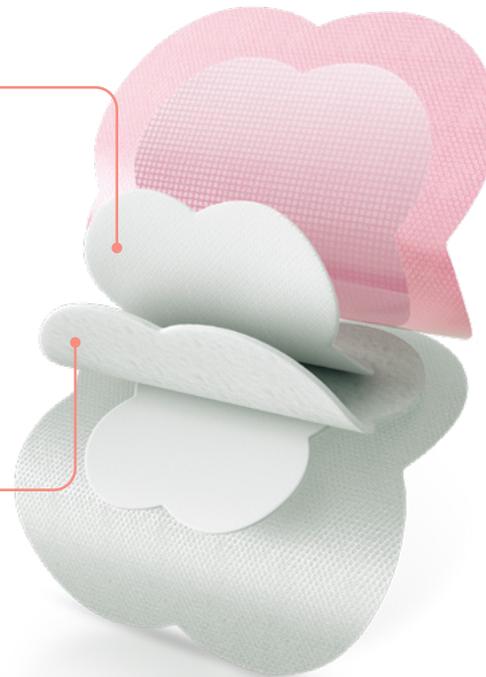

Pressure injury prevention

ALLEVYN LIFE Dressings **reduce shear and friction**⁵ and **support pressure distribution**.⁶ They allow repositioning **without compromising adherence**.⁷⁻⁹

ALLEVYN LIFE Dressings have been shown to:


- Reduce incidence of sacral pressure injuries by **up to 71% in elderly patients** with hip fracture.¹⁰
- Produce per-patient treatment cost savings **between 37-69%**.^{§11}

[†] As part of a comprehensive pressure injury prevention protocol

[§] When used as part of a comprehensive prevention program and when compared to standard care alone

ALLEVYN[◇] GENTLE BORDER Foam Dressings

Multi-stretch technology improves comfort and fit, **stays in place even on awkward areas.**¹⁻⁵ Improves **adherence and compliance** with patients.^{7,8}

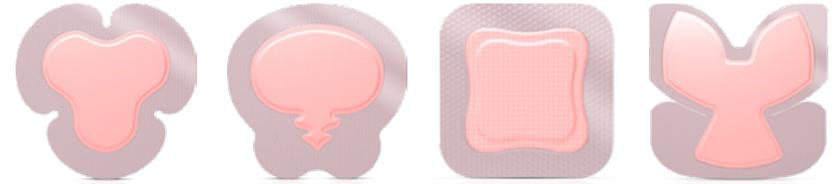

Multi-way stretch technology

- Helps with application on **awkward areas**¹⁻⁴
- Facilitates conformability and helps to **maximise patient comfort**¹⁻⁵



Multi-site unique trilobe dressing

- Unique, trilobe design conforms closely to difficult to dress body areas such as the knee whilst **allowing full flexibility of the joint**¹⁻⁴



Cuttable^{‡6}



Medical-device-related pressure injury prevention[†]

The **ability to cut**^{‡6} and **multiway stretch** helps conform to awkward areas¹⁻⁴ making the dressings **versatile and adaptable** for use under medical devices such as collars and facial masks such as CPAP.

* Cutting will compromise the bacterial barrier properties of the dressing.



COMPLEXITY & VARIABILITY IMPACT CARE



SIMPLIFYING THE CHOICE



DRESSING SELECTION GUIDE



THE ALLEVYN RANGE



ORDERING

REFS

09

ALLEVYN[®] GENTLE BORDER LITE Foam Dressings

A thinner fit provides **conformability and protection** for low draining wounds, skin tears and facial medical-device-related pressure injuries.



Multi-way stretch technology

- Helps with application on **awkward areas and areas of high patient movement**^{3,9}




Multi-site unique trilobe dressing

- Unique, trilobe design **conforms closely to difficult to dress body areas**³



Cuttable^{11,10}

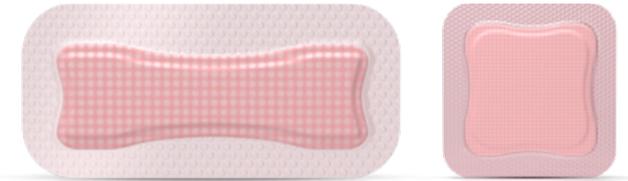

Thinner fit

- **Half the thickness**[†] of ALLEVYN GENTLE BORDER dressings
- Improved **comfort and fit**¹⁻⁸



Pressure injury prevention

High conformability to the contours of the face^{12,13} helps **protect skin exposed to pressure and friction**,^{11,12,14-16} making the dressing adaptable for use under tubing and face masks.

ALLEVYN[◇] NON-ADHESIVE Foam Dressings

Designed for patients with delicate skin, with unique shapes.
Manages fluid effectively even under compression.^{10,11}




Non-adherent

- Suitable for **fragile skin**¹⁻³
- Suitable **for use under compression**, and retains fluid when compressed^{6,7}



Unique heel shape

- Facilitates **conformability to the heel area**⁸

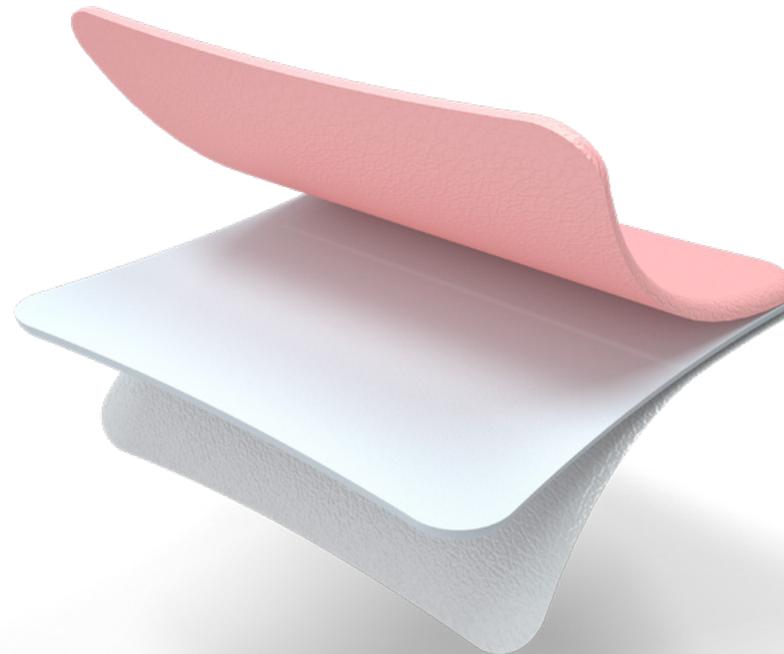

Unique tracheostomy shape

- 'Keyhole' aperture allows **use with a tracheostomy tube** stoma, or other drainage tubes⁹
- Available for **pediatrics**



Cuttable*⁸

* Cutting will compromise the bacterial barrier properties of the dressing.


ALLEVYN[◊] Ag Antimicrobial Foam Dressings

Combining the benefits of the ALLEVYN Range with added **antimicrobial activity of silver sulfadiazine (SSD)**,¹⁻³ the ALLEVYN Ag Range is **designed for patient comfort** when infection is a concern.⁴⁻⁷



Sustained release of silver sulfadiazine (SSD)

- Provides antimicrobial action for **up to 7 days**^{†1,2,8}
- Kills pathogenic organisms found in wounds to **reduce the risk of colonization and infection**^{‡2,3,9}
- **Minimises wound odour** causing bacteria^{†2,10,11}
- **Minimises bacterial load** which is causative of increased matrix metalloproteinase (MMP) levels in the wound and inflammation^{†1,2,8,12}

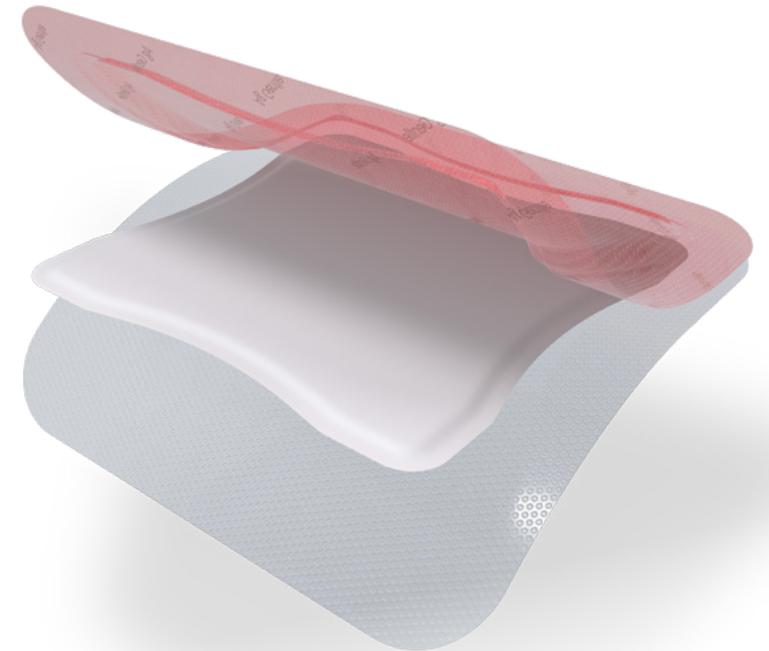

* as demonstrated in vivo † as demonstrated in vitro ‡ due to its non-adherent properties



COMPLEXITY & VARIABILITY IMPACT CARE



SIMPLIFYING THE CHOICE



DRESSING SELECTION GUIDE



THE ALLEVYN RANGE



ORDERING

REFS

12

ALLEVYN[®] LIFE Foam Dressings



ALLEVYN[®] LIFE
Foam Dressing



ALLEVYN[®] LIFE
Heel Foam Dressing



ALLEVYN[®] LIFE
Sacrum Foam Dressing

Code	Size	Shape	Pad size	Pad area	Qty
ALLEVYN[®] LIFE Foam Dressings					
66801067	10.3cm x 10.3cm	Square	5.1cm x 5.1cm	22.7cm ²	10
66801068	12.9cm x 12.9cm	Square	7.6cm x 7.6cm	51.8cm ²	10
66801069	15.4cm x 15.4cm	Square	10.2cm x 10.2cm	92.9cm ²	10
66801070	21cm x 21cm	Square	15.4cm x 15.4cm	210.5cm ²	10
66801304	25cm x 25.2cm	Heel	20cm x 20.2cm	242.7cm ²	5
66801306	17.2cm x 17.5cm	Sacrum	8.5cm x 12.4cm	86.9cm ²	10
66801307	21.6cm x 23cm	Sacrum	12.3cm x 17cm	172.8cm ²	10

Can be left in place for up to 7 days (excluding sacral variants).

Can be left in place for up to 5 days in the sacral area.

For sacrum shapes the pad sizes and areas do not include the 'tail' of the dressing



COMPLEXITY & VARIABILITY IMPACT CARE



SIMPLIFYING THE CHOICE



DRESSING SELECTION GUIDE



THE ALLEVYN RANGE




ORDERING

ALLEVYN[®] GENTLE BORDER & ALLEVYN[®] GENTLE BORDER LITE Foam Dressings



ALLEVYN[®]
GENTLE BORDER
Foam Dressing

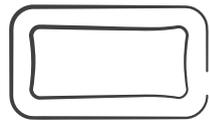

ALLEVYN[®]
GENTLE BORDER
Foam Dressing



ALLEVYN[®]
GENTLE BORDER
Sacrum Foam Dressing




ALLEVYN[®]
GENTLE BORDER
Multisite Foam Dressing




ALLEVYN[®]
GENTLE BORDER
Heel Foam Dressing



ALLEVYN[®] GENTLE
BORDER LITE
Foam Dressing

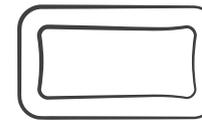

ALLEVYN[®] GENTLE
BORDER LITE
Foam Dressing

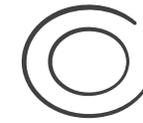

ALLEVYN[®] GENTLE
BORDER LITE
Oval Foam Dressing



ALLEVYN[®] GENTLE
BORDER LITE
Multisite Foam Dressing

Code	Size	Shape	Pad size	Pad area	Qty
------	------	-------	----------	----------	-----

ALLEVYN[®] GENTLE BORDER Foam Dressings

66800269	7.5cm x 7.5cm	Square	5cm x 5cm	22.6cm ²	10
66800270	10cm x 10cm	Square	7.5cm x 7.5cm	51.8cm ²	10
66800900	10cm x 20cm	Rectangular	7cm x 17cm	116.3cm ²	10
66800264	10cm x 25cm	Rectangular	6.5cm x 21.8cm	135.9cm ²	10
66800265	10cm x 30cm	Rectangular	6.5cm x 26.8cm	159.8cm ²	10
66800272	12.5cm x 12.5cm	Square	10cm x 10cm	92.9cm ²	10
66800273	17.5cm x 17.5cm	Square	15cm x 15cm	210.5cm ²	5
66800898	16.8cm x 17.1cm	Sacrum	8.5cm x 12.4cm	86.9cm ²	10
66800959	17.1cm x 17.9cm	Multisite	11.6cm x 12.3cm	100cm ²	10
66800506	23cm x 23.2cm	Heel	20cm x 20.2cm	242.7cm ²	5

Code	Size	Shape	Pad size	Pad area	Qty
------	------	-------	----------	----------	-----

ALLEVYN[®] GENTLE BORDER LITE Foam Dressings

66800833	5cm x 5cm	Square	3cm x 3cm	8.2cm ²	10
66800836	5.5cm x 12cm	Rectangular	3cm x 9.5cm	24.9cm ²	10
66801013	8.6cm x 7.7cm	Oval	6.1cm x 4.8cm	22.9cm ²	10
66801041	8cm x 8.4cm	Multisite	5.4cm x 5.8cm	21.9cm ²	10

For sacrum shapes the pad sizes and areas do not include the 'tail' of the dressing



COMPLEXITY & VARIABILITY IMPACT CARE



SIMPLIFYING THE CHOICE



DRESSING SELECTION GUIDE




THE ALLEVYN RANGE



ORDERING

ALLEVYN[®] NON-ADHESIVE Foam Dressings

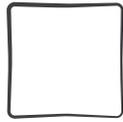

ALLEVYN[®]
NON-ADHESIVE
Foam Dressing



ALLEVYN[®]
NON-ADHESIVE
Foam Dressing



ALLEVYN[®]
NON-ADHESIVE
Heel Foam Dressing

Code	Size	Shape	Pad size	Pad area	Qty
------	------	-------	----------	----------	-----

ALLEVYN[®] NON-ADHESIVE Foam Dressings

66007643	5cm x 5cm	Square	5cm x 5cm	25cm ²	10
66007637	10cm x 10cm	Square	10cm x 10cm	100.3cm ²	10
66007335	10cm x 20cm	Rectangular	10cm x 20cm	204cm ²	10
66000093	15cm x 15cm	Square	15cm x 15cm	226cm ²	10
66007638	20cm x 20cm	Square	20cm x 20cm	400cm ²	10
66007630	Heel	Heel	10.6cm x 13.6cm	200.2cm ²	5
66007640	Tracheostomy	Tracheostomy	9cm x 9cm	81cm ²	10



COMPLEXITY & VARIABILITY IMPACT CARE



SIMPLIFYING THE CHOICE



DRESSING SELECTION GUIDE



THE ALLEVYN RANGE



ORDERING

ALLEVYN[®] Ag Antimicrobial Foam Dressings



ALLEVYN[®] Ag
ADHESIVE
Antimicrobial Foam Dressing



ALLEVYN[®] Ag
ADHESIVE
Antimicrobial Sacrum Foam Dressing

Code	Size	Shape	Pad size	Pad area	Qty
ALLEVYN[®] Ag ADHESIVE Foam Dressings					
66800073	7.5cm x 7.5cm	Square	5cm x 5cm	22.6cm ²	10
66800075	10cm x 10cm	Square	7.5cm x 7.5cm	51.8cm ²	10
66800078	12.5cm x 12.5cm	Square	10cm x 10cm	92.9cm ²	10
66800081	17.5cm x 17.5cm	Square	15cm x 15cm	210.5cm ²	10


ALLEVYN[®] Ag NON-ADHESIVE Foam Dressings

66800083	5cm x 5cm	Square	5cm x 5cm	25cm ²	10
66800086	10cm x 10cm	Square	10cm x 10cm	100.3cm ²	10
66800089	15cm x 15cm	Square	15cm x 15cm	226cm ²	10
66800091	20cm x 20cm	Square	20cm x 20cm	400cm ²	10



ALLEVYN[®] Ag
GENTLE BORDER
Antimicrobial Foam Dressing

Code	Size	Shape	Pad size	Pad area	Qty
ALLEVYN[®] Ag GENTLE BORDER Foam Dressings					
66800460	7.5cm x 7.5cm	Square	5cm x 5cm	22.6cm ²	10
66800461	10cm x 10cm	Square	7.5cm x 7.5cm	51.8cm ²	10
66800462	12.5cm x 12.5cm	Square	10cm x 10cm	92.9cm ²	10
66800463	17.5cm x 17.5cm	Square	15cm x 15cm	210.5cm ²	10

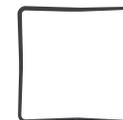

ALLEVYN[®] Ag
NON-ADHESIVE
Antimicrobial Foam Dressing



ALLEVYN[®] Ag
NON-ADHESIVE
Antimicrobial Heel Foam Dressing

For sacrum shapes the pad sizes and areas do not include the 'tail' of the dressing



COMPLEXITY & VARIABILITY IMPACT CARE



SIMPLIFYING THE CHOICE



DRESSING SELECTION GUIDE



THE ALLEVYN RANGE



ORDERING

Smith & Nephew Pty Ltd
Australia
T +61 2 9857 3999
F +61 2 9857 3900

smith-nephew.com/en-au

Smith & Nephew Ltd
New Zealand
T +64 9 820 2840
F +64 9 820 2841

smith-nephew.com/en-nz

◇ Trademark of Smith+Nephew
All trademarks acknowledged
© April 2025 Smith+Nephew
40493-anz V1 04/25

This material is intended for healthcare professionals.
For detailed product information, including indications
for use, contraindications, precautions and warnings,
please consult the product's applicable Instructions
for Use (IFU) prior to use.



COMPLEXITY & VARIABILITY IMPACT CARE



SIMPLIFYING THE CHOICE



DRESSING SELECTION GUIDE



THE ALLEVYN RANGE



ORDERING

Complexity, variability and increasing demand impacts care delivery and patient outcomes

In hospitals, managing wound care and selecting dressings often involves **navigating through different levels of complexity:**



Page 3:

1. Brigid M Gillespie et al. J Clin Nurs 2014 Sep;23(17-18):2618-26 Wound care practices: a survey of acute care nurses. 2. Australia Mental health. Annual survey of healthcare professionals 2022.

Choosing the right dressing shouldn't complicate wound care

Common **challenges with foam dressings** include leakage with higher exuding wounds, the dressing not staying in place, and increased dressing changes due to lack of durability.

The ALLEVYN Range **simplifies the choice** with advanced dressings designed to **meet the needs of wound care nurses and their patients**. Providing you with the **essentials you expect** from a foam dressing, and more.

Absorbent¹⁻³

Protective^{1,4}

Gentle^{6,7}

Page 4:

1. DS/16/424/R Version 3 - AGB physical evaluation **2.** ALLEVYN Adhesive dressing physical properties. DS/09/013/R7. **3.** DOF 0702001. - Ag Non adhesive physical properties **4.** Forni C, Gazineo D, Allegrini E, et al. Effectiveness of a multi-layer silicone-adhesive polyurethane foam dressing as prevention for sacral pressure ulcers in at-risk in-patients: Randomized controlled trial. International Journal of Nursing Studies. 2022;127:104172 **5.** Leonard S, McCluskey P, Long S, Butters V, Winter R, Smith G. An evaluation of Allevyn™ Adhesive and Non-Adhesive foam dressings. Wounds UK. 2009;5(1):17-26. **6.** Hurd T, Gregory L, Jones A, Brown S. A multi-centre in-market evaluation of ALLEVYN Gentle Border. Wounds UK. 2009;5(3):32 - 44. **7.** Smith+Nephew 2022. Post Market Clinical Follow-Up (PMCF) Report for ALLEVYN GENTLE. CSD.AWM.22.050 **8.** Rossington A, Drysdale K, Winter R. Clinical performance and positive impact on patient wellbeing of ALLEVYN Life. Wounds UK. 2013;9(4):91 - 95 **9.** White, R., Hartwell, S., and Brown, S., Interim report on a study to assess the effectiveness and improved fluid uptake of new ALLEVYN. Wounds UK. 2007; 3(4):122-127 Available here: <https://www.woundsinternational.com/resources/details/interim-report-on-the-study-to-assess-the-effectiveness-and-improvedfluid-uptake-of-new-allevyn> **10.** Smith+Nephew 2023. Bacterial barrier properties of the ALLEVYN™ Dressing Range. Internal Report. CSD.AWM.23.032 **11.** Smith & Nephew 2017. An open, prospective, randomised, comparative volunteer trial to compare the performance of silicone adhesive dressings. Internal Report. GMCA-DOF-10. **12.** Smith+Nephew 2016. Product Performance of Next Generation ALLEVYN Life Internal Report. (HVT080) GMCA-DOF/08. **13.** Martini L, Reali U, Borgognoni L, Brandani P, Andriessen A. Comparison of two dressings in the management of partial-thickness donor sites. J Wound Care. 1999;8(9):457-460. **14.** Bale S, Squires D, Varnon T, Walker A, Benbow M, Harding K. A comparison of two dressings in pressure sore management. J Wound Care. 1997;6(10):463-466. **15.** Smith & Nephew 1996. A multicentre pilot study to investigate the physical performance of a novel Allevyn Adhesive dressing. Internal Report. SR/00000/CT9336.

ALLEVYN[®] Foam Dressings

Trusted to work in every environment

ALLEVYN Foam Dressings are **trusted leaders in fluid management for moist healing of every wound**¹⁻⁵ and are our **first line of defence for pressure injuries**. Whether it's different settings, different patients, different wounds, or different needs, the ALLEVYN family of dressings is **trusted with improving patient quality of life**.^{1-3,9,10}



[Learn more](#) about the ALLEVYN Range:

Page 7:

1. Smith+Nephew 2022.Post Market Clinical Follow-Up (PMCF) Report for ALLEVYN GENTLE. CSD.AWM.22.050 **2.** Hurd T, Gregory L, Jones A, Brown S.A multi-centre in-market evaluation of ALLEVYN Gentle Border. Wounds UK. 2009;5(3):32-44. **3.** Leonard S, Mccluskey P, Long S, et al. An evaluation of Allevyn Adhesive and Non-Adhesive foam dressings. Wounds UK. 2009;5(1):17 - 28. **4.** Rossington A, Drysdale K, Winter R. Clinical performance and positive impact on patient wellbeing of ALLEVYN Life. Wounds UK. 2013;9(4):91 – 95 **5.** White, R., Hartwell, S., and Brown, S., Interim report on a study to assess the effectiveness and improved fluid uptake of new ALLEVYN. Wounds UK. 2007; 3(4):122-127 Available here: <https://www.woundsinternational.com/resources/details/interim-report-on-the-study-to-assess-the-effectiveness-and-improvedfluid-uptake-of-new-allevyn> **6.** Allen D, Saunders C, Tiskowitz K. Reduced weekly dressing changes with a five-layer foam dressing. Results of a systematic literature review and meta-analysis of clinical studies. Paper presented at: SAWC; 2022; Las Vegas **7.** Smith+Nephew. Decision Record. 2022. PCS. DR.AWM.005 **8.** Bian Y et al. Wound-healing improvement by resurfacing split-thickness skin donor sites with thin split-thickness grafting. Burns. 2016; 42: 123-130 **9.** Simon D, Bielby A. A structured collaborative approach to appraise the clinical performance of a new product. Wounds UK. 2014;10(3):80 - 87. **10.** Stephen-Haynes J, Greenwood M. Clinical evaluation of Allevyn Gentle Border Lite within one health trust. British Journal of Nursing. 2011;20(20):36-42.

ALLEVYN[◇] LIFE Foam Dressings



An all-in-one dressing for **effective exudate management** and **pressure injury prevention**. The **ExuLOCK[◇]** advanced lock-away core helps to **prevent leakages**, **reduce dressing changes** and helps wounds to **heal undisturbed for longer**.¹⁻⁴



ExuMASK[◇] change indicator technology

• **Minimises visual impact** of absorbed exudate^{2,3}




Pressure injury prevention

Page 8:

1. Rossington A, Drysdale K, Winter R. Clinical performance and positive impact on patient wellbeing of ALLEVYN Life. Wounds UK. 2013;9(4):91–95. **2.** Tiscar-González V, Menor- Rodríguez MJ, Rabadán-Sainz C, et al. Clinical and Economic Impact of Wound Care Using a Polyurethane Foam Multilayer Dressing. Adv Skin Wound Care. 2021;34(1):23–30. **3.** Simon D, Bielby A. A structured collaborative approach to appraise the clinical performance of a new product. Wounds UK. 2014;10(3):80–87. **4.** S+N Data on File. Wound Model Testing of New ALLEVYN Life Gen2 wcl Dressing using Horse Serum at a Flow Rate Modelling that of a Moderately Exuding Wound. DS/14/303/R. 2016. **5.** Marché C, Creehan S, Gefen A. The frictional energy absorber effectiveness and its impact on the pressure ulcer prevention performance of multilayer dressings. Int Wound J. 2024;21(4):e14871. **6.** S+N 2021. Pressure Mapping. Benchtop Testing Report. CSD.AWM.21.006. **7.** Smith+Nephew 2016. Product Performance of Next Generation ALLEVYN Life Internal Report. (HVT080) GMCA-DOF/08. **8.** Clarke B. Positive patient outcomes: The use of a new silicone adhesive foam dressing for pressure ulcer prevention and treatment. Paper presented at: CAET; 2013. **9.** Lisco C. Evaluation of a new silicone gel-adhesive hydrocellular foam dressing as part of a pressure ulcer prevention plan for ICU patients. Paper presented at: WOCN; 2013. **10.** Forni C, D'Alessandro F, Gallerani P, et al. Effectiveness of using a new polyurethane foam multi-layer dressing in the sacral area to prevent the onset of pressure ulcer in the elderly with hip fractures: A pragmatic randomised controlled trial. Int Wound J. 2018;15(3):383–390. **11.** Forni C, Searle R. A multilayer polyurethane foam dressing for pressure ulcer prevention in older hip fracture patients: an economic evaluation. J Wound Care. 2020;29(2):120–127

ALLEVYN[◇] GENTLE BORDER Foam Dressings

Multi-stretch technology improves comfort and fit, **stays in place even on awkward areas.**¹⁻⁵ Improves **adherence and compliance** with patients.^{7,8}

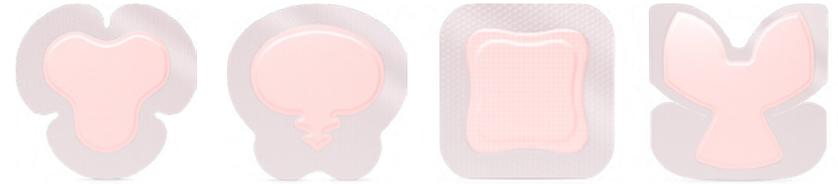

Multi-way stretch technology

- Helps with application on **awkward areas**¹⁻⁴



Cuttable^{†6}



Page 9:

1. T. Hurd, L. Gregory, A. Jones and S. Brown, "A multi-centre, in market evaluation of ALLEVYN Gentle Border", Wounds UK, Vol 5, 32-44. 2009. **2.** Smith & Nephew 2016. ALLEVYN Gentle Border. Heel Gen 2 - physical evaluation. Internal Report. DS/16/465/R. **3.** Smith & Nephew 2017. ALLEVYN Gentle Border Gen 2 - Physical Evaluation. Internal Report. DS/16/424/R V3. **4.** Rafter L, Reynolds T, Rafter M. An audit of patient outcomes in the management of skin tears using silicone dressings. Wounds UK. 2016;12(2):70-78. **5.** Hampton S. An evaluation of a silicone adhesive shaped heel dressing. Br J Nurs. 2010;19(6):S30-33. **6.** Smith & Nephew 2015. Cutting of ALLEVYN variants. Internal Report. DS/14/318/R. **7.** Smith & Nephew 2017. An open, prospective, randomised, comparative volunteer trial to compare the performance of silicone adhesive dressings. Internal Report. GMCA-DOF-10. **8.** Smith & Nephew 2017. An open, prospective, randomised, comparative study to compare the performance of ALLEVYN Gentle Border Multisite with an alternative silicone adhesive dressing. Internal Report. GMCA-DOF-09.

ALLEVYN[◇] GENTLE BORDER LITE Foam Dressings

A thinner fit provides **conformability and protection** for low draining wounds, skin tears and facial medical-device-related pressure injuries.



Multi-way stretch technology

- Helps with application on **awkward areas and areas of high patient movement**^{3,9}



Thinner fit

- **Half the thickness[†]** of ALLEVYN GENTLE BORDER dressings

Page 10:

1. Hampton S. An evaluation of a silicone adhesive shaped heel dressing. British journal of nursing (Mark Allen Publishing). 2010;19(6):S30-33. **2.** Hurd T, Gregory L, Jones A, Brown S. A multicentre in-market evaluation of ALLEVYN Gentle Border. Wounds UK. 2009;5(3):32-44. **3.** Stephen-Haynes J, Greenwood M. Clinical evaluation of Alleevyn Gentle Border Lite™ within one health trust. British journal of nursing. 2011;20:S36-S42. **4.** Smith & Nephew 2016. ALLEVYN Gentle Border Heel Gen 2 – physical evaluation. Internal Report. DS/16/465/R. **5.** Smith & Nephew 2017. ALLEVYN Gentle Border Gen 2 - Physical Evaluation. Internal Report. DS/16/424/R V3. **6.** Rafter L, Reynolds T, Rafter M. An audit of patient outcomes in the management of skin tears using siliconedressings. Wounds UK. 2016;12(2):70-78. **7.** Grothier L. Gentle foam dressings: interim results of an evaluation of the Alleevyn range. Br J Nurs. 2009;18(11). **8.** Vowden K, Harding K, Moiemmen N, Drysdale K, Mistry C. An open, prospective randomised, multi-centre clinical evaluation of a hydrocellular silicone foam dressing* in the management of exuding chronic and acute wounds. Paper presented at: EWMA; 2011; Brussels. **9.** Smith+Nephew 2019. Technical Rationale of Multiway Stretch Claim for ALLEVYN Gentle Boarder Lite. Internal Report. RD/19/201. **10.** Smith+Nephew 2015. Cutting of ALLEVYN Variants. Internal Report. DS/14/318/R. **11.** Smith+Nephew 2021. Pressure Redistribution Testing of Alleevyn Gentle Border Lite. Internal Report. DS/21/336/R. **12.** Smith+Nephew 2021. PMS 274 ALLEVYN. Gentle Border Lite: PMCF ACTIVITY SUMMARY REPORT. Internal Report. **13.** Smith+Nephew 2010. ALLEVYN Gentle Border Lite Dressing Physical Properties. Internal report. DS/10/100/R1. **14.** Smith+Nephew 2021. Pressure Transmission of Face Mask over ALLEVYN Gentle Border and Mepilex™ Border Lite. Internal report. DS/21/274/R. **15.** Smith+Nephew 2012. Friction Testing Comparison of ALLEVYN Gentle Border Lite versus other Competitor Products. Internal report. DS/11/125/R1. **16.** Smith+Nephew 2019. Properties of ALLEVYN life advanced wound care dressing that can contribute to the effective use as part of a Pressure Injury Prevention protocol. Internal Report. RD/19/177.

ALLEVYN[◇] NON-ADHESIVE Foam Dressings

Designed for patients with delicate skin, with unique shapes.
Manages fluid effectively even under compression.^{10,11}



Non-adherent

- Suitable for **fragile skin**¹⁻³
- Suitable **for use under compression**, and retains fluid when compressed^{6,7}



Unique tracheostomy shape

Page 11:

1. Leonard S, McCluskey P, Long S, Butters V, Winter R, Smith G. An evaluation of Allevyn™ Adhesive and Non-Adhesive foam dressings. Wounds UK. 2009;5(1):17-26. **2.** Smith+Nephew 2006. A prospective user evaluation on the effect of process changes to the manufacture of ALLEVYN Non-Adhesive wound dressings. Internal Report. CER/CE/022/ANA. **3.** Kurring PA, Roberts CD, Quinlan D. Evaluation of a hydrocellular dressing in the management of exuding wounds in the community British Journal of Nursing 1994;3(20):1049-1050, 1052-1043. **4.** Smith+Nephew 2008. An evaluation of the use of ALLEVYN Adhesive and ALLEVYN Non-Adhesive dressings. Internal Report. SR/CIME/003. **5.** Foster AVM, Greenhill MT, Edmonds ME. Comparing two dressings in the treatment of diabetic foot ulcers. Journal of Wound Care. 1994;3(5):224 - 228. **6.** Smith & Nephew 2018. Project Peridot Testing - ALLEVYN Non-Adhesive dressings. Internal Report. DS/18/264/R. **7.** Franks PJ, Moody M, Moffatt CJ, et al. Randomised Trial of Two Foam Dressings in the Management of Chronic Venous Ulceration. Wound Repair and Regeneration. 2007;15(2):197 - 202. **8.** Smith+Nephew 2022. ALLEVYN Non-Adhesive (Inc. ALLEVYN Heel) Design Verification of Usability. Internal Report. W/02/119/B/R3. **9.** Smith+Nephew 2020. Verification testing of simulated use of ALLEVYN Tracheostomy. Internal Report. W/02/119/R1. **10.** Smith & Nephew 2018. Project Peridot Testing - ALLEVYN Non-Adhesive dressings. Internal Report. DS/18/264/R. **11.** Franks PJ, Moody M, Moffatt CJ, et al. Randomised Trial of Two Foam Dressings in the Management of Chronic Venous Ulceration. Wound Repair and Regeneration. 2007;15(2):197- 202.

ALLEVYN[◊] Ag Antimicrobial Foam Dressings

Combining the benefits of the ALLEVYN Range with added **antimicrobial activity of silver sulfadiazine (SSD)**,¹⁻³ the ALLEVYN Ag Range is **designed for patient comfort** when infection is a concern.⁴⁻⁷



Page 12:

1. Smith+Nephew 2007. Antimicrobial Activity of ALLEVYN Ag Adhesive Dressing Over 7 Days. Internal Report. DOF 0703014. **2.** Smith+Nephew 2009. The equivalence of the antimicrobial efficacies of the ALLEVYN Ag foam variants against key wound pathogens. Internal Report. RR-WMP07330-10-03. **3.** Smith+Nephew 2007. Antimicrobial Activity of Allevyn Ag Non-Adhesive Dressing against a Broad Spectrum of Microorganisms. Internal Report. DOF 0703006. **4.** Foster AVM, Greenhill MT, Edmonds ME. Comparing two dressings in the treatment of diabetic foot ulcers. *Journal of Wound Care*. 1994;3(5):224 - 228. **5.** Kurring PA, Roberts CD, Quinlan D. Evaluation of a hydrocellular dressing in the management of exuding wounds in the community *British Journal of Nursing* 1994;3(20):1049-1050, 1052-1043. **6.** Smith+Nephew 2008. A multi-centre in-market evaluation of ALLEVYN Ag dressings. Internal Report. SR/CIME/009. **7.** Smith+Nephew 2008. An open, prospective, randomised, within volunteer comparison of Dakota and Mepilex border. Internal Report. HVT050/ST788. **8.** Smith+Nephew 2007. Antimicrobial Activity of Allevyn Ag Non-Adhesive Dressing over 7 days. Internal Report. DOF 0703003. **9.** Smith+Nephew 2007. Antimicrobial Activity of ALLEVYN Ag Adhesive Dressing Against a Broad Spectrum of Microorganisms. Internal Report. DOF 0703007. **10.** Vaughan K, Woodmansey, E. ALLEVYN Ag Adhesive - Claims Support: Reduces Wound Odour by Reducing the Amount of Odour Causing Organisms in the Wound. In. Internal Statement: Smith+Nephew; 2011. **11.** Vaughan K, Woodmansey E. Allevyn Ag Non-Adhesive - Claims Support: Reduces Wound Odour by Reducing the Amount of Odour Causing Organisms in the Wound. In. Internal Statement: Smith+Nephew; 2011. **12.** Smith+Nephew 2018. A reduced bacterial count reduces the levels of MMP, which are causative of prolonged/chronic inflammation. Internal Statement. EO.AWM.PCSgen.003.v1.