





Pump status and troubleshooting

PICO 7 has visual indicators to let the user know when there is an issue. PICO 7 does not contain audible alerts. The pump should be carried so that it is accessible and the patient/healthcare professional can check the status routinely in case there is a fault or in case of damage.

Display/indicator status	Possible cause	Comments/troubleshooting
<p>All indicators off</p> 	<p>The pump is in standby.</p> <p>The pump has completed its course of NPWT.</p> <p>The batteries have depleted.</p>	<p>NPWT is paused. Press the orange button to restart NPWT.</p> <p>Pressing the orange button will not restart NPWT. Healthcare professional to apply new pump and dressing if further NPWT is required.</p> <p>If the pump has not yet completed its course of NPWT, replace the batteries.</p>
<p>Green 'OK' and orange 'leak' indicators flash</p> 	<p>The pump is working to achieve NPWT but has not reached the intended pressure.</p>	<p>Wait up to 65 seconds. Assess whether NPWT has been established.</p>
<p>Green 'OK' indicator flashes</p> 	<p>System is functioning properly. No issues.</p>	<p>The pump may be heard running occasionally as it maintains the negative pressure. This is normal.</p>
<p>Green 'OK' and orange 'battery low' indicators flash</p> 	<p>System is functioning properly but the batteries are low.</p>	<p>Replace the batteries and press the orange button to restart therapy.</p>
<p>Orange 'leak' indicator flashes</p> 	<p>A high air leak has been detected. NPWT is not being applied.</p> <p>Note: the pump will automatically try to restart NPWT after 1 hour.</p>	<p>Smooth down the dressing and strips to remove any creases. Ensure that the tube connectors have been twisted together securely. Press the orange button to restart NPWT. The green OK indicator and orange air leak indicator will start to flash together (indicates pump working to re-establish NPWT). Depending on the size of the wound, the pump should take up to 65 seconds to re-establish NPWT. If the air leak remains, the orange 'leak' indicator will flash again after approximately 65 seconds. If the air leak persists, contact healthcare professional.</p>

Display/indicator status	Possible cause	Comments/troubleshooting
<p>Orange 'leak' and orange 'battery low' indicators flash</p> 	<p>A high air leak has been detected and the batteries are low. NPWT is not being applied.</p> <p>Note: the pump will automatically try to restart NPWT after 1 hour.</p>	<p>Resolve the air leak according to instructions detailed overleaf. Also replace the batteries and press the orange button to restart therapy.</p>
<p>Orange 'dressing full' indicator flashes</p> 	<p>Dressing is saturated or filter is blocked. NPWT is not being applied.</p> <p>Note: the pump will automatically try to restart NPWT after 1 hour.</p>	<p>Healthcare professional to replace the dressing with a new one and press the orange button to restart NPWT.</p>
<p>Orange 'dressing full' and orange 'battery low' indicators flash</p> 	<p>Dressing is saturated or filter is blocked and the batteries are low. NPWT is not being applied.</p> <p>Note: the pump will automatically try to restart NPWT after 1 hour.</p>	<p>Healthcare professional to replace the dressing with a new one. Also replace the batteries and press the orange button to restart therapy.</p>
<p>All indicators solidly illuminated</p> 	<p>A pump error has been detected. The pump can no longer apply NPWT.</p>	<p>Healthcare professional to apply a new pump and dressing.</p>



For detailed product information, including indications for use, contraindications, effects, precautions, warnings, and important safety information, please always consult product Information for Use (IFU) prior to use.

Supporting healthcare professionals for over 150 years

Smith & Nephew Pty Ltd
Australia
 T +61 2 9857 3999 F +61 2 9857 3900
www.smith-nephew.com/australia
Customer Service
 T 13 13 60 F 1800 671 000

Smith & Nephew Ltd
New Zealand
 T +64 9 820 2840 F +64 9 820 2841
www.smith-nephew.com/new-zealand
Customer Service
 T 0800 657 799 (Surgical) T 0800 807 663 (Wound)

®Trademark of Smith & Nephew
 10402_AU V1 11/17 SN13683