


Hip Reference Guide

Step-by-step guide



1. Expose the bone at the desired insertion location for the implant. Attach the appropriate drill bit to a standard orthopedic drill. Be sure to chuck the drill no farther than the flats on the proximal end of the drill.

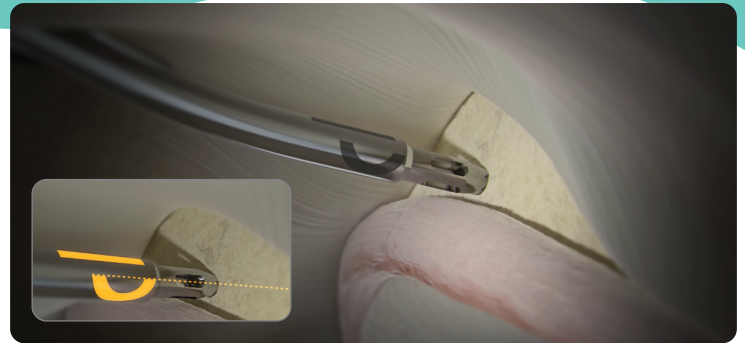

2. Position the tip of the Drill Guide with Obturator against the bone surface at the desired position then remove the Obturator. The crescent laser mark indicates the direction of the curve, while the vertical laser mark helps project the trajectory of the drill bit.




3. Insert the drill into the drill guide and drill the bone hole. Advance the drill until the drill bottoms out in the drill guide.


CAUTION: Do not use the drill without the drill guide as the bone surface can be damaged. Confirm that bone hole is clear of bone debris.



4. Remove the drill and insert the implant through the drill guide into the bone hole. Be sure that the implant inserter is fully engaged with the drill guide throughout the deployment of the anchor. Be sure to maintain the alignment of the drill guide to the bone hole during insertion.

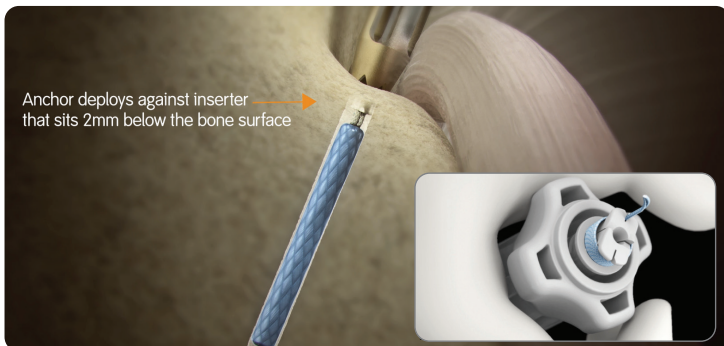

5. Rotate the activation knob clockwise on the proximal end of the inserter to deploy the implant. Rotate the knob until a hard stop is reached and the suture cleat is fully exposed at the proximal end of the inserter.

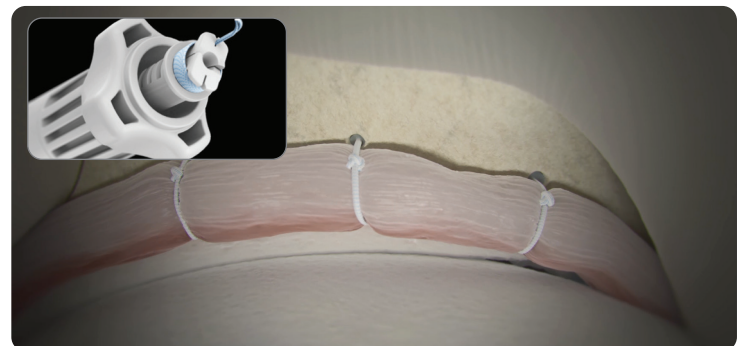

6. Release the sutures from the cleat and remove the inserter and drill guide. Pull on the suture tails to ensure that the implant is set in the bone hole. If placing more than one implant, repeat steps 2 through 5 to complete repair.

Ordering information

Q-FIX[®] All-Suture Anchors

Reference #	Description
1.8mm Q-FIX MINI All-Suture Anchor	
Anchors	
72290123	1.8mm Q-FIX MINI All-Suture Anchor
Reusable Instruments	
25-1812	1.8mm Reusable PATHFINDER [®] Obturator*
Disposable Kits	
72290125	1.8mm Q-FIX MINI Disposable Kit
72290126	1.8mm Q-FIX MINI XL Disposable Kit
1.8mm Q-FIX All-Suture Anchor	
Anchors	
25-1800	1.8mm Q-FIX All-Suture Anchor
Reusable Instruments	
72290119	1.8mm Reusable Curved Drill Guide, XL
72290120	1.8mm Reusable Curved Drill Guide
25-1812	1.8mm Reusable PATHFINDER Obturator*
Disposable Kits	
25-1810	1.8mm Q-FIX Disposable Kit
25-1811	1.8mm Q-FIX Disposable Kit XL
Disposable Accessories	
72290118	1.8mm Disposable Flex Drill
2.8mm Q-FIX All-Suture Anchor	
Anchors	
25-2800	2.8mm Q-FIX All-Suture Anchor
Reusable Instruments	
25-2812	2.8mm Reusable PATHFINDER Obturator*
25-2820	2.8mm Reusable Bone Punch for Q-FIX All-Suture Anchor
Disposable Kits	
25-2810	2.8mm Q-FIX Disposable Kit

*The PATHFINDER may be used to confirm bone hole location and depth.

All photos and diagrams are for illustrative purposes only. Products not to scale.

For more information on the application of any products discussed in the presentation, as well as indications for use, contraindications, and product safety information, please consult the Instructions for Use (IFU) for such product.

Smith & Nephew Pty Ltd
Australia
T +61 2 9857 3999
F +61 2 9857 3900
smith-nephew.com/australia

Smith & Nephew Ltd
New Zealand
T +64 9 820 2840
F +64 9 820 2841
smith-nephew.com/new-zealand

◇Trademark of Smith+Nephew
All trademarks acknowledged
©2021 Smith+Nephew
11939-anz V2 01/21