

Implant Card training: Un-boxing, delivery methods, and patient leaflet

What is the Implant Card?

The Implant Card is required by the EU/ANZ to be shipped with the device and given to the patient. The Implant Card contains device specific information with a link to the eIFU website for patients to obtain the Implant Leaflet and view additional device information.

Un-boxing the Implant Card

Un-box the Implant device and retrieve the Chartstik labels. The Implant Cards are labels and appear as the first two labels

i Note: If no Implant Card is available, please use the Chartstik label

DO NOT discard the Implant Cards or the Chartstik labels

Health care Provider (HCP) must save the Implant Cards and Chartstik labels in the patient folder




Delivery Methods

What are the Implant Card delivery methods?

The Implant Card delivery methods are comprised of two (2) printed pieces, an accordion folder and a sleeve. The accordion can hold up to 7 Implant Cards and the sleeve is the protective pouch for the accordion.

i Note: these delivery steps are not mandatory, however, HCP needs to ensure the Patient is provided the Implant Card/ Charstik label.

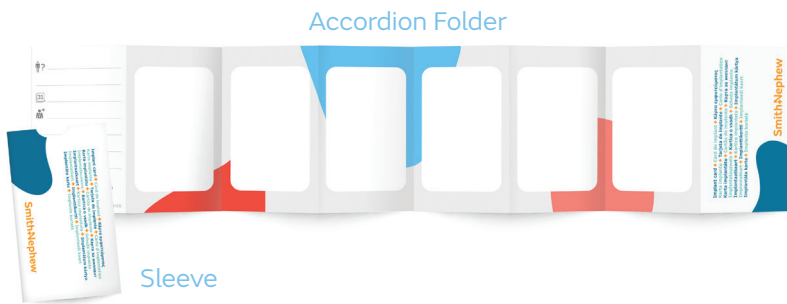

HCP adheres the Implant Card/Chartstik labels to the accordion folder

i Note up to 7 Implant Cards will fit on the accordion folder

HCP fills in the Patient, Surgery Information, and Manufacturer name and address on the folder


John Doe	Patient Name/ Identification
12/07/2020	Date of Implantation
Dr. Smith Great Healthcare Memphis, TN	Surgeon/Healthcare facility name and Address

Place the accordion folder into the S+N Implant Card Sleeve

HCP provides the Implant Card(s)/Chartstik labels in the sleeve to the patient



Smith+Nephew

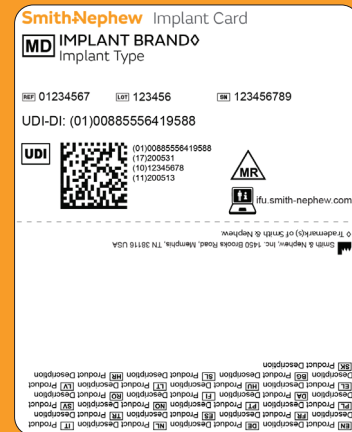

Patient leaflet

HCP directs patient to visit the S+N eFU website to view and download the device specific Patient Implant Leaflet

Implant device information is available to patients via the eFU website ifu.smith-nephew.com located on the accordion folder.

HCP to direct patient to the eFU website for further information about their Smith+Nephew Implanted Device

Navigate to the eFU website found on the accordion folder and select your **Language**

Select "Country" and enter the REF# from the Implant Card into the **Product Search** box

Select "I am a" **Patient**

Select **Accept and Continue** and **Search**

The Implant Leaflet will appear below the search area

Click on the language of the PDF to open or save a copy

Print and save the leaflet with the Implant Card(s)

Smith+Nephew

