

+ A unique quadrilobe shape with wide borders allows it to stay in place for longer*¹⁻³ and a change indicator lets you know when to change the dressing.^{3,4}


Smith+Nephew

ALLEVYN[®] LIFE
Foam Dressing

Application



- Peel the protectors from the centre, without removing completely
- Centre the dressing
- Anchor some of the border on the peri-wound
- You can use your fingers to help the anchoring of the border
- Take hold of one of the partially peeled protectors

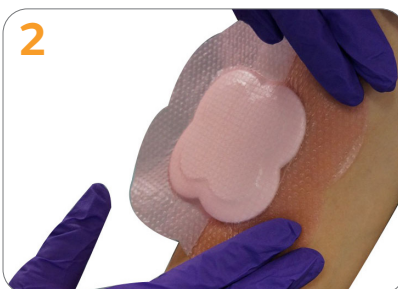


- Remove remaining Liner / Liners
- Smooth over.
- If on a wound ensure absorbent pad covers the wound.
- Do not stretch the dressing.



- Gently shape the pad on/around the body part, without stretching it
- Gently peel away the other two lobes of protector, while laying the border down
- Do not stretch the border, pat it down

ALLEVYN LIFE Foam Dressings mask excess fluid from the wound and also help to keep potential leaks or odours locked away.^{2,4,6}

Here's a helpful diagram to show you when to get the dressing changed:

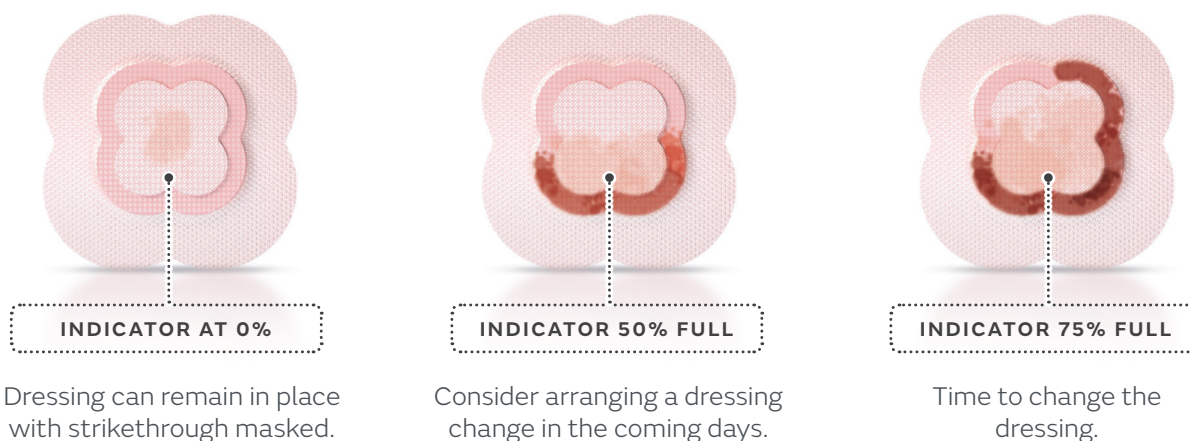

ALLEVYN LIFE DRESSING ordering codes



Size (cm)	S+N Code	NHSCC Code	PIP Code	Carton
10.3cm x 10.3cm	66801067	ELA607	373-9448	10
12.9cm x 12.9cm	66801068	ELA608	373-9430	10
15.4cm x 15.4cm	66801069	ELA609	373-9380	10
21cm x 21cm	66801070	ELA610	373-9364	10



Size (cm)	NHSCC Code	PIP Code	S+N Code	Carton
17.2cm x 17.5cm Small Sacrum	ELA645	380-7575	66801306	10
21.6cm x 23cm Large Sacrum	ELA646	380-7583	66801307	10
25cm x 25cm Heel	ELA650	382-9710	66801304	5

For detailed product information, including indications for use, contraindications, precautions and warnings, please consult the product's applicable Instructions for Use (IFU) prior to use.

*ALLEVYN LIFE can be worn almost twice as long as standard dressings. n=37; dressing retention was 1.92x longer.

References 1. Joy H, Bielby A, Searle R. A collaborative project to enhance efficiency through dressing change practice. J Wound Care. 2015;24(7):312, 314 - 317. 2. Rossington A, Drysdale K, Winter R. Clinical performance and positive impact on patient wellbeing of ALLEVYN Life. Wounds UK. 2013;9(4):91 - 95. 3. Simon D, Bielby A. A structured collaborative approach to appraise the clinical performance of a new product. Wounds UK. 2014;10(3):80 - 87. 4. Tiscar-González V, Menor-Rodríguez MJ, Rabadán-Sainz C, et al. Clinical and Economic Impact of Wound Care Using a Polyurethane Foam Multilayer Dressing. Adv Skin Wound Care. 2021;34(1):23-30. 5. Smith+Nephew 2016.Wound Model Testing of New ALLEVYN Life Gen2 wcl Dressing using Horse Serum at a Flow Rate Modelling that of a Moderately Exuding Wound. DS/14/303/R. 6. Smith+Nephew 14 June 2012. Odour reducing properties of ALLEVYN Life. Internal Report. DS/12/127/DOF.