ULTRABRAID° Suture

Smith & Nephew Orthopaedics GmbH
Alemannenstraße 14
78532 Tuttlingen
Germany
ec.rep@smith-nephew.com

ULTRABRAID° Suture

ULTRABRAID-Nahtmaterial

Sutura ULTRABRAID

Fil de suture ULTRABRAID

Sutura ULTRABRAID

ULTRABRAID sutur

ULTRABRAID hechtdraad

Sutura ULTRABRAID

ULTRABRAID sutur

ULTRABRAID-sutura

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Instructions for Use

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Mode d'emploi

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Bruksanvisning

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Brugsanvisning

Bruksanvisning

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MR (magnetic resonance) safe. • MR-geeignet (Magnetresonanz). • Seguro para procedimientos de RM (resonancia magnética). • Conforme aux normes de sécurité sur la résonance magnétique. • Sicuro per la RM (risonanza magnetica). • MR-säker (magnetisk resonans). • MR (magnetische resonantie) veilig. • Seguro em termos de RM (ressonância magnética). • MR (magnetisk resonans)- sikker. • MR-sikkert utstyr (magnetisk resonanstomografi). • MR(자기 공명) 안전.

Taper • Taper • Sin filo • Pointe • Conico • Avsmalnande • Taps toelopend • Cone alargado • Konusformet • Skjærende nål • 침형

www.smith-nephew.com
+1 978 749 1000
+1 978 749 1108 Fax
+1 800 343 5717 U.S. Customer Service

English

ULTRABRAID° Suture

Device Description

The Smith & Nephew ULTRABRAID° Suture is a non-absorbable, sterile, surgical suture composed of white and/or blue UHMW polyethylene. The ULTRABRAID Suture meets or exceeds USP standards, except for diameter.

This device is provided sterile, for single use only.

How Supplied

The ULTRABRAID Suture is supplied braided in sterile, pre-cut lengths. The suture may be provided with suture ends preattached to needles in a variety of sizes or styles.

Indications for Use

The Smith & Nephew ULTRABRAID Suture is indicated for use in approximation and/or ligation of soft tissues, including allograft tissue for orthopedic surgeries.

Contraindications

None known.

⚠ Warnings

 **Do not use if package is damaged. Do not use if the product sterilization barrier or its packaging is compromised.**

- Contents are sterile unless package is opened or damaged. DO NOT RSTERILIZE. For single use only. Discard any open, unused product. Do not use after the expiration date.**

- It is the surgeon's responsibility to be familiar with the appropriate surgical techniques prior to use of this device.**

- Read these instructions completely prior to use.**

- As with any foreign body, prolonged contact of this or any other suture with salt solutions, such as those found in the urinary or biliary tracts, may result in calculus formation.**

Precautions

R_{ONLY} U.S. Federal law restricts this device to sale by or on the order of a physician.

- Hazards associated with reuse of this device include, but are not limited to, patient infection and/or device malfunction.

- Prior to use, inspect the device to ensure it is not damaged. Do not use a damaged device.

- In handling this or any other suture material, care should be taken to avoid damage from handling. Avoid crushing or crimping damage due to the application of surgical instruments such as forceps or needle holders.

- Adequate knot security requires the accepted surgical technique of flat, square ties, with additional throws as warranted by surgical circumstance and the experience of the surgeon.

- After use, this device may be a potential biohazard and should be handled in accordance with accepted medical practice and applicable local and national requirements.

Adverse Reactions

The Smith & Nephew ULTRABRAID Suture elicits a minimal acute inflammatory reaction in tissues, followed by gradual encapsulation of the suture by fibrous connective tissue.

Adverse effects associated with the use of this device include: wound dehiscence, calculi formation in urinary and biliary tracts when prolonged contact with salt solutions such as urine and bile occurs, infected wounds, minimal acute inflammatory tissue reaction, and transitory local irritation.

Warranty

For single use only. This product is warranted to be free from defects in material and workmanship. Do not reuse.

For Further Information

If further information on this product is needed, contact Smith & Nephew Customer Service at +1 800 343 5717 in the U.S., or an authorized representative.

°Trademark of Smith & Nephew, registered U.S. Patent & Trademark Office.
©2010, 2018 Smith & Nephew, Inc.
All rights reserved.

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^[1] Varumärke som tillhör Smith & Nephew, och har registrerats vid U.S. Patent and Trademark Office (USA:s Patent och registreringsverk).