SmithNephew

Q-FIX^o Suture Anchor demonstrates clinically relevant reductions in displacement compared to other all-suture anchors

Plus points

Compared with other all-suture anchors tested, Q-FIX Suture Anchors showed:



Overview

- Biomechanical study evaluating the cyclic and load to failure properties of four all-suture anchors for glenoid labral repair:
- Q-FIX 1.8mm Suture Anchor
- SUTUREFIX<sup>
 </sup> ULTRA 1.7mm Suture Anchor
- Iconix 1 Suture Anchor (Stryker Endoscopy, CA, USA)
- JuggerKnot[®] 1.5mm Suture Anchor (Zimmer Biomet, IN, USA)

Results

- Only Q-FIX Suture Anchors had no clinical failures during cyclic loading (Table)
- Q-FIX Suture Anchors had significantly less displacement after 100 cycles and 200 cycles compared to all other anchors tested (p<0.001)
- Iconix anchors had a significantly lower load to failure (143.5N) than Q-FIX anchors (191.3N; p=0.01), SUTUREFIX ULTRA anchors (188.3N; p=0.012) and JuggerKnot anchors (183.6N; p=0.021)

- Anchors were inserted into 13 matched pairs of human cadaveric glenoids before cyclic loading and a pull-to-failure test
- Displacement, load to failure and failure mode were recorded
- Clinical failure was defined as displacement >5mm
- Catastrophic failure was defined as anchor pull-out during cyclic loading

Table. Failure mode during cyclic loading

All-suture anchor		Catastrophic failures	Clinical failures
Q-FIX (n=25)	1	1	0
SUTUREFIX ULTRA (n=22)	9	4	5
lconix (n=23)	9	3	6
JuggerKnot (n=23)	7	3	4

Conclusions

Citation

*Ruder JA, Dickinson EY, Peindl RD, Habet NA, Trofa DP, Fleischli JE. Cyclic and load-to-failure properties of all-suture anchors in human cadaveric shoulder glenoid bone. Arthroscopy. 2019;35(7):1954-1959. Available at: Arthroscopy

Q-FIX is manufactured by ArthroCare Corporation and distributed by Smith & Nephew, Inc. ArthroCare Coporation, 7000 West William Cannon Drive, Austin, TX 78735. SUTUREFIX ULTRA is manufactured by S+N.

Sports Medicine, Smith & Nephew, Inc. 150 Minuteman Road, Andover, MA 01810 USA. 21867-en V1. Published November 2019. ©2019 Smith+Nephew. All rights reserved. ◊Trademark of Smith+Nephew. All trademarks acknowledged.