

# IV3000 Application

IV3000 Non-Ported Dressing 6cm x 7cm


Suitable for dressing peripheral catheters

## Step A




Clean and thoroughly dry the application area in accordance with local protocols. First remove section 1 from the rounded end of the dressing. This contains two sterile strips and a date label.

## Step B

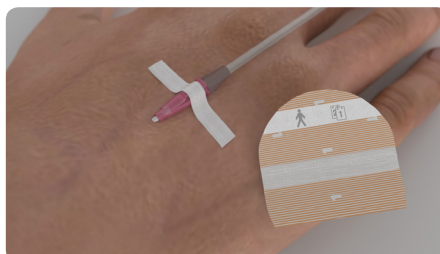


The sterile strips can be used on the catheter wings or body for improved security, to secure tubing or to stabilize catheter lumens.

## Step C

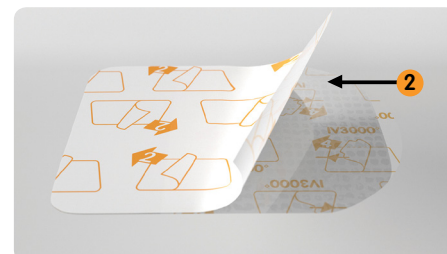

Remove the backing paper marked 2.

## Step D

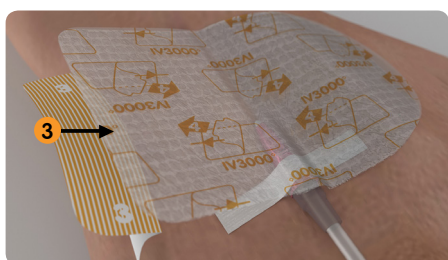


Centre the dressing over the catheter insertion site.

Place the dressing on the catheter port and adhere the dressing to the patient's skin.

Remove the backing paper marked 3.


## Step E

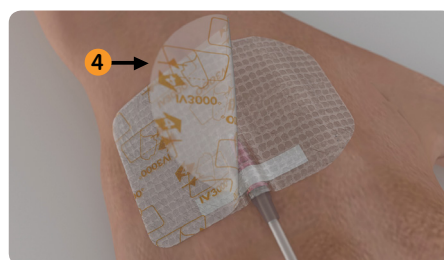

Smooth the edges of the dressing.


Using the tab, gently peel off the top backing paper marked 4.

## Step F

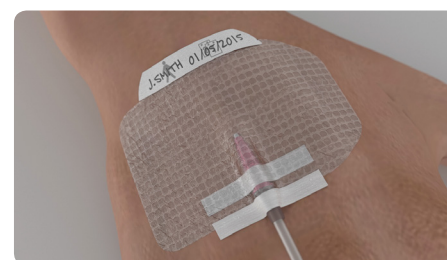

Mould the film around the catheter, ensuring that the dressing is secure and the patient is comfortable.

Finally, after recording relevant information on the date label, place the date label from section 1 on the skin or the dressing (without obscuring the view of the catheter site).

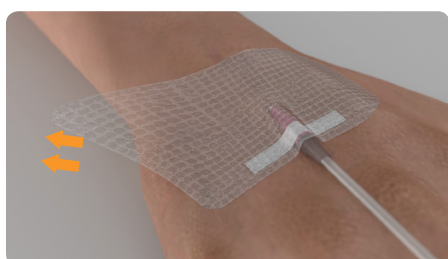


## Removal

Take a corner of the film and stretch it parallel to the patient's skin while stabilizing the patient's skin and catheter with the other hand.

Release and repeat the process.

## Ordering codes

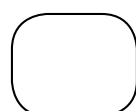

Product code	Dressing size	Quality per carton	Recommended indication
4007	6cm x 7cm	100	Peripheral
4008	10cm x 12cm	50	Central

Please consult product labels and inserts for any indications, contraindications, hazards, warnings, cautions and instructions for use.

Smith & Nephew Pty Ltd  
Australia  
T +61 2 9857 3999 F +61 2 9857 3900  
www.smith-nephew.com/australia  
Customer Service  
T 13 13 60 F 1800 671 000

Smith & Nephew Ltd  
New Zealand  
T +64 9 820 2840 F +64 9 820 2841  
www.smith-nephew.com/new-zealand  
Customer Service  
T 0800 657 799 (Surgical) T 0800 807 663 (Wound)