



# Shoulder Reference Guide

## Step-by-step guide

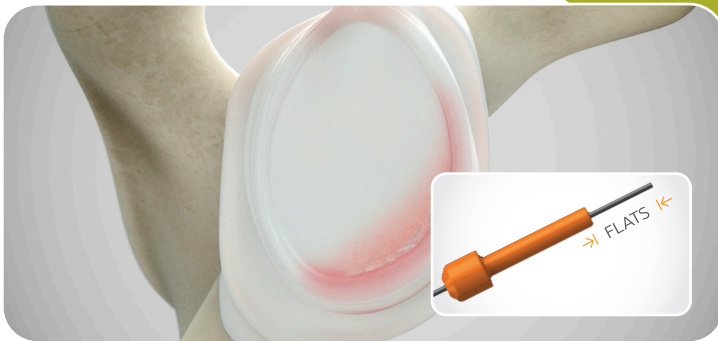

1. Expose the bone at the desired insertion location for the implant. Attach the appropriate drill bit to a standard orthopedic drill. Be sure to chuck the drill no farther than the flats on the proximal end of the drill.

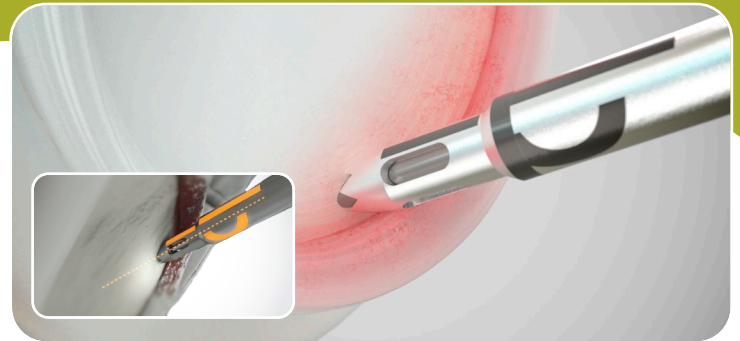

2. Position the tip of the Drill Guide with Obturator against the bone surface at the desired position then remove the Obturator. If using the curved guide, the crescent laser mark indicates the direction of the curve, while the vertical laser mark helps project the trajectory of the drill bit.




3. Insert the drill into the drill guide and drill the bone hole. Advance the drill until the drill bottoms out in the drill guide.


**CAUTION:** Do not use the drill without the drill guide as the bone surface can be damaged. Confirm that bone hole is clear of bone debris.



4. Remove the drill and insert the implant through the drill guide into the bone hole. Be sure that the implant inserter is fully engaged with the drill guide throughout the deployment of the anchor.

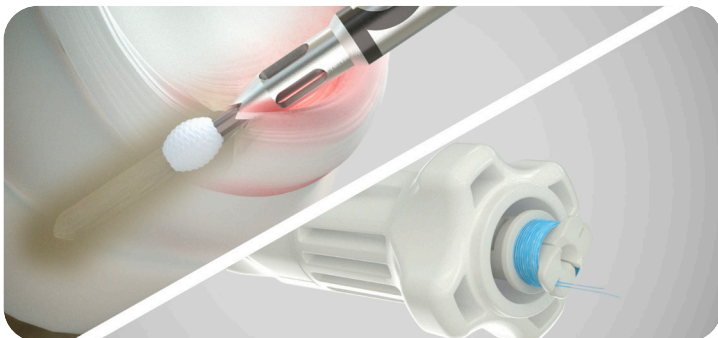

5. Rotate the activation knob on the proximal end of the inserter clockwise. Rotate the knob until a hard stop is reached and the suture cleat is fully exposed.

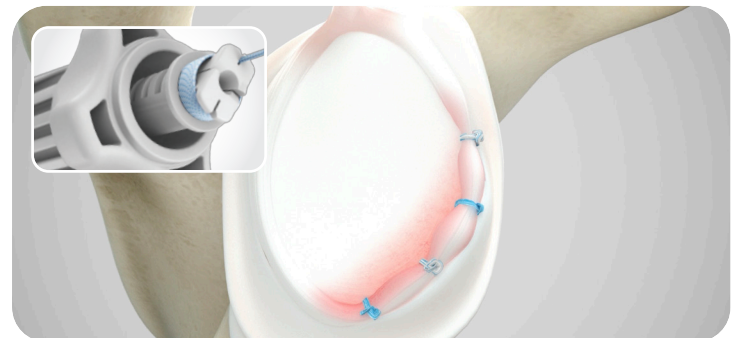

6. Release the sutures from the cleat and remove the inserter and drill guide. If placing more than one implant, repeat steps 2 through 5 to complete repair.

## Ordering information

### Q-FIX<sup>®</sup> All-Suture Anchors Pre-loaded with MINITAPE<sup>®</sup> Suture

Reference	Description
72205858	Q-FIX All-Suture Anchor, 1.8mm, with one MINITAPE Suture, cobraid-white
72205859	Q-FIX All-Suture Anchor, 1.8mm, with one MINITAPE Suture, blue

### Q-FIX All-Suture Anchors Pre-loaded with MAGNUMWIRE Suture

Reference	Description
72290123	Q-FIX MINI All-Suture Anchor, 1.8mm, with one MAGNUMWIRE Suture, cobraid-blue
25-1800	Q-FIX All-Suture Anchor, 1.8mm, with one MAGNUMWIRE Suture, cobraid-blue

### Q-FIX Disposable Kits

Reference	Description
72290125	Q-FIX MINI All-Suture Anchor Disposable Kit, 1.8mm, includes drill, fish mouth tip drill guide and sharp obturator
25-1810	Q-FIX All-Suture Anchor Disposable Kit, 1.8mm, includes drill, fish mouth tip drill guide and sharp obturator

### Q-FIX Reusable Accessories

Reference	Description
72290032	Q-FIX All-Suture Anchor Drill Guide, fish mouth tip, 1.8mm, nonsterile, reusable
72290120	Q-FIX All-Suture Anchor Curved Drill Guide, crown tip, 1.8mm, nonsterile, reusable
72290033	Q-FIX All-Suture Anchor Obturator, 1.8mm, sterile, disposable

### Q-FIX Disposable Accessories

Reference	Description
72290124	Q-FIX MINI All-Suture Anchor Twist Drill, 1.8mm, sterile, disposable
72290030	Q-FIX All-Suture Anchor Twist Drill, 1.8mm, sterile, disposable
72290118	Q-FIX All-Suture Anchor Flexible Drill, 1.8mm, sterile, disposable

## Additional instruction

All photos and diagrams are for illustrative purposes only. Products not to scale.

For more information on the application of any products discussed in the presentation, as well as indications for use, contraindications, and product safety information, please consult the Instructions for Use (IFU) for such product. To order these instruments, call **+1 800 343 5717** in the U.S. or contact an authorized Smith+Nephew representative.

Products may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Please contact your Smith+Nephew representative or distributor if you have questions about the availability of Smith+Nephew products in your area.

Learn more at [smith-nephew.com](https://www.smith-nephew.com)