


A route to effective exudate management

Improve patient outcomes¹ with accurate decision making, fast response and effective treatment choices

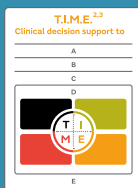

Start with the following steps to undertake a comprehensive assessment²⁻⁴

- A** Assess patient, wellbeing and wound
- B** Bring in a multi-disciplinary team and informal carers to promote holistic patient assessment
- C** Control and treat the underlying causes and barriers to wound healing
- D** Decide appropriate treatment
- E** Evaluate and reassess the treatment and wound management outcomes

Low



- Appearance** - no visible maceration. Dressing has light strike through on removal
- Action** to maintain a moist healing environment

Moderate



- Wounds that are pre-assessed as needing up to **2-3 dressings per week**
- Appearance** - potential visible peri wound maceration. Dressings may have moderate strike through
- Action** to absorb exudate and maintain a moist wound healing environment

High*



- Wounds that are pre-assessed as needing **4-7 weekly dressing changes**
- Appearance** - Peri wound maceration very likely. Dressings may be leaking
- Action** to absorb exudate and maintain a moist wound healing environment

For low to moderate exudate levels consider the use of:

ALLEVYN GENTLE
BORDER LITE
Foam Dressing




ALLEVYN
GENTLE
Foam Dressing



ALLEVYN
GENTLE BORDER
Foam Dressing



ALLEVYN
NON-ADHESIVE
Foam Dressing

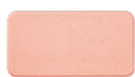

ALLEVYN
ADHESIVE
Foam Dressing



For wounds that require more than 2 dressings per week consider the use of **ALLEVYN LIFE Foam Dressing**

(depending on exudate levels ALLEVYN LIFE Foam Dressing has a 7-day wear time^{†5-7} which can promote undisturbed wound healing⁸ and wound care optimisation)^{9**}

ALLEVYN LIFE
Foam Dressing



When to change
ALLEVYN
GENTLE BORDER
Foam Dressing?



No need to change



No need to change




Exudate within 0.5cm from the edge of the dressing pad or at 7 days whichever is sooner, **dressing requires changing**

When to change
ALLEVYN LIFE
Foam Dressing?




INDICATOR AT 0%

Dressing can remain in place with strikethrough masked

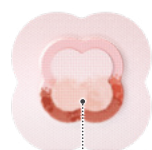

INDICATOR 50% FULL

Consider arranging a dressing change in the coming days

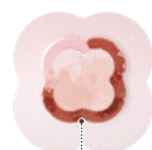

INDICATOR 75% FULL

Time to change the dressing

This pathway has been adapted from Checketts C. et al. Service redesign: how a tissue viability specialist lead nurse developed and improved acute wound care through real-world evidence and partnership working. Brit J Health Care Manag. 2021;27(5):116-125.

*Assess for infection as high amounts of exudate may be a sign of wound infection. **for wounds that need more than 1 dressing per week. †Up to 5 days for the sacral area

References 1. WUWHS Consensus Document. Wound exudate effective assessment and management. Wounds International. London (2019). 2. Schultz GS, Sibbald RG, Falanga V et al., Wound Rep Reg (2003);11:1-28. 3. Leaper DJ, Schultz G, Carville K, Fletcher J, Swanson T, Drake R. Extending the TIME concept: what have we learned in the past 10 years? Int Wound J 2012; 9 (Suppl. 2):1-191. 4. Moore Z, et al. J Wound Care 28(3):154-161 (2019). 5. Smith+Nephew 2016.Wound Model Testing of New ALLEVYN Life Gen2 wcl Dressing using Horse Serum at a Flow Rate Modelling that of a Moderately Exuding Wound. DS/14/303/R. 6. Smith+Nephew 2016.Product Performance of Next Generation ALLEVYN Life Internal Report. (HVT080) GMCA-DOF/08. 7. Lisco C. Evaluation of a new silicone gel-adhesive hydrocellular foam dressing as part of a pressure ulcer prevention plan for ICU patients. Paper presented at: WOCN; 2013. 8. Rippon M, Davies P, White R. Taking the trauma out of wound care: The importance of undisturbed healing. Journal of Wound Care. 2012; 21: 359-368. 9. Best Practice Statement: Optimising wound care. Wounds UK, Aberdeen, 2008.

Smith & Nephew Pty Ltd (Australia) smith-nephew.com/en-au. Smith & Nephew Ltd (New Zealand) smith-nephew.com/en-nz. ©Trademark of Smith+Nephew. All trademarks acknowledged. © January 2025 Smith+Nephew. 35986-anz V1 REVA 01/25. This material is intended for healthcare professionals. For detailed product information, including indications for use, contraindications, precautions and warnings, please consult the product's applicable Instructions for Use (IFU) prior to use.

Ordering information

ALLEVYN® LIFE Dressings

Code	Size	Shape	Pad size	Pad area	Qty
66801067	10.3cm x 10.3cm	Square	5.1cm x 5.1cm	22.7cm ²	10
66801068	12.9cm x 12.9cm	Square	7.6cm x 7.6cm	51.8cm ²	10
66801069	15.4cm x 15.4cm	Square	10.2cm x 10.2cm	92.9cm ²	10
66801070	21cm x 21cm	Square	15.4cm x 15.4cm	210.5cm ²	10
66801304	25cm x 25.2cm	Heel	20cm x 20.2cm	242.7cm ²	5
66801306	17.2cm x 17.5cm	Sacrum	8.5cm x 12.4cm	86.9cm ²	10
66801307	21.6cm x 23cm	Sacrum	12.3cm x 17cm	172.8cm ²	10

ALLEVYN GENTLE BORDER Dressings

Code	Size	Shape	Pad size	Pad area	Qty
66800269	7.5cm x 7.5cm	Square	5cm x 5cm	22.6cm ²	10
66800270	10cm x 10cm	Square	7.5cm x 7.5cm	51.8cm ²	10
66800900	10cm x 20cm	Rectangular	7cm x 17cm	116.3cm ²	10
66800264	10cm x 25cm	Rectangular	6.5cm x 21.8cm	135.9cm ²	10
66800265	10cm x 30cm	Rectangular	6.5cm x 26.8cm	159.8cm ²	10
66800272	12.5cm x 12.5cm	Square	10cm x 10cm	92.9cm ²	10
66800273	17.5cm x 17.5cm	Square	15cm x 15cm	210.5cm ²	5
66800898	16.8cm x 17.1cm	Sacrum	8.5cm x 12.4cm	86.9cm ²	10
66800959	17.1cm x 17.9cm	Multisite	11.6cm x 12.3cm	100cm ²	10
66800506	23cm x 23.2cm	Heel	20cm x 20.2cm	242.7cm ²	5

ALLEVYN GENTLE BORDER LITE Dressings

Code	Size	Shape	Pad size	Pad area	Qty
66800833	5cm x 5cm	Square	3cm x 3cm	8.2cm ²	10
66800836	5.5cm x 12cm	Rectangular	3cm x 9.5cm	24.9cm ²	10
66801013	8.6cm x 7.7cm	Oval	6.1cm x 4.8cm	22.9cm ²	10
66801041	8cm x 8.4cm	Multisite	5.4cm x 5.8cm	21.9cm ²	10

ALLEVYN ADHESIVE Dressings


Code	Size	Shape	Pad size	Pad area	Qty
66000043	7.5cm x 7.5cm	Square	5cm x 5cm	22.6cm ²	10
66000599	10cm x 10cm	Square	7.5cm x 7.5cm	51.8cm ²	10
66000044	12.5cm x 12.5cm	Square	10cm x 10cm	92.9cm ²	10
66000045	17.5cm x 17.5cm	Square	15cm x 15cm	210.5cm ²	10
66000046	22.5cm x 22.5cm	Square	20cm x 20cm	374.6cm ²	10
66000451	22cm x 22cm	Sacrum	18.5cm x 18.1cm	232cm ²	10
66000700	17cm x 17cm	Sacrum	13.2cm x 13.5cm	123.4cm ²	10

ALLEVYN NON-ADHESIVE Dressings

Code	Size	Shape	Pad size	Pad area	Qty
66007643	5cm x 5cm	Square	5cm x 5cm	25cm ²	10
66007637	10cm x 10cm	Square	10cm x 10cm	100.3cm ²	10
66007335	10cm x 20cm	Rectangular	10cm x 20cm	204cm ²	10
66000093	15cm x 15cm	Square	15cm x 15cm	226cm ²	10
66007638	20cm x 20cm	Square	20cm x 20cm	400cm ²	10
66000663	40cm x 70cm	Rectangular	40cm x 70cm	2800cm ²	2
66007630	Heel	Heel	10.6cm x 13.6cm	200.2cm ²	5



Square

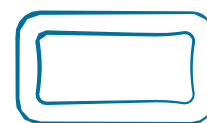

Rectangle



Quadrilobe



Heel



Sacrum



Sacrum



Oval



Multisite



Square
(Non-Adhesive)



Rectangle
(Non-Adhesive)



Heel
(Non-Adhesive)

Your local Smith+Nephew
representative's details:

Name:

Tel: