

Simple to apply.



Prepare the wound bed to ensure it is free of exudate and devitalized tissue.



Select the size and form of OASIS® Matrix needed.

OASIS Burn Matrix and OASIS ULTRA Tri-Layer Matrix: 7 x 20cm, 7 x 10cm, 5 x 7cm, 3 x 7cm, 3 x 3.5cm
OASIS Wound Matrix: 3 x 7cm, 3 x 3.5cm



*Product code for OASIS ULTRA Tri-Layer Matrix **Product code for OASIS Burn Matrix



Apply, secure and hydrate OASIS Matrix after placing it in a dry state over the wound.

Once OASIS has been secured to the wound, it should be covered with a non-adherent dressing and a secondary dressing. A moist environment is important for successful integration of OASIS.

Caramelization: What to expect



Successful integration of OASIS Matrix Products may form a caramel-colored or off-white gel.

Caramelization occurs as part of the natural wound progression process. Because this caramelization or off-white gel contains ECM components, which continue to replace deficient and missing ECM components in the wound, the caramelized material should not be removed.



Gently cleanse the wound surface with sterile saline, then apply new OASIS Matrix if necessary.