

+ Redefining shoulder repair

Smith+Nephew


RITEFIT[◇]
High Viz Cannula



RITEFIT[®] High Viz Cannula

Streamline your access

The RITEFIT cannula helps optimize procedural visibility and access, with a distal flange providing soft tissue retraction.¹ Designed for a precise, patient-specific fit, the ratcheting crown allows the cannula to adjust from 2 to 8.3cm — eliminating the need to measure for length and helping reduce inventory with fewer sizes to stock.¹

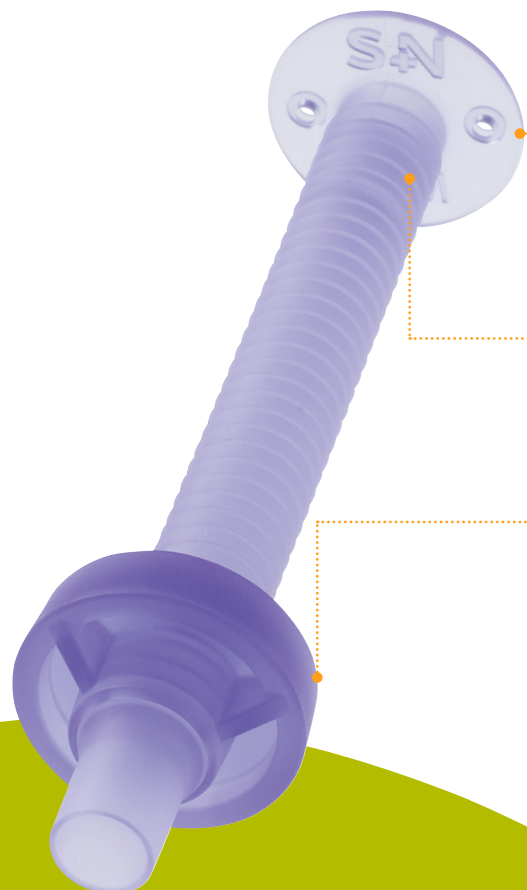

Distal flange

Low-profile design improves maneuverability within the joint compared to rigid cannulas

Double dams

Integrated dual-dam system manages fluid leakage during instrument and implant passage

Ratcheting crown

Adjustable for a patient-specific fit, with the ability to trim for a customized working length

Dual point stabilization

The crown and flange function in tandem to resist displacement throughout the procedure



RITEFIT[®] High Viz Cannula Reference Guide



1. Select the appropriate cannula diameter. Create a skin incision approximately 1mm larger than the cannula diameter, then dilate the soft tissue tract to facilitate smooth insertion. Fold the flange for insertion using a curved hemostat then advance it through the incision into the joint space.



2. Withdraw the insertion tool once the flange is fully seated intraarticularly. Adjust the crown so that it sits flush against the skin.



3. Cut the excess length, leaving approximately 1.5cm of the cannula extending above the crown to allow for intraoperative adjustment.

Ordering information

RITEFIT [®] High Viz Cannula	
Reference	Description
72205900	RITEFIT High Viz Cannula, 6mm
72205901	RITEFIT High Viz Cannula, 8mm
72205902	RITEFIT High Viz Cannula, 10mm

Products may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Please contact your Smith+Nephew representative or distributor if you have questions about the availability of Smith+Nephew products in your area. For detailed product information, including indications for use, contraindications, precautions and warnings, please consult the product's applicable Instructions for Use (IFU) prior to use.

Learn more at [smith-nephew.com](https://www.smith-nephew.com)

Smith & Nephew, Inc.
150 Minuteman Road
Andover, MA 01810
USA

www.smith-nephew.com
T +978 749 1000
US Customer Service:
+1 800 343 5717

[®]Trademark of Smith+Nephew.
All trademarks acknowledged.
©2025 Smith+Nephew. All rights reserved. 47400 V2 09/25

References

- 1. Smith+Nephew. Marketing Claims Technical Report 10151414, Flexible Cannula Rev. A