

Procedure set-up reference packet





Smith+Nephew

SPIDER2[◇]
Limb Positioner





Shoulder: Lateral Decubitus

S+N Part #	Product Name	Quantity	Reference
All Procedures Require			
72203299	SPIDER2 Limb Positioner	1	N/A
7210570	Piggy Back (2/box)	1	C
72203300	Switch Drape (20/box)	1	N/A
72200918	SPIDER Cart	1	N/A
Shoulder: Lateral Decubitus			
7210576	Lateral Shoulder Connection Bar (2/box)	1	D
7210577	Traction Accessory	1	B
7210573	Shoulder Stabilization Kit (12/box)	1	A
Optional			
72201428	Lateral JACK	1	N/A
72202466	Lateral JACK Sterile Cover (case of 25)	1	N/A

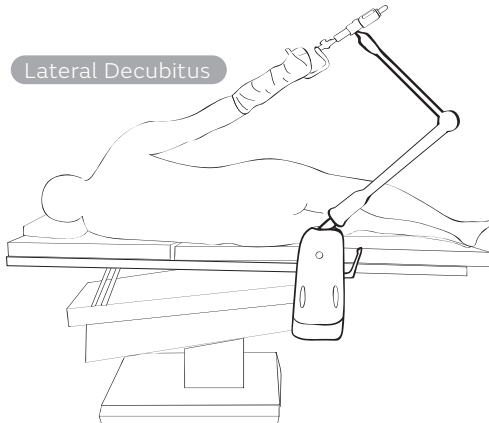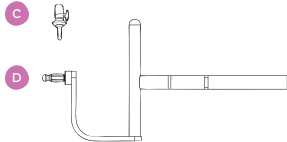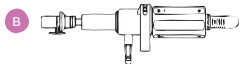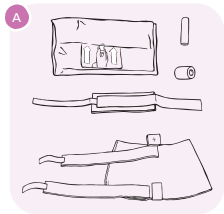


Shoulder: Beach Chair


S+N Part #	Product Name	Quantity	Reference
All Procedures Require			
72203299	SPIDER2 Limb Positioner	1	N/A
7210570	Piggy Back (2/box)	1	B
72203300	Switch Drape (20/box)	1	N/A
72200918	SPIDER Cart	1	N/A
Shoulder: Beach Chair			
7210571	Shoulder Connection Bar (2/box)	1	C
7210573	Shoulder Stabilization Kit (12/box)	1	A
Optional Accessories			
7210551	T-Max Beach Chair Positioner - N. America	1	N/A

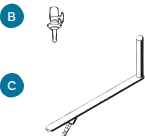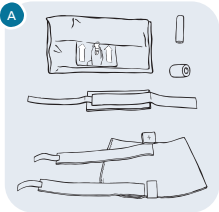

Beach Chair

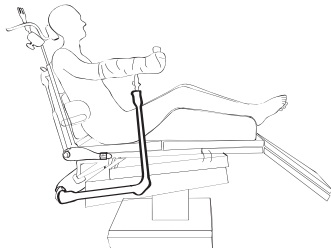

Spider2 with T-Max

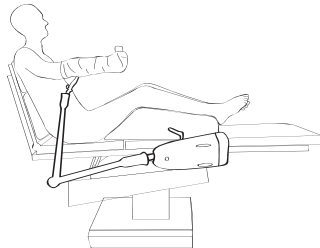




Spider2

SPIDER2 Limb Positioner

Elbow

Smith+Nephew



S+N Part #	Product Name	Quantity	Reference
All Procedures Require			
72203299	SPIDER2 Limb Positioner	1	N/A
7210570	Piggy Back (2/box)	1	B
72203300	Switch Drape (20/box)	1	N/A
72200918	SPIDER Cart	1	N/A
Elbow			
7210572	Elbow Connection Bar (2/box)	1	D
7210749	90 Degree Connector	1	C
7210574	Elbow Stabilization Kit (12/box)	1	A

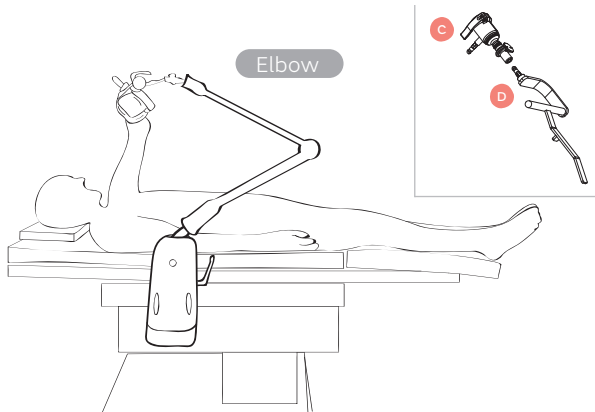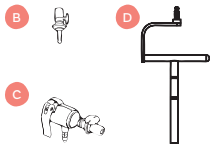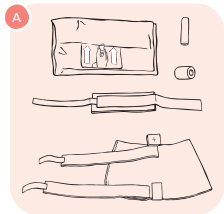

SPIDER2 Limb Positioner

Wrist

S+N Part #	Product Name	Quantity	Reference
All Procedures Require			
72203299	SPIDER2 Limb Positioner	1	N/A
7210570	Piggy Back (2/box)	1	E
72203300	Switch Drape (20/box)	1	N/A
72200918	SPIDER Cart	1	N/A
Wrist			
72200660	Wrist Arthroscopy Accessory	1	C
7210577	Traction Accessory	1	D
72200664	Wrist Counter Traction Post	1	A
72201550	Wrist Stabilization Kits (5/box) - small	1	B
72201551	Wrist Stabilization Kits (5/box) - medium	1	B
72201552	Wrist Stabilization Kits (5/box) - large	1	B

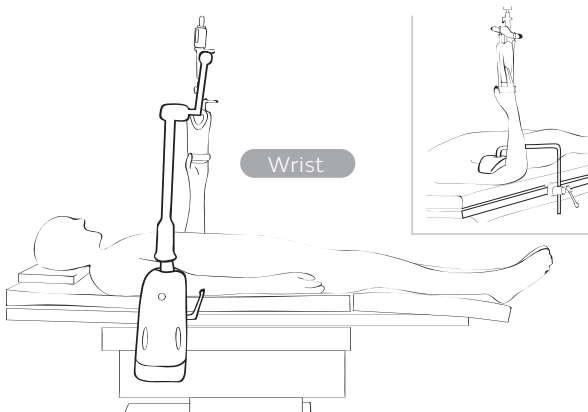
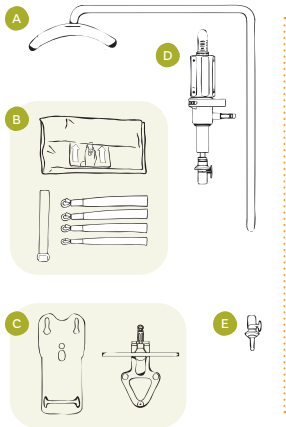


SPIDER2 Limb Positioner

Ankle




Smith+Nephew

S+N Part #	Product Name	Quantity	Reference
All Procedures Require			
72203299	SPIDER2 Limb Positioner	1	N/A
7210570	Piggy Back (2/box)	1	C
72203300	Switch Drape (20/box)	1	N/A
72200918	SPIDER Cart	1	N/A
Ankle			
72202932	Ankle Arthroscopy Accessory (2/box)	1	B
7210577	Traction Accessory	1	A
72202931	Ankle Stabilization Kit (6/box)	1	D

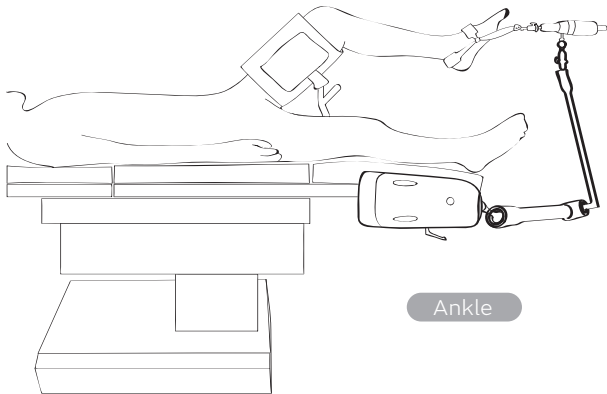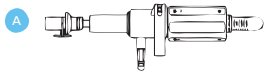

Knee: Leg Accessory

S+N Part #	Product Name	Quantity	Reference
All Procedures Require			
72203299	SPIDER2 Limb Positioner	1	N/A
7210570	Piggy Back (2/box)	1	C
72203300	Switch Drape (20/box)	1	N/A
72200918	SPIDER Cart	1	N/A
Knee (Leg Accessory)			
72203236	Leg Accessory (Right)	1	B
72203235	Leg Accessory (Left)	1	B
72203239	3D Connector	1	D
72203238	Leg Stabilization Kits (10/box)	1	A

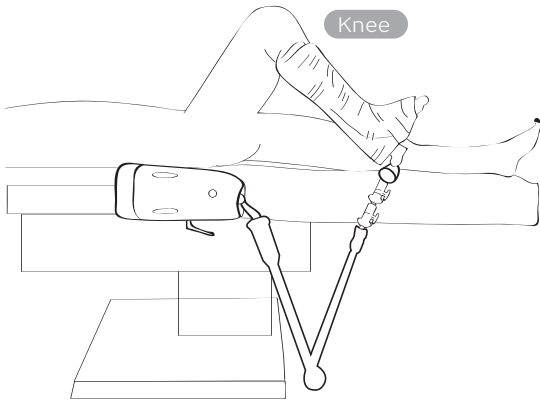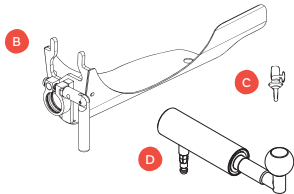
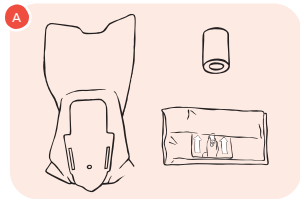


Knee: Distal Thigh Holder

S+N Part #	Product Name	Quantity	Reference
All Procedures Require			
72203299	SPIDER2 Limb Positioner	1	N/A
7210570	Piggy Back (2/box)	1	B
72203300	Switch Drape (20/box)	1	N/A
72200918	SPIDER Cart	1	N/A
Knee (Distal Thigh Holder)			
72204133	Distal Thigh Holder, 1 each, non-sterile	1	C
72204134	Thigh Stabilization Kit (10/box)	1	A

SPIDER2 Limb Positioner Knee: Distal Thigh Holder


A

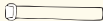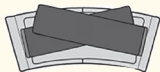


B



C

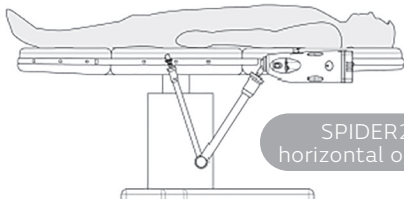

SPIDER2
horizontal on rail

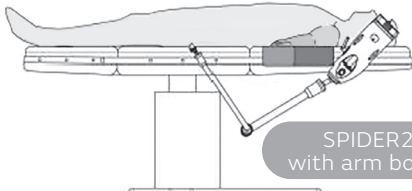

SPIDER2
with arm board

SPIDER2 Limb Positioner

Procedure set-up

Procedure	SPIDER Table Attachment Site	SPIDER Location to Patient	SPIDER Orientation	Required Accessories	Optional Accessories
All Procedures Require				• SPIDER2 • Piggy Back	• Switch Drape • SPIDER Cart
Ankle	Non-Operative Side	Calf level	Horizontal on Rail	• Ankle Arthroscopy Accessory • Traction Accessory • Ankle Stabilization Kit^D	
Knee: Leg Accessory	Operative Side	Hip level	Horizontal on Rail	• 3D Connector • Leg Accessory (L or R) • Leg Stabilization Kit^D	
Knee: Distal Thigh Holder	Operative Side	Shoulder level	Horizontal on Rail with the elbow joint of the SPIDER2 under the patient's hip When used with arm board: 45° Angle on Rail with the elbow joint of the SPIDER2 under the patient's hip	• Distal Thigh Holder Plate • Distal Thigh Holder Rotational Collar • Thigh Stabilization Kit^D	

D = Disposable

Procedure	SPIDER Table Attachment Site	SPIDER Location to Patient	SPIDER Orientation	Required Accessories	Optional Accessories
All Procedures Require				• SPIDER2 • Piggy Back	• Switch Drape • SPIDER Cart
Shoulder: Beach Chair	Operative Side or Attachment Rail/ T-MAX frame	If operative side at knee level	Horizontal on Rail	• Shoulder Connection Bar • Shoulder Stabilization Kit^D	T-MAX
Shoulder: Lateral Decubitus	Operative Side	Thigh level	Vertical on Rail	• Lateral Shoulder Connection Bar • Traction Accessory • Shoulder Stabilization Kit^D	• Lateral JACK (non - operative side, secure to table rail) • Lateral JACK sterile cover
Elbow	Non-operative Side	Hip level	Vertical on Rail	• Elbow Connection Bar • 90 Degree Connector • Elbow Stabilization Kit^D	Western Elbow Positioner (if facility does a lot of elbow procedures but does not want to purchase SPIDER2)
Wrist	Non-Operative Side (Counter Traction attached on Operative Side via Clark Socket)	Chest level	Vertical on Rail	• Wrist Arthroscopy Accessory • Wrist Counter Traction Post • Traction Accessory • Wrist Stabilization Kit (S, M, L)^D	Wrist Tower (if facility does a lot of wrist procedures but does not want to purchase SPIDER2)

For questions related to the SPIDER2 Limb Positioner please contact customer support.

Endo-DL_AND_Technical_Support@smith-nephew.com

+1 978 749 2412 or +1 800 343 5717

Sports Medicine
Smith & Nephew, Inc.
150 Minuteman Road
Andover, MA 01810

www.smith-nephew.com
T +978 749 1000
US Customer Service:
+1 800 343 5717

◊Trademark of Smith+Nephew.
All trademarks acknowledged.
©2020 Smith+Nephew. All rights
reserved. Printed in USA. 03325 V3 04/20