

+ A gentle silicone adhesive dressing, providing a moist wound environment for healing and available in a range of shapes and sizes for awkward areas of the body¹⁻⁵

Smith+Nephew

ALLEVYN[®]
GENTLE BORDER
Foam Dressing

Application



Cleanse the application site, ensuring the surrounding skin is clean, dry and free from excess hair.

Select appropriate dressing size. Ensure the absorbent pad covers the entire wound or area to be protected. The scale on the pouch is for guidance only




Remove first liner and anchor the adhesive side of the dressing to the skin. If applying the sacrum dressing, place the narrow end a minimum of 2cm (3/4in.) above the anal sphincter, then smooth the dressing over the sacrum.

Remove remaining liner/liners. Smooth over. If used on a wound, ensure absorbent pad covers the wound.




Smooth down the borders. Ensure there are no creases that could compromise the seal. The dressing can be repositioned as required.


The following pictures give you an indication of when **ALLEVYN GENTLE BORDER** needs to be changed

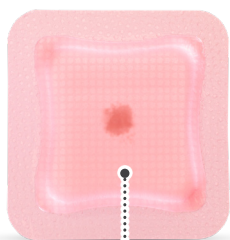

NO NEED TO CHANGE

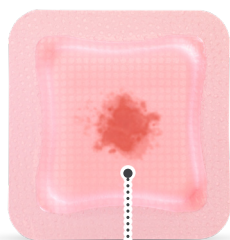

NO NEED TO CHANGE

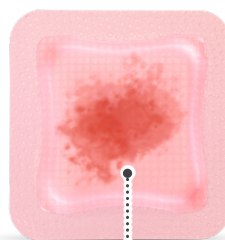

CHANGE DRESSING

Dressing can be left in place for up to 7 days (up to 5 days for sacral variant), unless exudate is within 0.5cm from the edge of the dressing pad - whichever is sooner.

Always adhere to local dressing change protocol.

Removal

To remove the dressing, lift one corner of the dressing and slowly peel back until completely removed from the wound



For detailed product information, including indications for use, contraindications, precautions and warnings, please consult the product's applicable Instructions for Use (IFU) prior to use.

ALLEVYN GENTLE BORDER ordering codes

Size (cm)	S+N Code	Carton
7.5cm x 7.5cm	66800269	10
10cm x 10cm	66800270	10
10cm x 20cm	66800900	10
12.5cm x 12.5cm	66800272	10
15cm x 15cm	66800975	10
17.5cm x 17.5cm	66800273	10
23cm x 23.2cm Heel	66800506	5
16.8cm x 17.1cm Sacrum	66800897	10
21.6cm x 23cm Sacrum	66801031	10
17.1cm x 17.9cm Multisite	66800959	10

References 1. Hurd T, Gregory L, Jones A, Brown S. A multi-centre in-market evaluation of ALLEVYN Gentle Border. Wounds UK. 2009;5(3):32-44. 2. Smith & Nephew 2016. ALLEVYN Gentle Border Heel Gen 2 - physical evaluation. Internal Report. DS/16/465/R. 3. Smith & Nephew 2017. ALLEVYN Gentle Border Gen 2 - Physical Evaluation. Internal Report. DS/16/424/R V3. 4. Smith & Nephew 2019. Use of Moisture Vapour Permeability (MVP) and Moisture Vapour Transmission Rate (MVTR) data to support product claims referring to moist wound healing. Internal report. EO.AWM.PCSgen.001.v2. 5. Rafter L, Reynolds T, Rafter M. An audit of patient outcomes in the management of skin tears using silicone dressings. Wounds UK. 2016;12(2):70-78.

Smith+Nephew
Kay Fiskers Plads 9
2300 København S
Denmark
T: +45 45 80 61 00

Smith+Nephew
Krokslätts Fabriker 39
431 37 Mölndal
Sweden
T: +46 31 74 65 800

Smith+Nephew
Snarøyveien 36
1364 Fornebu
Norway
T: +47 66 84 20 20

Smith+Nephew
Lentäjantie 1
01530 Vantaa
Finland
T: +358 (0)207 866 333

©Trademark of Smith+Nephew
All Trademarks acknowledged
© 2024 Smith+Nephew
41772-nord 03/24