

Procedural set-up guide





Smith+Nephew

SPIDER2[◇]
Limb Positioner



Shoulder: Lateral Decubitus

Reference	Description	Quantity	Figure
All procedures require			
72203299	SPIDER2 Limb Positioner	1	N/A
7210570	Piggy Back (2/box)	1	C
Shoulder: Lateral Decubitus			
7210576	Lateral Shoulder Connection Bar (2/box)	1	D
7210577	Traction Accessory	1	B
7210573	Shoulder Stabilization Kit (12/box)	1	A
Optional equipment			
72203300	Switch Drape (20/box)	1	N/A
72200918	SPIDER Cart	1	N/A
72201428	Lateral JACK	1	N/A
72202466	Lateral JACK Sterile Cover (case of 25)	1	N/A

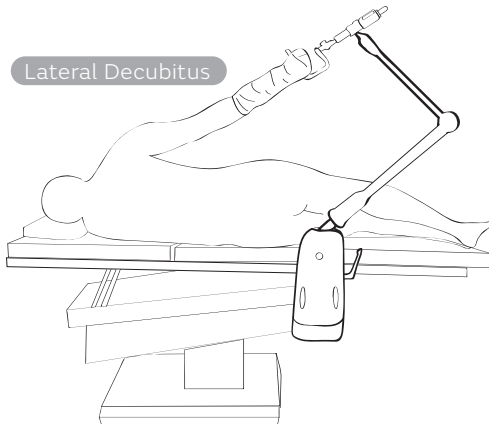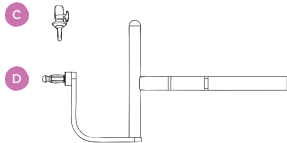
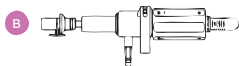
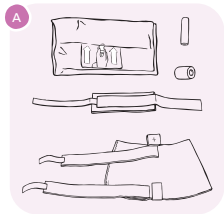


SPIDER2[®] Limb Positioner

Shoulder: Beach Chair


Smith+Nephew

Reference	Description	Quantity	Figure
All procedures require			
72203299	SPIDER2 Limb Positioner	1	N/A
7210570	Piggy Back (2/box)	1	B
Shoulder: Beach Chair			
7210571	Shoulder Connection Bar (2/box)	1	C
7210573	Shoulder Stabilization Kit (12/box)	1	A
Optional Accessories			
7210551	T-MAX Beach Chair Positioner - N. America	1	D
72203300	Switch Drape (20/box)	1	N/A
72200918	SPIDER Cart	1	N/A

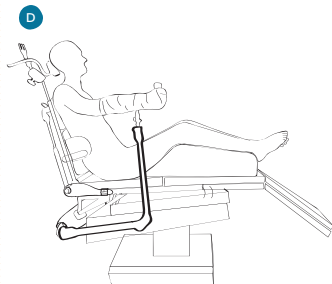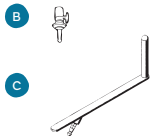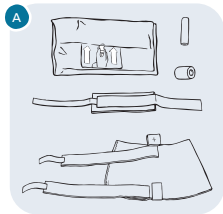

SPIDER2 with T-MAX

Beach Chair

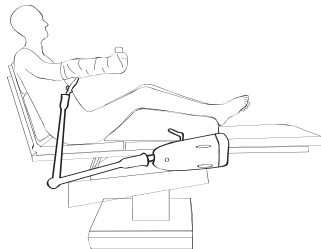




SPIDER2

SPIDER2[®] Limb Positioner

Elbow

Smith+Nephew

Reference	Description	Quantity	Figure
All procedures require			
72203299	SPIDER2 Limb Positioner	1	N/A
7210570	Piggy Back (2/box)	1	B
Elbow			
7210572	Elbow Connection Bar (2/box)	1	D
7210749	90 Degree Connector	1	C
7210574	Elbow Stabilization Kit (12/box)	1	A
72203300	Switch Drape (20/box)	1	N/A
72200918	SPIDER2 Cart	1	N/A

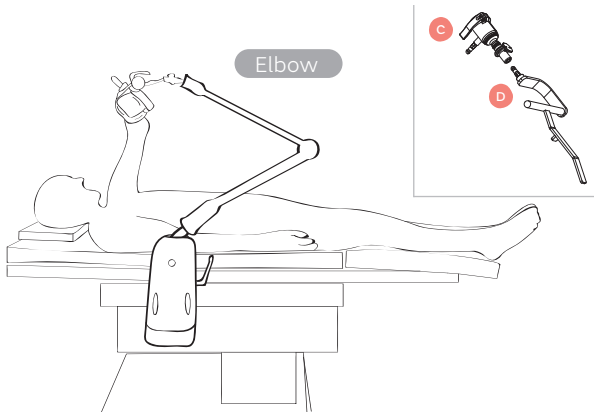
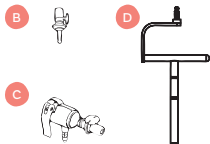
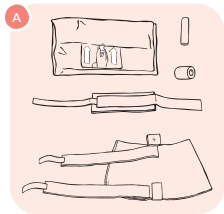


SPIDER2[®] Limb Positioner

Wrist

Smith+Nephew

Reference	Description	Quantity	Figure
All procedures require			
72203299	SPIDER2 Limb Positioner	1	N/A
7210570	Piggy Back (2/box)	1	E
Wrist			
72200660	Wrist Arthroscopy Accessory	1	C
7210577	Traction Accessory	1	D
72200664	Wrist Counter Traction Post	1	A
72201550	Wrist Stabilization Kits (5/box) - small	1	B
72201551	Wrist Stabilization Kits (5/box) - medium	1	B
72201552	Wrist Stabilization Kits (5/box) - large	1	B
72203300	Switch Drape (20/box)	1	N/A
72200918	SPIDER Cart	1	N/A

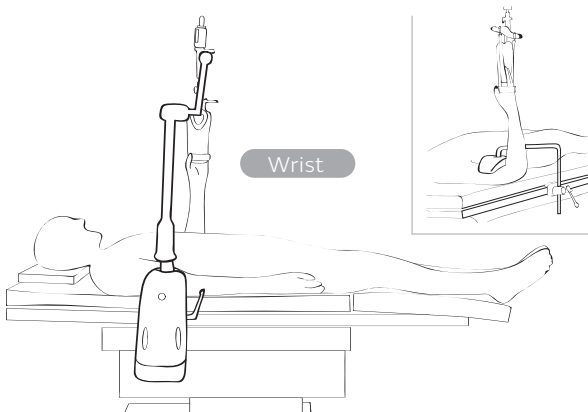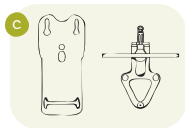
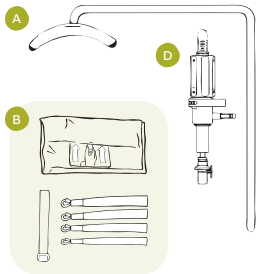


SPIDER2[®] Limb Positioner

Ankle

Smith+Nephew

Reference	Description	Quantity	Figure
All procedures require			
72203299	SPIDER2 Limb Positioner	1	N/A
7210570	Piggy Back (2/box)	1	C
Ankle			
72202932	Ankle Arthroscopy Accessory (2/box)	1	B
7210577	Traction Accessory	1	A
72202931	Ankle Stabilization Kit (6/box)	1	D
72201812	FERKLE™ Thigh Holder	1	E
72203300	Switch Drape (20/box)	1	N/A
72200918	SPIDER Cart	1	N/A


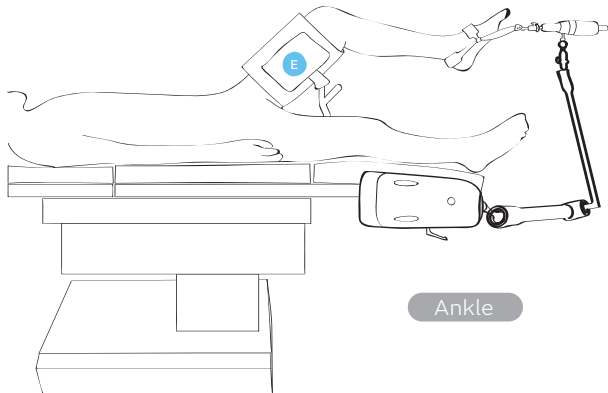
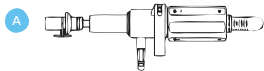


SPIDER2[®] Limb Positioner

Knee: Leg Accessory

Smith+Nephew

Reference	Description	Quantity	Figure
All procedures require			
72203299	SPIDER2 Limb Positioner	1	N/A
7210570	Piggy Back (2/box)	1	C
Knee (Leg Accessory)			
72203236	Leg Accessory (Right)	1	B
72203235	Leg Accessory (Left)	1	B
72203239	3D Connector	1	D
72203238	Leg Stabilization Kits (10/box)	1	A
72203300	Switch Drape (20/box)	1	N/A
72200918	SPIDER Cart	1	N/A

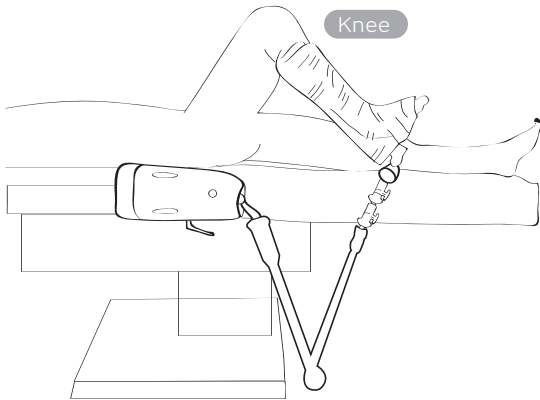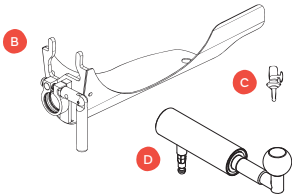
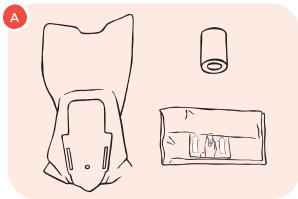


Knee: Distal Thigh Holder

Reference	Description	Quantity	Figure
All procedures require			
72203299	SPIDER2 Limb Positioner	1	N/A
7210570	Piggy Back (2/box)	1	B
Knee (Distal Thigh Holder)			
72204133	Distal Thigh Holder, 1 each, non-sterile	1	C
72204134	Thigh Stabilization Kit (10/box)	1	A
72203300	Switch Drape (20/box)	1	N/A
72200918	SPIDER Cart	1	N/A

SPIDER2[◇] Limb Positioner **Knee: Distal Thigh Holder**


A

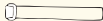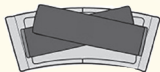


B



C

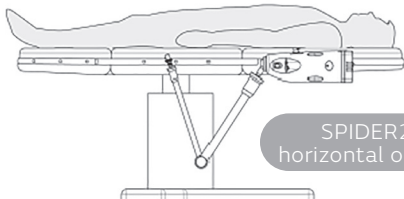

SPIDER2
horizontal on rail

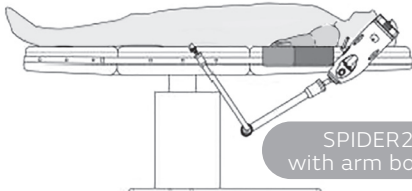

SPIDER2
with arm board

SPIDER2[®] Limb Positioner

Procedure set-up

Procedure	SPIDER table attachment site	SPIDER location to patient	SPIDER orientation	Required accessories	Optional accessories
All Procedures Require				• SPIDER2 • Piggy Back	• Switch Drape • SPIDER Cart
Shoulder: Lateral Decubitus	Operative side	Thigh level	Vertical on rail	• Lateral Shoulder Connection Bar • Traction Accessory • Shoulder Stabilization Kit^D	• Lateral JACK (non - operative side, secure to table rail) • Lateral JACK sterile cover
Shoulder: Beach Chair	Operative side or attachment rail/ T-MAX frame	If operative side at knee level	Horizontal on rail	• Shoulder Connection Bar • Shoulder Stabilization Kit^D	• T-MAX
Elbow	Non-operative side	Hip level	Vertical on rail	• Elbow Connection Bar • 90 Degree Connector • Elbow Stabilization Kit^D	
Wrist	Non-operative side (counter traction attached on operative side via clark socket)	Chest level	Vertical on rail	• Wrist Arthroscopy Accessory • Wrist Counter Traction Post • Traction Accessory • Wrist Stabilization Kit (S, M, L)^D	

SPIDER2[◇] Limb Positioner

Procedure set-up

Procedure	SPIDER table attachment site	SPIDER location to patient	SPIDER orientation	Required accessories	Optional accessories
All procedures require				• SPIDER2 • Piggy Back	• Switch Drape • SPIDER Cart
Ankle	Non-operative side	Calf level	Horizontal on rail	• Ankle Arthroscopy Accessory • Traction Accessory • Ankle Stabilization Kit^D	
Knee: Leg Accessory	Operative side	Hip level	Horizontal on rail	• 3D Connector • Leg Accessory (L or R) • Leg Stabilization Kit^D	
Knee: Distal Thigh Holder	Operative side	Shoulder level	Horizontal on rail with the elbow joint of the SPIDER2 under the patient's hip When used with arm board: 45° angle on rail with the elbow joint of the SPIDER2 under the patient's hip	• Distal Thigh Holder Plate • Distal Thigh Holder Rotational Collar • Thigh Stabilization Kit^D	

For questions related to the SPIDER2[°] Limb Positioner please contact customer support.

endocustomerservice.global@smith-nephew.com

+1 978 749 2412 or +1 800 343 5717

Products may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Please contact your Smith+Nephew representative or distributor if you have questions about the availability of Smith+Nephew products in your area. For detailed product information, including indications for use, contraindications, precautions and warnings, please consult the product's applicable Instructions for Use (IFU) prior to use.

Sports Medicine
Smith & Nephew, Inc.
150 Minuteman Road
Andover, MA 01810

www.smith-nephew.com
T +978 749 1000
US Customer Service:
+1 800 343 5717

[°]Trademark of Smith+Nephew.
All trademarks acknowledged.
©2025 Smith+Nephew. All rights
reserved. Printed in USA. 03325 V4 07/25