

Smith+Nephew

ACUFEX[®] ACCESS
Positioning System

Quick setup guide

Step-by-step guide



A step-by-step guide to help you set up the ACUFEX® Access Positioning System in your OR.

If the lateral traction boom will NOT be used, skip steps 4, 5 & 10.*



1. Attach clamp onto end of table rail.



2. Slide drape onto primary boom, align and insert orange t-fitting.



3. Slide blue ring down to lock in place. Continue pulling drape back and secure with tape.



4. Insert lateral traction boom onto desired bracket, lower in place and telescope to desired length.*



5. Drape lateral traction boom, align and insert orange t-fitting. Slide blue ring down to lock in place. Secure with tape.*



6. Place hand inside mitt and secure foam wrist strap.



7. Secure outer foam tabs and straps. Wrap using Smith & Nephew self-adhesive bandage.



8. Attach operative arm to system by aligning t-fitting and lower into place.



9. Attached desired weight to system.



10. Place sling under upper arm and secure buckle. Attach sling by aligning t-fitting and lower into place.*



ACUFEX Access Positioning System shown in the advanced setup configuration with patient attached.



ACUFEX Access Positioning System shown on cart in storage configuration.

*Consult Instructions For Use document: 10601379.

For detailed information, including indications, contraindications, warnings, precautions, cleaning instructions please consult the Instructions for Use (IFU) 10601379 prior to use.

Learn more at smith-nephew.com

Sports Medicine
Smith & Nephew, Inc.
150 Minuteman Road
Andover, MA 01810

www.smith-nephew.com
T +978 749 1000
US Customer Service:
+1 800 343 5717

®Trademark of Smith+Nephew.
All trademarks acknowledged.
©2020 Smith+Nephew. All rights reserved.
Printed in USA. 16256 V2 04/20

