

Physical appearance check

Precautions

Prior to the start of this process confirm that the unit has been cleaned and disinfected correctly. A formal record of cleaning and disinfection must be present and available prior to the start of the following activities. Use gloves until cleaning/disinfecting is completed.

Cleaning and disinfecting

- 1 Open the shipping carton and place the pump in its plastic bag under the hood.
- 2 Verify that the customer included his Declaration of cleaning and disinfection as required. If cleaning cannot be verified do not continue, notify customer/supervisor.
- 3 Remove pump from plastic and discard bag.
- 4 Carefully clean all surfaces of the pump, power cord, canister holders and universal holder (if returned) with germicidal/antibacterial treated disposable wipe. Pay close attention to the handle and areas closest to the location of the canister. For stubborn or larger debris use a soft bristle brush.
- 5 Use a second wipe to thoroughly clean the surface.
- 6 Allow to remain wet for 5 minutes.
- 7 Dry surfaces with a disposable towel or cloth.

Do not immerse any part of the pump in fluid or use unnecessarily wet cloth.

Wound Management

Smith & Nephew, Inc. Customer Care Center www.smith-nephew.com
970 Lake Carillon Drive 1-800-876-1261 www.npwt.com
Suite 110 T 727-392-1261
St. Petersburg, FL 33716 F 727-392-6914
USA