



+ PICO[◇] dressing removal

Smith+Nephew

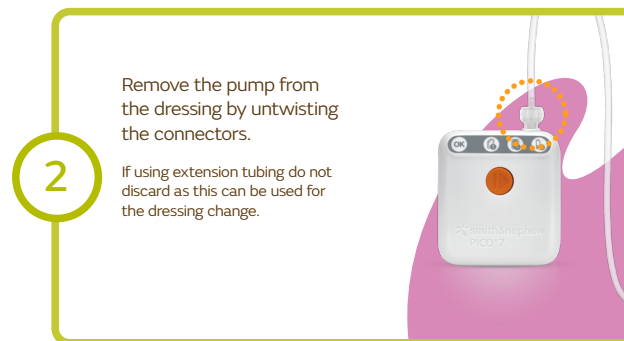

Remove the PICO dressing by stretching the fixation strips away from the skin. Lift the dressing at one corner and peel back until it has been fully removed.

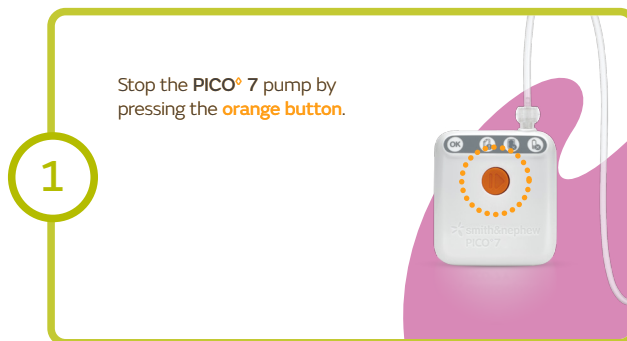

The PICO dressing and fixation strips should be disposed of as clinical waste.



At the end of therapy the batteries should be removed from the PICO 7 pump; both batteries and the pump should be disposed of according to local regulations.

+ PICO[◇] sNPWT


inspection and monitoring

Smith+Nephew




PICO DRESSING


- Inspect the **PICO** dressing regularly.
- This can give you an indication if the **PICO** dressing appears ready for changing.

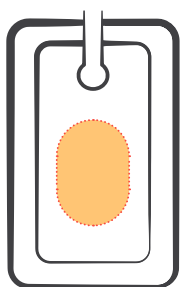


Dressing properly positioned and this is acceptable to be left in place.

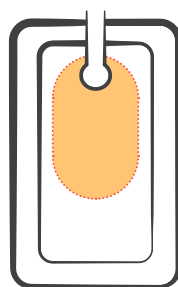

Dressing requires change.

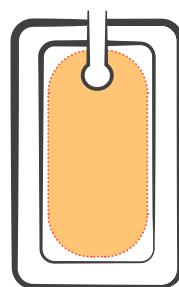

Dressing requires change.

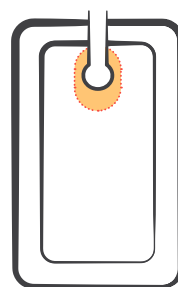


Dressing requires change – port may block with fluid.



PICO PUMP

- The **PICO** pump should be accessible and checked routinely.
- It does not contain audible alerts.

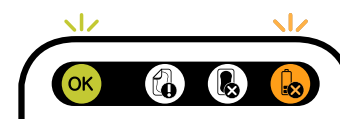

Flashing **green** OK indicator and **orange** battery low indicator together = **battery needs changing**.



Flashing **orange** leak indicator will flash = **air leak detected**. You will hear the pump make a buzzing sound as it tries to get to the right vacuum.




Flashing **orange** 'dressing full' indicator = **dressing saturated** or the **filter blocked**.

+ PICO[◇] sNPWT

At-risk incision management

PATIENT DISCHARGE CHECKLIST

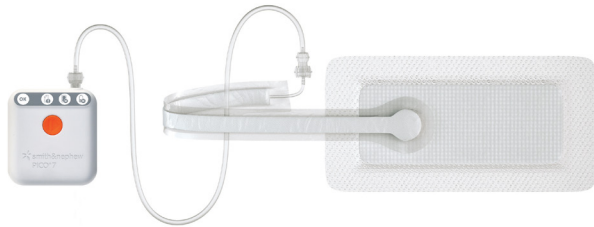

If PICO therapy is continuing following discharge ONLY.

PICO pump and dressing inspection and troubleshooting:

- ☐ Demonstration of PICO dressing inspection and PICO pump status checks performed.

PICO therapy patient guide and user manual:

- ☐ Patient provided with copies of Patient Guide and User Manual for reference.

Useful contacts:

- ☐ Patient provided with HCP contact details.