


+ ALLEVYN[◇]

COMPLETE CARE

Foam Dressing

Helps to minimise leakage^{*1-3}
and defends against factors that contribute
to pressure injuries.^{†4-7}

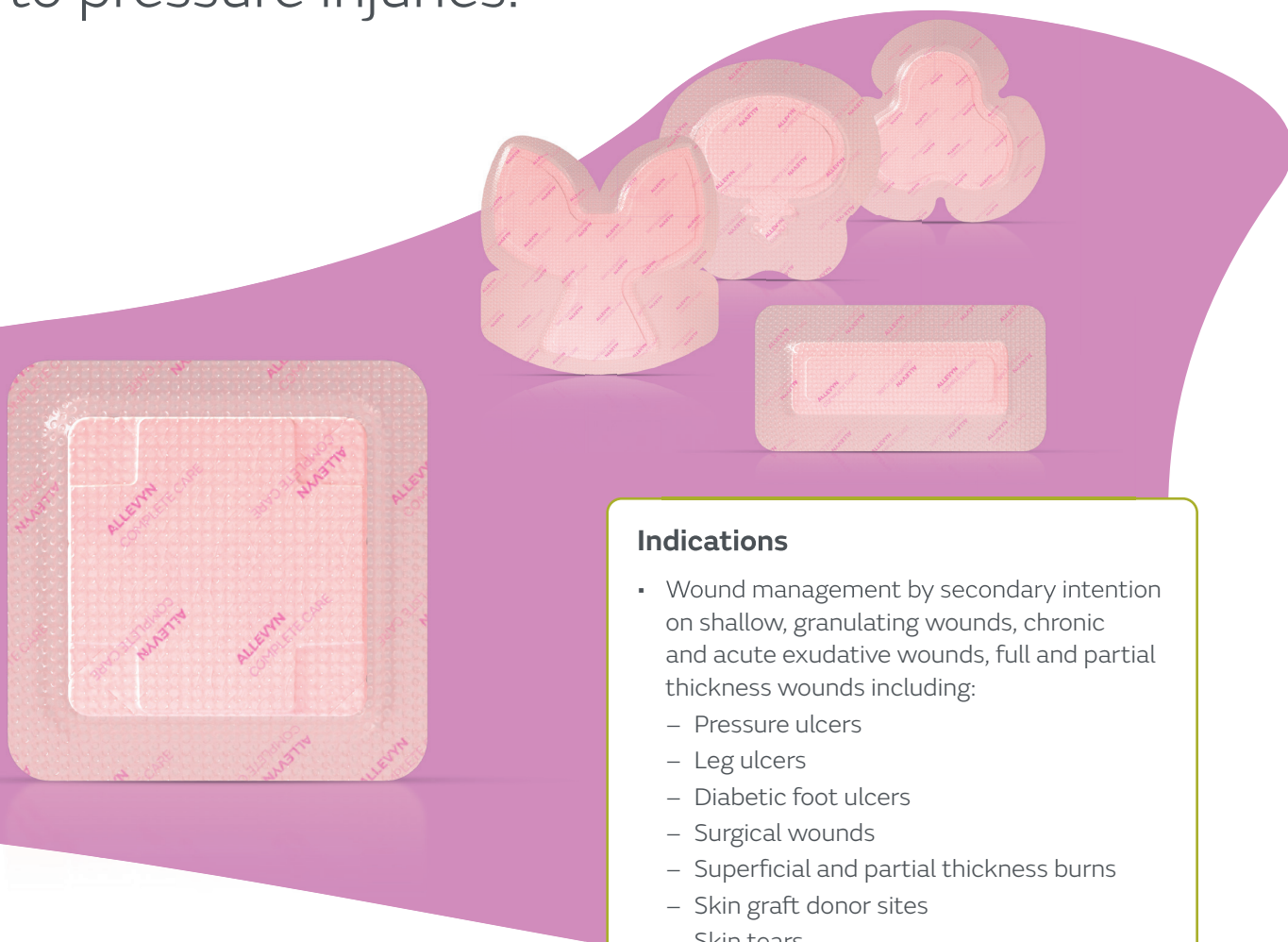

Smith+Nephew

ALLEVYN[◇]
COMPLETE CARE
Foam Dressing

www.smith-nephew.com

Indications

- Wound management by secondary intention on shallow, granulating wounds, chronic and acute exudative wounds, full and partial thickness wounds including:
 - Pressure ulcers
 - Leg ulcers
 - Diabetic foot ulcers
 - Surgical wounds
 - Superficial and partial thickness burns
 - Skin graft donor sites
 - Skin tears
 - Fungating ulcers
- Pressure injury prevention as part of a pressure injury prevention protocol

Combination use

- Can be used in conjunction with INTRASITE[◇] GEL for necrotic or sloughy wounds
- Can be used in conjunction with PROFORE[◇] Multi-layer Compression Bandage System



LOW – HIGH
ABSORBENCY

ALLEVYN[®] COMPLETE CARE Foam Dressings with their unique construction help to minimise leakage^{*1-3} and defend against factors that contribute to pressure injuries^{†4-7}

Features and benefits

Exudate management




ExuLOCK[®] advanced lock-in technology

- Lock-in of exudate and bacteria even under compression^{1,8}
- Uses lateral absorption across the entire dressing area^{*9} to help prevent leakage^{2,3}
- Provides a moist wound healing environment[‡] which is optimal for healing and managing the volume of fluid in the dressing^{3,5,9,10}


ExuMASK[®] Change indicator technology

- Visually masks absorbed exudate, keeping it looking fresher for longer^{2,3,9,11-14}
- Visually indicates when to change the dressing¹¹


ExuMASK change indicator visual guide

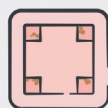

No need to change

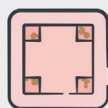

Consider changing

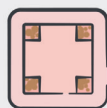

Change




ComfortSTAY[®] soft silicone adhesive technology

- Balances adhesion with gentleness¹⁵⁻¹⁷
- Is atraumatic to the wound and surrounding skin, minimising pain on removal^{3,16-18}



ALLEVYN highly absorbent foam

- Sustained vertical absorption helps minimise pooling and maceration^{1,3,9}
- Foam pore size manages low to high viscosity⁹

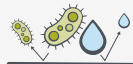

Highly breathable top film

- Sustained bacterial barrier performance^{‡19}
- Showerproof²⁰

HighFLEX multi-way stretch technology



- Highly conformable and flexible^{7,21}
- Securely conforms to the contours of the body^{§3,22}
- Helps with application on awkward areas^{§3,12,22}

Pressure injury prevention



ShearDEFENSE[®] Unbonded Layer Technology

- Proprietary unbonded design enables layers to frictionally slide within the dressing^{*4}
- Absorbs shear forces and reduces soft tissue strain^{*4,23} to minimise the risk of pressure injuries²⁴
- Highly effective shear and friction absorption^{11,4}



ComfortSTAY soft silicone adhesive technology

- Can be repositioned whilst maintaining adhesion to facilitate skin inspections over dressing wear time^{*15}



Highly breathable top film

- Helps to create an optimal microclimate^{5,10}



Pressure redistribution

- Spreads the pressure from external impact^{*6}

ALLEVYN COMPLETE CARE Dressing Range

S+N Code	Size	Carton
66807695	7.5cm x 7.5cm (3in x 3in)	10
66807696	10cm x 10cm (4in x 4in)	10
66807697	12.5cm x 12.5cm (5in x 5in)	10
66807698	15cm x 15cm (6in x 6in)	10
66807699	17.5cm x 17.5cm (7in x 7in)	10
66807700	10cm x 20cm (3.9in x 7.9in)	10
66807701	10cm x 30cm (3.9in x 11.8in)	10
66807702	Multisite 17.1cm x 17.9cm (6.7in x 7in)	10
66807703	Small Sacrum 17.2cm x 17.5cm (6.8in x 6.9in)	10
66807704	Large Sacrum 21.6cm x 23cm (8.5in x 9.1in)	10
66807705	Heel 25cm x 25cm (9.8in x 9.9in)	10



^{*}As demonstrated in vitro. [†]As demonstrated in silico and in vitro. [‡]Over the wear time of the dressing. [§]ALLEVYN 5-layer dressings. [¶]As demonstrated in silico.

Advanced Wound Management
Smith & Nephew Medical Ltd
Croxley Park Building 5
Lakeside, Hatters Lane, Watford
Hertfordshire WD18 8YE - UK

www.smith-nephew.com

T +44 (0) 1482 225181
F +44 (0) 1482 328326

◇ Trademark of Smith+Nephew
All Trademarks acknowledged
©August 2025 Smith+Nephew
AWM-AWC-47986. GMC1944

Products may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Please contact your Smith+Nephew representative or distributor if you have questions about the availability of Smith+Nephew products in your area. For detailed product information, including indications for use, contraindications, warnings and precautions, please consult the product's Instructions for Use (IFU) prior to use.

References: 1. Smith+Nephew 2025. Internal report CSD.AWM.25.008. 2. Tiscar-González V, et al. *Adv Skin Wound Care*. 2021;34(1):23–30. 3. Rossington A, et al. *Wounds UK*. 2013;9(4):91–95. 4. Smith+Nephew 2024. Internal report CSD.AWM.24.057 V2. 5. Smith+Nephew 2025. Internal report CSD.AWM.25.011. 6. Smith+Nephew 2025. Internal report CSD.AWM.25.016. 7. Smith+Nephew 2025. Internal report CSD.AWM.25.012. 8. Smith+Nephew 2025. Internal report CSD.AWM.25.019 V2. 9. Smith+Nephew 2025. Internal report CSD.AWM.25.009 V2. 10. Smith+Nephew 2025. Internal report CSD.AWM.25.010. 11. Smith+Nephew 2025. Internal report CSD.AWM.25.023 V2. 12. Simon D, Bielby A. *Wounds UK*. 2014;10(3):80–87. 13. Joy H, et al. *J Wound Care*. 2015;24(7):312, 314–317. 14. Costa B, Allen D. Poster presented at: EWMA; 2023. 15. Smith+Nephew 2025. Internal report CSD.AWM.25.015 V2. 16. Vowden K, et al. Paper presented at: EWMA; 2011. 17. Smith+Nephew. Internal report EA/AWM/GENERAL/001/v2. 18. Hurd T, Gregory L, Jones A, Brown S. A multi-centre in-market evaluation of ALLEVYN[®] Gentle Border. *Wounds UK*. 2009;5(3):32–44. 19. Smith+Nephew 2025. Internal report: CSD.AWM.25.014. 20. Smith+Nephew 2025. Internal report CSD.AWM.25.017. 21. Smith+Nephew 2025. Internal report: CSD.AWM.25.013. 22. Stephen-Haynes J, et al. *Journal of Community Nursing*. 2013;27(5):50–59. 23. Smith+Nephew 2024. Internal report CSD.AWM.24.058 V2. 24. Atkinson L, Costa B. Poster presented at: EWMA; 2024.