

RR900
 inflatable posterior
 epistaxis device

Inflatable unilateral models

Without airways

Size	Model
4.5cm	RR450
5.5cm	RR550
7.5cm	RR750
9.0cm	RR900

With airways


Size	Model
5.5cm	RR551
7.5cm	RR751

Non-inflatable tamponade


Size	Model
5.5cm	RR530




Product usage directions

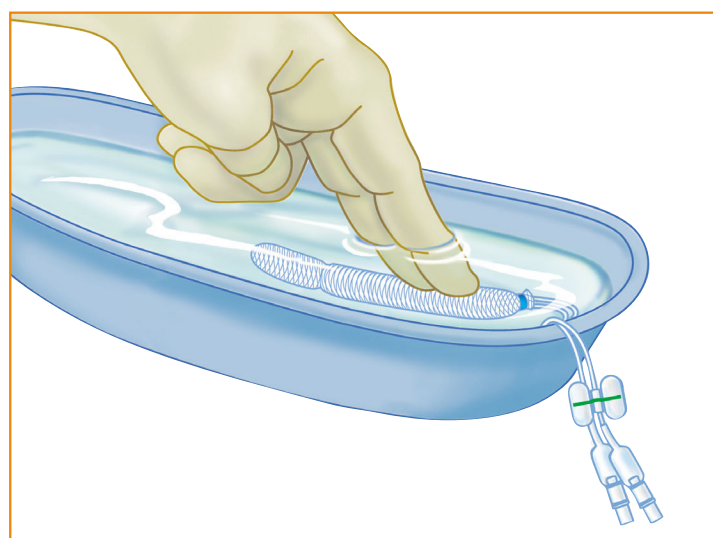


1 Soak in sterile water for 30 seconds.

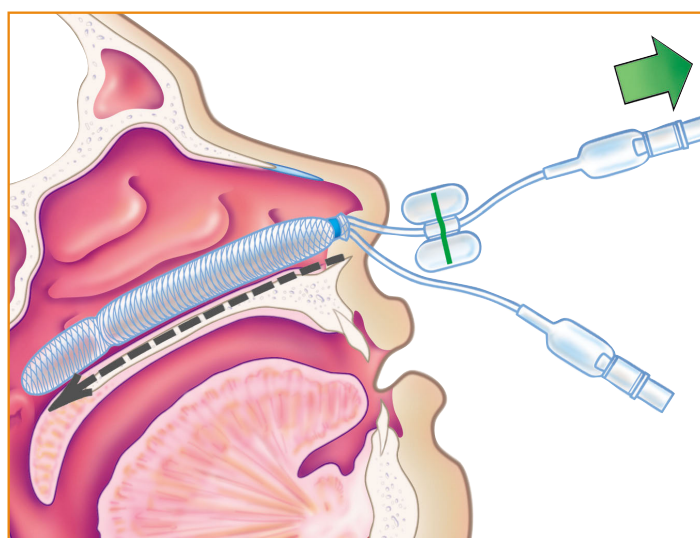


2 Insert along superior aspect of the hard palate until the blue indicator is past the nares.

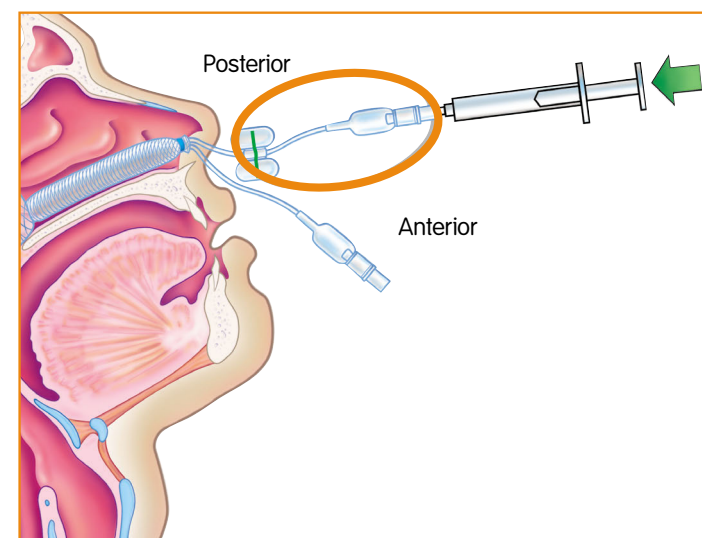


3 Inflate posterior balloon 1st (green line indicator) with air.

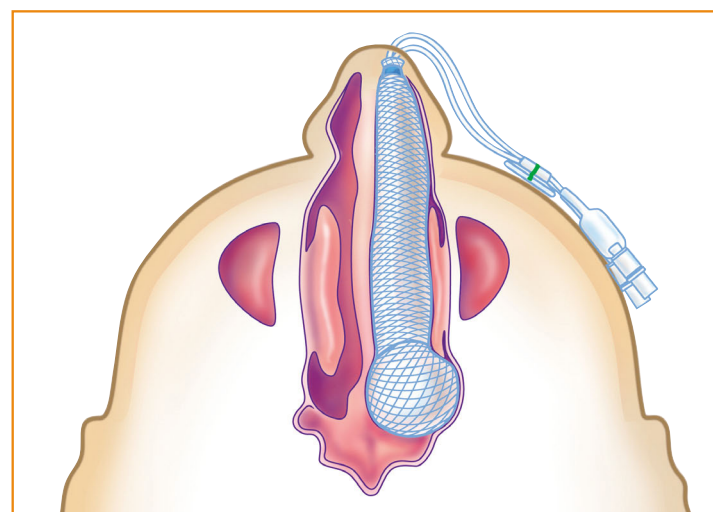

4 Inflate anterior balloon and ensure both pilot cuffs are firm.

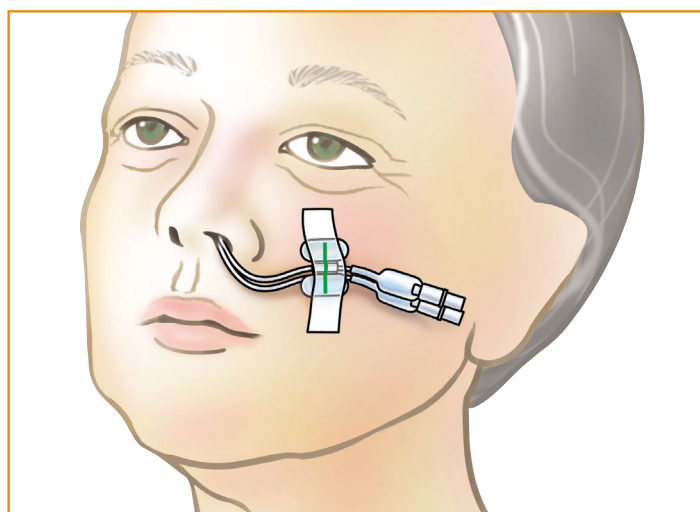

5 Discharge patient after a minimum of 15-20 minutes. Re-inflate to insure proper pressure and tape to patient's cheek.

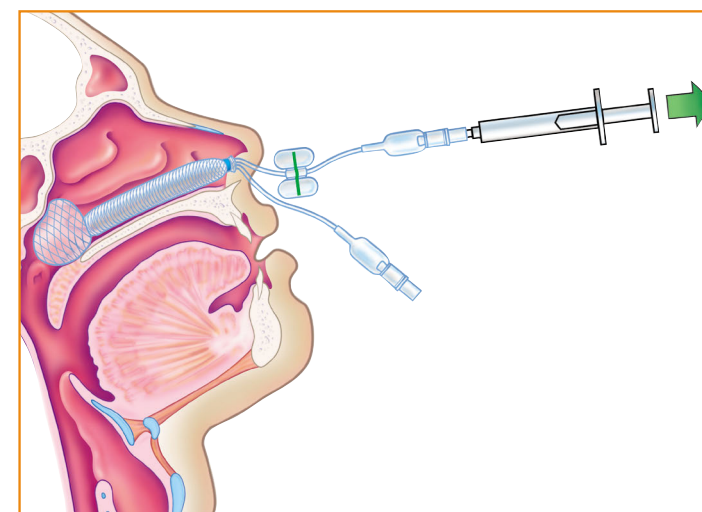

6 Removal should occur 24-72 hours after treatment.

NOTE: When there is sufficient pressure in the posterior balloon, remove the syringe from the inflation valve and gently pull the device anteriorly (as necessary) to ensure there is sufficient pressure on the posterior bleed site.

Please refer to the Instructions for Use (IFU) packaged with the product for a complete list of warnings, precautions and contraindications.

ArthroCare Corporation
 7000 West William Cannon Drive
 Austin, TX 78735
 USA

www.smith-nephew.com

Order Entry: 1-800-343-5717
 Order Entry Fax: 1-888-994-2782

© 2015 Smith & Nephew, Inc.
 *Trademark of Smith & Nephew. Reg. US Pat. & TM Office.