

+ ALLEVYN[◇] LIFE for one more day

A patient identifier for healthcare professionals

Smith+Nephew



ALLEVYN[◇] LIFE
Foam Dressing

Helping you get **CLOSER TO ZERO[◇]** human and economic consequence of wounds

smith-nephew.com/allevyn

Is ALLEVYN[®] LIFE right for your patient?

ALLEVYN LIFE can help a variety of your more independent patients. This could include people who struggle with finding the time for frequent dressing changes, as well as those who lead an active lifestyle or care for dependents.

After assessing the wound following agreed protocols, use this handy checklist to identify which of your patients could benefit from the ALLEVYN LIFE for one more day approach:



1. Consider the following in relation to your patient:	
a. Do they live an active lifestyle?	<input type="checkbox"/>
b. Do they have low to moderate exudate?	<input type="checkbox"/>
c. Are they engaged in their own care?	<input type="checkbox"/>
d. Are they keen to get back to their usual daily routine?	<input type="checkbox"/>
2. Explain what the dressing is and how the change indicator works	<input type="checkbox"/>
If the patient is suitable:	
3. Give your patient their ALLEVYN LIFE dressing reminder	<input type="checkbox"/>
4. Write your details, or your practice's, on the reminder so they can contact you	<input type="checkbox"/>



ALLEVYN[®] LIFE Dressings can help reduce the number of unnecessary changes^{1*} whilst maintaining **excellent exudate management**^{2**} and patient comfort. This could give you and your patients at least **one more day** to do more of the things that matter to you.

Why ALLEVYN LIFE?

We know that **your time is precious**, and that you want to spend as much of it as you can with the patients **who need it most**.

What you might not know is that:



Almost

70%

of dressing changes are driven by schedules³



50%


of dressing changes in the community are clinically unnecessary¹

With ALLEVYN LIFE, you could avoid these unnecessary dressing changes and get more from your time. You might be surprised by the difference that **one more day** can make.



How does it work?

With its unique change indicator and advanced lock-in technology, **ALLEVYN[®] LIFE** ensures **optimal patient-centred wound care**. The quadrilobe shape, wide borders, and flexible five layers are designed to help patients **get back to their real lives**.

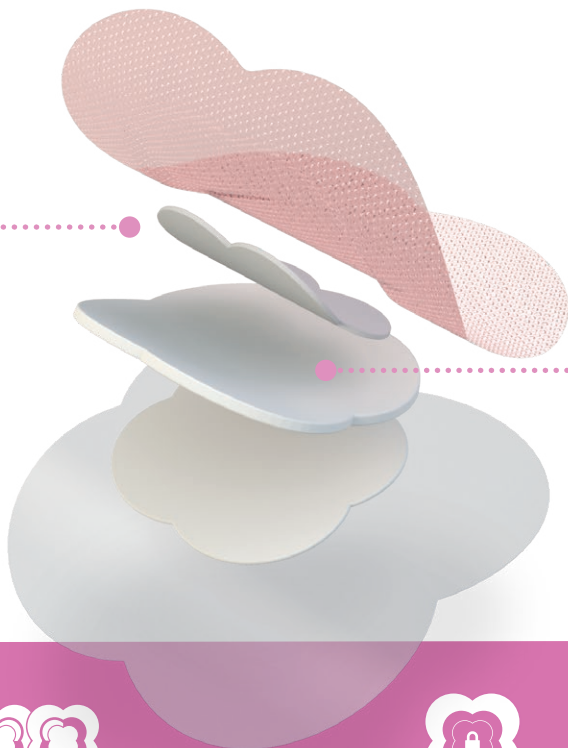

EXUMASK[◇] Change Indicator

Our unique change indicator lets you and your patients know the best time to change the dressing⁴



EXULOCK[◇] Advanced Lock-in

Our advanced lock-in technology minimises leakage⁵⁻⁸, helping it last longer^{1*} than other foam dressings

The difference a day makes

Free yourself and your patients from dressing changes for at least **one more day**. Here are some thoughts on ALLEVYN[®] LIFE from the nurses already using it⁹:

“Reducing the number of visits we do [and the number of dressing changes we conduct] gives us more time to spend with our patients.”

District nurse, UK

“If you’re looking at five people that are on twice weekly [dressing changes] instead of three times weekly, that’s a huge impact on our team, that’s why we use ALLEVYN LIFE.”

District nurse, UK

“We have such a huge case-load and not having to go in to see patients to change a dressing on a regular basis, because the dressing’s so effective, is fabulous to us.”

District nurse, UK

Your patients could benefit from **ALLEVYN LIFE**. Give them the treatment that best suits their lifestyle.

Find out what **one more day** can bring.





Advanced Wound Management
Smith & Nephew Inc.
2280 Argentia Road
Mississauga, ON
L5N 6H8, Canada
T 1-800-463-7439
F 1-800-671-9140

www.smith-nephew.com

◇ Trademark of Smith+Nephew
All Trademarks acknowledged
© March 2020 Smith+Nephew
23951 | CA59703 04/20

For detailed product information, including indications for use, contraindications, precautions and warnings, please consult the product's applicable Instructions for Use (IFU) prior to use.

* ALLEVYN[®] LIFE can be worn almost twice as long as standard dressings. n=37; dressing retention is 1.92x longer

** Based on a product evaluation

References: **1.** Joy H, Bielby A, Searle R. A collaborative project to enhance efficiency through dressing change practice. *J Wound Care*. 2015 Jul;24(7):312, 314-7. **2.** Stephen-Haynes J, Bielby A, Searle R. The clinical performance of a silicone foam in an NHS community trust. *Journal of Community Nursing*. 2013;27(5):50-59. **3.** Smith & Nephew Wound Management Clinical Report UKI-CP-PDTK-2013-DCF/RFC-1-5-14 May 2014. In-practice dressing change frequencies and reason for dressing change 2013 Community Practice Development Tool Kit (PDTK) survey data analysis. **4.** Simon D, Biel A. A structured collaborative approach to appraise the clinical performance of a new product. *Wounds UK*. 2014;10(3):80-87. **5.** Smith & Nephew May 2016 2016. New ALLEVYN Life Gen2 wcl - Physical Testing. Internal Report. DS/15/025/R. **6.** Rossington A, Drysdale K, Winter R. Clinical performance and positive impact on patient wellbeing of ALLEVYN Life. *Wounds UK*. 2013;9(4):91-95. **7.** Smith & Nephew, April 2016. Wound Model Testing of New ALLEVYN Life Gen2 wcl Dressing using Horse Serum at a Flow Rate Modelling that of a Moderately Exuding Wound. **8.** Smith & Nephew 20 June 2016. A Randomised Cross-Over Clinical Evaluation to Compare Performance of ALLEVYN[™] Life and Mepilex[®] Border Dressings on Patient Wellbeing-Related Endpoints. Internal Report. CE/047/ALF. **9.** Smith & Nephew 2016, ALLEVYN LIFE nurse testimonials. GMC1053