







Multiple fixation options, a streamlined technique ideal for multi-suture constructs

 **smith&nephew**
MULTIFIX[®] S
ULTRA
Knotless Suture Anchor

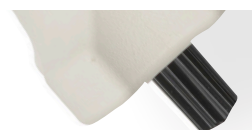


MULTIFIX[◇] S ULTRA Knotless Suture Anchor

The MULTIFIX S ULTRA Anchor provides multiple fixation options as well as a streamlined technique ideal in multi-suture constructs allowing surgeons to meet the needs of a diverse patient population.

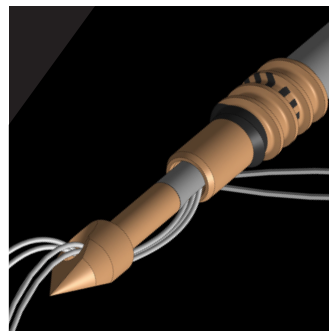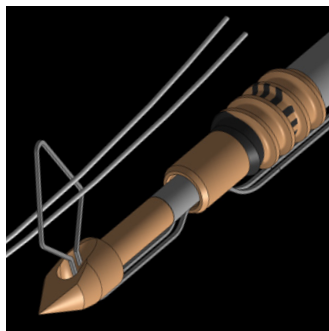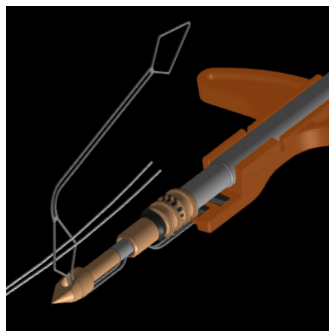


MULTIFIX S ULTRA Anchor specifications


Pre-deployed

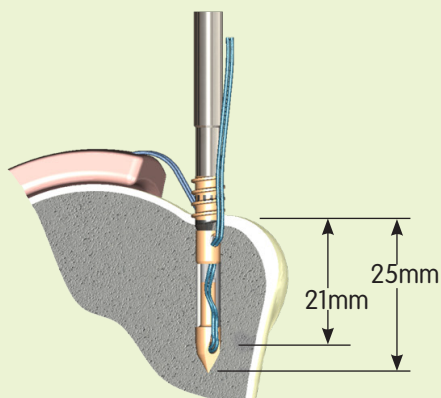

Deployed

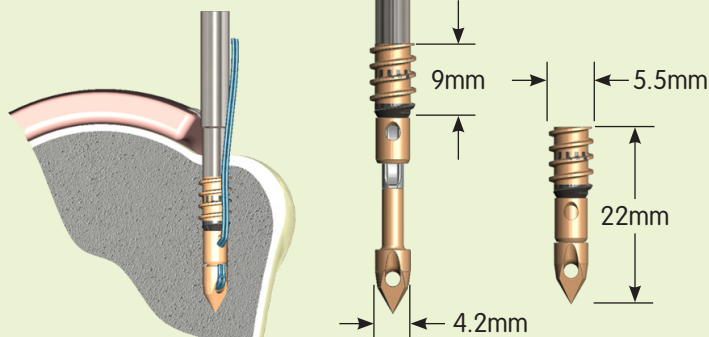

Accessories

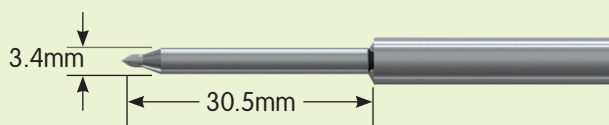

3.4mm bone punch (OM-9420)




Tapered punch (OM-9220)

Multiple fixation options

The MULTIFIX[®] S ULTRA Anchor allows the surgeon to create various biomechanically stable constructs with two bone lock options.

Functional testing 20/8 lb. foam*

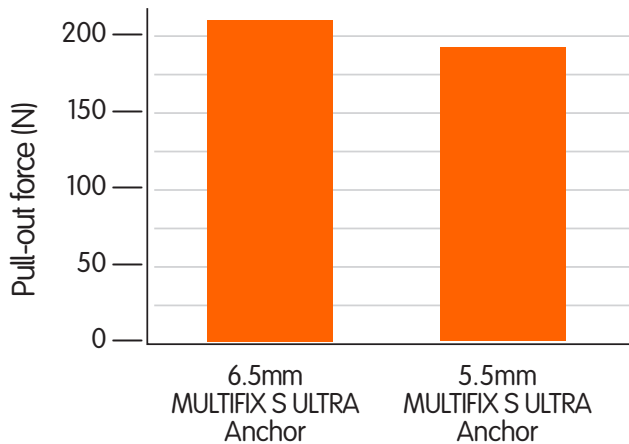

Streamlined technique

- Pound-in technique (bone hole may be required)
- Suture lock with the turn of a dial
- Hand tension sutures for tactile feedback



1. Insert PEEK tip into bone






2. Tension sutures



3. Turn dial to lock sutures

Ideal in multi-suture constructs

- Accommodates 2-4 suture strands to allow multiple construct configurations

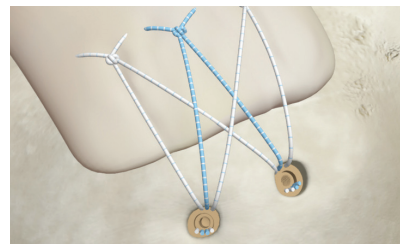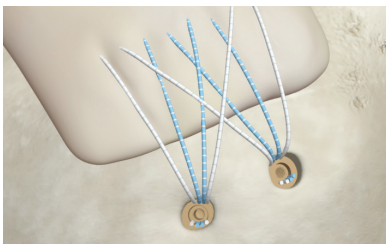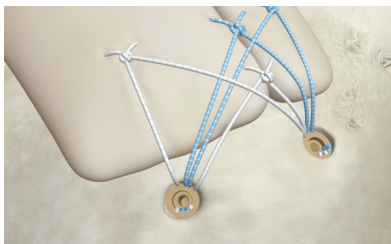

* Reference P/N 52553-01 and 47901-02. The results of in vitro simulation testing have not been proven to predict clinical performance. Test results pertain only to the two suture testing



FIRSTPASS[®] ST Suture Passer

The FIRSTPASS ST Suture Passer is a single-use suture passer pre-loaded with a needle and sterile packed allowing for immediate out-of-the-box use.




Two step trigger to grasp tissue and deploy needle



Round pre-curved needle for reliability and strength

Features and benefits

FEATURE	BENEFITS
Pre-loaded and sterile packed for single case use	<ul style="list-style-type: none"> Eliminates the time and inconvenience associated with sterilizing a reusable suture passer Needle loading and unloading are no longer required No capital cost or instrument maintenance needed
Round, pre-curved nitinol needle	<ul style="list-style-type: none"> Provides added strength and reliability Suture is carried in a recess on the needle which helps maintain suture integrity
Deep bite	Allows for tissue manipulation and suture passage through cuff tissue, up to 10mm in thickness and 18mm in bite depth
Ability to pass MAGNUMWIRE [®] , ULTRABRAID [®] and ULTRATAPE [®] sutures	The surgeon is not limited by suture type and can make product decisions based on the repair needs

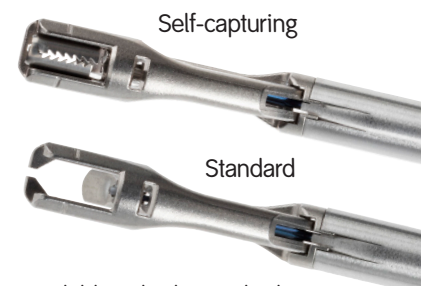

Available in both standard and self-capturing solutions

Ordering information

MULTIFIX[®] S ULTRA Knotless Suture Anchor

Reference #	Description
OM-1300	MULTIFIX P 4.5mm Knotless Suture Anchor
72290001	MULTIFIX S ULTRA 5.5mm Knotless Suture Anchor
72290002	MULTIFIX S ULTRA 6.5mm Knotless Suture Anchor
OM-9420	Bone Punch 3.4mm
OM-9220	Tapered Punch
OM-9615	SPEEDSCREW [®] Insertion Guide and Obturator
22-4038	FIRSTPASS [®] ST Suture Passer, Self-capture
22-4039	FIRSTPASS ST Suture Passer, Standard

Supporting healthcare professionals for over 150 years

ArthroCare Corporation
7000 West William Cannon Drive
Austin, TX 78735
USA

www.smith-nephew.com

Order Entry: 1-800-343-5717
Order Entry Fax: 1-888-994-2782