

+ Scalp Surgery

Get patients back to their daily routines.
Faster.

Aid repairs + Improve outcomes

Smith+Nephew

STRAVIX PL[◇]
Lyopreserved
Umbilical Tissue

STRAVIX[◇]
Cryopreserved
Umbilical Tissue



Contact me to learn more about
STRAVIX Tissues

Faster healing. Patient satisfaction.

A versatile barrier for head and scalp surgeries

Scalp wound s/p basal cell carcinoma removal

Case study

60-year-old male with diabetes treated with MOHs Surgery for BCC presented at 6 days with 16 cm x 5 cm x 0.5 cm wound.

Procedure

- Scalp debridement and STRAVIX Tissue applied as a cover, sutured in place. Wound covered with Low-Adherent Surgical Dressing and NPWT for first 2 weeks
- At one-month, split thickness skin graft applied and another week of NPWT.



Outcome

- Patient was pleased with the results and was able to avoid a free flap reconstruction or rotational scalp flap
- No hospital admission required, only two outpatient procedures and local wound care


Scalp reconstruction with STRAVIX Meshed Tissue

Case study

82-year-old male with A fib, hypertension, history of multiple skin cancers and renal cancer presented with melanoma.

Procedure

- Vertex of scalp resected down to the skull, wound measured 8 cm x 8 cm x 0.75 cm. Wide local excision of lesion with sentinel lymph node biopsy
- STRAVIX Meshed Tissue sewn into wound. Nonadherent dressing and NPWT applied over graft for 13-days.

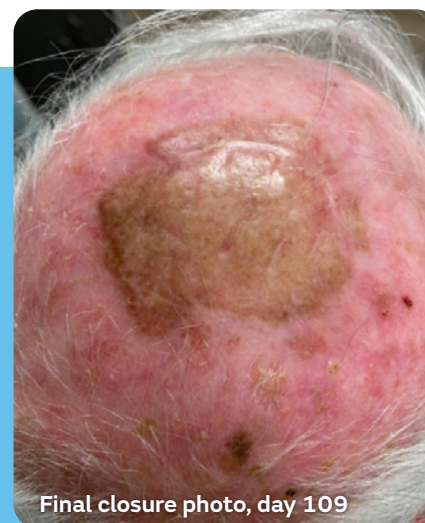

Outcome

- Complete healing in 3.5 months, 109 days
- Patient required 2 outpatient surgeries and 4 post-op clinic visits
- Patient completed adjuvant radiation treatment with no wound breakdown or tissue compromise

Case studies provided by Harishanker Jeyarajan, MD, and Allison W. Seamon, PA-C., Birmingham, AL

Improve outcomes faster, conveniently, affordably

Visit stravixpl.com to view detailed surgical videos and to see clinical studies using STRAVIX PL and STRAVIX Tissues.



Tissue source	Product description	Part #
Lyopreserved umbilical tissue (stored at room temperature)	STRAVIX PL ^o 2 cm x 2 cm (4 cm ²)	PS61022
	STRAVIX PL 2 cm x 4 cm (8 cm ²)	PS61024
	STRAVIX PL 3 cm x 6 cm (18 cm ²)	PS61036
Cryopreserved umbilical tissue (stored between -75°C & -85°C)	STRAVIX Meshed 3 x 6 cm (18 cm ² *)	PS60036
	STRAVIX ^o 2 x 2 cm (4 cm ²)	PS60006
	STRAVIX 2 cm x 4 cm (8 cm ²)	PS60005
	STRAVIX 3 cm x 6 cm (18 cm ²)	PS60008

*PS60036 can be stretched to cover up to 30 cm²

Please see each products' Instructions for Use for indications, contraindications, warnings, precautions and other important safety information.

©2023 Smith+Nephew, Inc. Osiris, STRAVIX, and STRAVIX PL are trademarks of Osiris Therapeutics, Inc., a wholly owned direct subsidiary of Smith+ Nephew Consolidated, Inc. All Trademarks acknowledged. STPE2-38474-0123

Advanced Wound Management
Smith+Nephew, Inc.
Fort Worth, TX 76109 USA

**STRAVIX PL and STRAVIX
Customer Care:**
T 888-674-9551
F 443-283-4419
smith-nephew.com
stravixpl.com